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Digital Jumbo LCD Thermometer

JUMBO DISPLAY  
ILLUMINATED  
DESIGN  
CURVED

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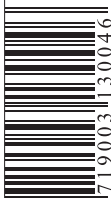
## IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children.
- Therefore never leave the thermometer unattended to children.
- To protect the thermometer from hitting 60 °C, NEVER hold the thermometer!
- Avoid ambient temperatures above 60 °C.
- Use only the commercial disinfectants listed in the section cleaning and disinfecting to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.
- Bistric instruments, not with domestic waste.

• Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

## TURNING ON THE THERMOMETER

• To turn on the thermometer, push the button on the top, a short signalling tone [beep] signals «thermometer ON». A display test is performed. Then at ambient temperature of less than 22 °C, an «-» and a flashing «C» appear at the upper right of the display. The thermometer is now ready for use.

## USING THE THERMOMETER

• Choose the preferred measurement method. During a measurement, the current temperature is displayed continuously. The «C» symbol flashes during this period of time. If the signalling tone sounds (deep-deep-beep 10 times), the «C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 8 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after 10 minutes.

## STORAGE OF MEASURED VALUES

• If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

## ILLUMINATED DISPLAY / NITE GLOW FEATURE

• The thermometer will keep the Nite Glow for 5 seconds when you push the on-off button to turn ON the unit. The display will read «18.8» for the first 2 seconds then automatically display the last temperature for the other 3 seconds. When the power button be pressed Switch-Off the unit, there will be a 4-second additional delay with the Nite Glow ON before power off, which allows user to read the measurement reading easily in the dark.

## TECHNICAL SPECIFICATIONS

Type / Type:	Maximum thermometer
Measurement range / Etendue de mesure:	Temp. < 42.9 °C, display «---» for low (too low) Temp. > 42.9 °C, display «HH» for high (too high)
Measurement accuracy / Précision:	± 0.1 °C between 35 °C and 42 °C ± 0.1 °C to 28 °C
Operating temperature / Temp. de fonctionnement:	Automatic internal check at test value of 37 °C. If there is a deviation of > 0.1 °C, «Err» (error) is displayed
Self-test / Dispositif d'autocontrôle:	Liquid crystal display (LCD) with 3 digits. Smallest unit of display: 0.1 °C.
Display / Affichage:	For signalling that the thermometer is ready to use and that the temperature increase is less than 0.1 °C / 8 seconds.
Signalling tone / Signal sonore:	For storing the last measured value.
Memory / Mémoire:	Storage temperature / Température de stockage: -10 °C to +80 °C
Battery / Pile:	EN12470-3-2000, clinical thermometers, ASTM E-1112, electronic thermometers
Reference to standards / Référence aux normes:	EN12470-3-2000, thermomètres médicaux, ASTM E-1112, électronique thermomètres

Type / Type:	Maximum thermometer
Measurement range / Etendue de mesure:	Temp. < 42.9 °C, display «---» for low (too low) Temp. > 42.9 °C, display «HH» for high (too high)
Measurement accuracy / Précision:	± 0.1 °C between 35 °C and 42 °C ± 0.1 °C to 28 °C
Operating temperature / Temp. de fonctionnement:	Automatic internal check at test value of 37 °C. If there is a deviation of > 0.1 °C, «Err» (error) is displayed
Self-test / Dispositif d'autocontrôle:	Liquid crystal display (LCD) with 3 digits. Smallest unit of display: 0.1 °C.
Display / Affichage:	For signalling that the thermometer is ready to use and that the temperature increase is less than 0.1 °C / 8 seconds.
Memory / Mémoire:	For storing the last measured value.
Battery / Pile:	Storage temperature / Température de stockage: -10 °C to +80 °C
Reference to standards / Référence aux normes:	EN12470-3-2000, clinical thermometers, ASTM E-1112, electronic thermometers
Reference to standards / Référence aux normes:	EN12470-3-2000, thermomètres médicaux, ASTM E-1112, électronique thermomètres

**GUARANTEE**  
We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling or other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your device in the original packaging and send with sufficient postage to:  
Date of purchase:

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