**lifetime
guarantee**

Digital Pacifier Thermometer

100 %
WATERPROOF
**CHILD FRIENDLY
DESIGN**
1 DATA
MEMORY

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MEMORY / MEMOIRE / MEMORIA / MEMÓRIA

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display.

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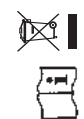
DESIGN

IMPORTANT SAFETY INSTRUCTIONS

- The pacifier thermometer is suitable exclusively for oral measurement of the body temperature of children up to three years of age!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The readings displayed by the pacifier thermometer may be too low under some conditions (e.g. if the child is crying or persistently breathing through his mouth). Therefore, in case of doubt, or if there are signs of fever (reddening of skin, sweating, agitation, unwillingness to drink), please always perform a follow-up rectal measurement if the result is more than 38.5 °C.
- This thermometer is not suitable for use as a pacifier!
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- The thermometer can/must not be opened! The batteries cannot be changed for safety reasons.
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- It is not necessary to calibrate the measurement accuracy within the limited 3-years battery life time.

 Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «C» appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

The pacifier thermometer is suitable only for oral measurement of the body temperature of children, and it must not be used as a pacifier. Place the thermometer in the child's mouth like a normal baby pacifier. Be sure the child keeps the thermometer in his mouth during the measurement, and keep the child's mouth closed as much as possible.

The measurement time signalled is normally about 5 minutes. During a measurement, the current temperature is displayed continuously. The «C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «C» is no longer flashing, this means that the maximum increase in temperature is now less than 0.1 °C in 32 seconds and that the thermometer is ready for reading.

Warning! Certain external factors can lead to a certain measurement inaccuracy in any measurement of body temperature, especially on small children. Therefore, please note the instruction given in the section «Safety instruction». After a measurement, the maximum temperature measured is shown on the display until the thermometer shuts off. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

TECHNICAL SPECIFICATIONS	
Type / Type:	Maximum thermometer
Pacifier / Tétrele:	Silicone rubber
Measurement range / Etendue de mesure:	32.0 °C to 43.9 °C Temp. < 32.0 °C display «L» for low (too low) Temp. > 43.9 °C display «H» for high (too high)
Measurement accuracy / Precision:	± 0.1 °C between 34 °C and 42 °C
Operating temperature / Température de fonctionnement:	18 °C to 28 °C
Self-test / Autocontrôle:	Automatic internal check at a test value of 37 °C (± 0.5 deviation of > 0.1 °C, «ERR» error) is displayed
Display / Affichage:	Liquid crystal display (LCD) with 3 digits Smallest unit of display: 0.1 °C
Signalling tone / Signal sonore:	For signalling that the thermometer is ready to use and that the temperature increase is less than 0.1 °C / 32 seconds
Memory / Mémoire:	For storing the last measured value
Storage temperature / Température de stockage:	-10 °C to +60 °C
Battery / Pile:	1.5 / 1.55 V, SR 41
Battery replacement / Changement de pile:	Impossible for safety reasons
Reference to standards / Référence aux normes:	EN 12470-3:2000, clinical thermometers ASTM E-1112, electronic thermometers

GUARANTEE

We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. Batteries and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage.

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