**Description of this Thermometer**

**EN**

- ① ON/OFF Button
- ② Display
- ③ Battery Compartment Cover
- ④ Measuring Sensor

**Important Safety Instructions**

- The instrument may be used only for measuring body temperature!
- The minimum measurement time until the beep is heard must be maintained without exception!
- Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed.
- Protect the instrument from impact and dropping!
- Avoid ambient temperatures above 60 °C. NEVER boil the instrument!
- Use only the commercial disinfectants listed in the section «Cleaning and Disinfecting» to clean the instrument. The instrument must be intact when immersed in liquid disinfectant.
- We recommend this instrument is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact Microlife-Service to arrange the test.

**⚠ WARNING:** Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

**Turning on the Thermometer**

To turn on the thermometer, press the ON/OFF button ①; a short beep signals «thermometer ON». A display test is performed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the display field ②. The thermometer is now ready for use.

**Function Test**

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case please contact the nearest Microlife service center.

**Using the Thermometer**

When taking a measurement, the current temperature is continuously displayed and the «°C» symbol flashes. If the beep is heard 10 times and the «°C» is no longer flashing, this means that the measured increase in temperature is less than 0.1 °C in 16 seconds and that the thermometer can now be read.

The temperature can be read after the time, necessary for measuring of the temperature by the chosen method.

To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button ①. Otherwise the thermometer will automatically turn off after about 10 minutes.

**Storage of Measured Values**

If the ON/OFF button ① is held down for more than 3 seconds when turning on the thermometer, the maximum temperature automatically stored during the last measurement will be displayed. At the same time, a «M» for memory will appear at the display field. About 2 seconds after the button is released, the temperature value disappears and the thermometer is ready to take a measurement.

**Measuring Methods**

**► In the mouth (oral)**

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor ④ must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

Approx. measuring time: 1 minute!

**► In the anus (rectal)**

This is the most reliable measuring method, and is especially suitable for infants and small children. Carefully insert the measuring sensor ④ of the thermometer 2 to 3 cm into the anal aperture.

Approx. measuring time: 1 minute!

**► In the armpit (axillary)**

To receive more reliable results we recommend to measure temperature orally or rectally.

Approx. measuring time axillary: 5 to 10 minutes!

**Cleaning and Disinfecting**

Name: Isopropyl alcohol 70 %; immerse: max. 24 hours.

**Battery Replacement**

When the «▼» symbol (upside-down triangle) appears at the display field, the battery is flat and needs replacing. Batteries can be purchased at any electrical store. Make sure you have a battery of the same type to hand. To replace the battery remove the battery compartment cover ③ from the thermometer. Insert the new battery with the + at the top.

**Technical Specifications**

<b>Type:</b>	Maximum thermometer
<b>Measurement range:</b>	32.0 °C to 42.9 °C Temp. < 32.0 °C: display «L» for low (too low) Temp. > 42.9 °C: display «H» for high (too high)
<b>Measurement accuracy:</b>	± 0.1 °C between 34 °C and 42 °C at an ambient temperature of 18 °C to 28 °C
<b>Storage temperature:</b>	-10 °C to +60 °C
<b>Battery:</b>	1.5/1.55 V
<b>Reference to standards:</b>	EN12470-3: 2000, clinical thermometers ASTM standard

**Guarantee**

We grant you 3 year warranty after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. Batteries and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please contact the nearest Microlife service center.

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