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Measuring sensor  
Embout thermosensible

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### IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60°C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- **Warning:** The thermometer contains a long-life battery. For disposal, please comply with local regulations for special waste.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

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FR

### TURNING ON THE THERMOMETER

To turn on the thermometer, push the button on the top; a short signalling tone [beep] signals «thermometer ON.» A display test is performed. Then at an ambient temperature of less than 32°C, an «-->» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

### FUNCTION TEST

Proper functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

### USING THE THERMOMETER

The thermometer is suitable only for rectal measurement of body temperature.

During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1°C in 8 seconds and that the thermometer is ready for reading.

**Warning!** To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

### STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. About 2 seconds after release the button, this value will disappear and the thermometer will be turned off.

### MEASURING METHODS

#### In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

Approx. measuring time: 20 seconds.

#### In the anus (rectal)

This is the most reliable measuring method, and is especially appropriate for infants and small children. Carefully insert the tip of the thermometer 2 to 3 cm into the anal aperture.

Approx. measuring time: 10 seconds!

#### In the armpit (axillary)

From a medical viewpoint, this method will always provide inaccurate readings, and should not be used if precise measurements are required.

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### CLEANING AND DISINFECTING

Name	Dilution with water	Immerse
Isopropyl alcohol 70%	95.0 %	max. 24 hours

### WARRANTY

We grant you a lifetime warranty after the date of purchase. Any damage caused by improper handling shall not be covered by the warranty. Batteries and packaging are also excluded from the warranty. All other damage claims excluded. A warranty claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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