
microlife[®] MT 1931

Instruction Manual

FLEXIBLE TIP FEVER THERMOMETER

IB MT 1931 Ver1 1110

DESCRIPTION OF THIS THERMOMETER DESCRIPTION DU THERMOMETRE

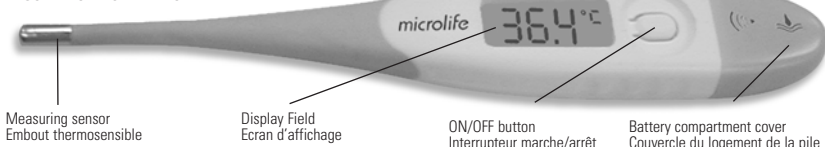Measuring sensor
Embout thermosensible

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Battery compartment cover
Couvercle du logement de la pile

IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid bending the tip more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

EN

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

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TURNING ON THE THERMOMETER

To turn on the thermometer, push the button on the top; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» appears and a «°C» flashes at the upper right of the display field. The thermometer is now ready for use.

FUNCTION TEST

Proper functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 16 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

MEASURING METHODS

In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/ exhaled air.

Approx. measuring time: 60 seconds.

In the anus (rectal)

This is the most reliable measuring method, and is especially appropriate for infants and small children. Carefully insert the tip of the thermometer 2 to 3 cm into the anal aperture. Approx. measuring time: 60 seconds.

In the armpit (axillary)

To receive more reliable results we recommend to measure temperature orally or rectally. A minimum measurement time of 3-5 minutes is recommended regardless of the beep sound. Approx. measuring time: 60 seconds.

CLEANING AND DISINFECTING

Name: Isopropyl alcohol 70%; Immersion: max. 24 hours.

BATTERY REPLACEMENT

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions».

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This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

GUARANTEE

We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. Batteries and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:
Date of purchase:

Read the instructions carefully before using this device.
Veuillez lire attentivement les instructions avant d'utiliser ce produit.
Lea atentamente las instrucciones antes de usar este dispositivo.
Leia atentamente este manual de instruções antes de utilizar.

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Microlife AG