microlife[®]

www.microlife.com
Email: ms@microlifus.com
Tel: +1 727 451 0494
Dunnell, USA
424 Skinner Blvd, Suite C
MicroLife USA, Inc.
North / Central / South America

www.microlife.com
Email: service@microlife.com
Fax: 866 2 8737 2823
Tel: 866 2 8737 2828
Tape: 1 14 Taiwan, R.O.C
9F, 421, Mulung Road, NaitHu
MicroLife Corporation
www.microlife.com
Email: adm@microlife.ch
Fax: +41 71 727 70 39
Tel: +41 71 727 70 30
g4c3 M6001-1-2, CE0434
Espacepress 139
MicroLife AG
Europe / Middle-East / Africa

Distributor


TR
GR
AR
FR
ES
PT

MicroLife NEB 100

NEB 100 microlife

IB NEB 100 V8 3910

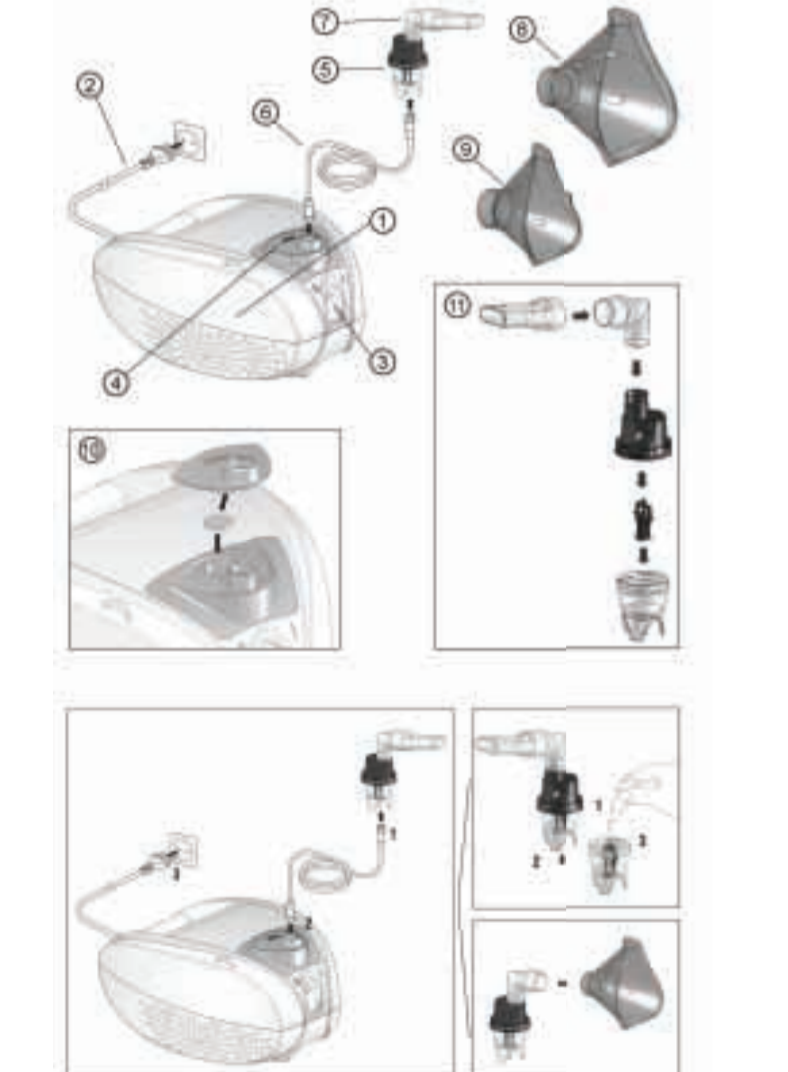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CE 0434

Professional Piston Compressor Nebuliser EN

- Piston compressor
- Power cord
- ON/OFF Switch
- Air filter compartment
- Nebuliser
- Air hose
- Mouthpiece
- Adult face mask
- Child face mask
- Replacing air filter
- Assembling nebuliser kit

Dear Customer,
This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.
The device is capable of generating a medicinal mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts, please contact MicroLife-Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the Internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – MicroLife AG!

Read the instructions carefully before using this device.

Type B applied part

- Important Safety Instructions**
 - This device may only be used for the purposes described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application.
 - This device is not suitable for anaesthesia and lung ventilation.
 - This device should only be used with original accessories as shown in these instructions.
 - Do not use the device if you think it is damaged or notice anything unusual.
 - Never open the device.
 - This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
 - Protect it from
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
 - Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
 - The device and all accessories should only be used by one person.
 - Never bend the nebuliser over 60°.
 - Ensure that children do not use the device (unsupervised; some parts are small enough to be swallowed).
 - Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
 - Use of this device is not intended as a substitute for consultation with your physician.

2. Preparation and Usage of this Device
Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nebuliser kit . Ensure that all parts are complete.
- Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser with the air hose to the compressor and plug the power cord to the socket (230V - / 50Hz).
- Switch the ON/OFF switch to position «I» to turn on the device and place the mouthpiece in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult or child face mask and make sure that it encloses the mouth and nose area completely.
- Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
- After completing the inhalation period recommended by your doctor, switch the ON/OFF switch to position «O» to turn off the device and unplug it from the socket.
- Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

- Cleaning and Disinfecting**
Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor and the air hose should be cleaned with a clean, moist cloth.
 - Always wash your hands well before cleaning and disinfecting the accessories.
 - Do not expose the compressor to water or heat.
 - Replace the air hose for each treatment with a new patient or in case of impurities.
 - Unplug the device before cleaning.

- Cleaning with water**
- Wash all nebuliser components (except air hose) under warm tap water (max 60 °C) for about 5 minutes** adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.
 - Rinse thoroughly making sure that all deposits are washed away and leave to dry.

- Disinfecting**
- All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.
 - Disinfectants are usually available at pharmacies.

- Sterilizing with steam**
- All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min) or 134 °C (7 min.). EN554/ISO11134.
 - The sterilisation packaging must conform to EN868/ISO11607 and be suitable for steam sterilisation.
 - After sterilisation always let all components cool down to ambient temperature before further use.
 - Do not repeat sterilization cycle when components are still warm.

- Maintenance, Care, and Service**
Order all spare parts from your dealer or pharmacist, or contact MicroLife-Service (see forward).
- We recommend replacing the nebuliser, mouthpiece, and the face masks after 3 months use.

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