Professional Compressor Nebuliser

1 Piston compressor
2 Power lead
3 ON/OFF Switch
4 Air filter compartment
5 Nebuliser «NEB 2 in 1»
-A: Nasal washer
-B: Nebuliser for aerosol therapy

6 Air hose
7 Mouthpiece with valve
8 Adult face mask
9 Child face mask
10 Nose piece
11 Reaction air filter
12 Assembling the nasal washer
13 Assembling the nebuliser for aerosol therapy
14 Filling the nebuliser
15 Operating diagram of «NEB 2 in 1»

Dear Customer,
This professional nebuliser is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.
Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacist will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – MicroLife AG!

Read the instructions carefully before using this device.
Type BF applied part
IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never use the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nasal washer (2) or the nebuliser kit (3). Ensure that all parts are complete.
- Fill the nebuliser (4) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (23V ~ / 50Hz).
- Switch the ON/OFF switch (3) to position «1» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.
 - Push and hold button (13-A) for nebulisation, and then release the button to stop nebulisation.
- Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

3. NEB 2 in 1 - Nasal washer

This device is equipped with the advanced «NEB 2 in 1» nasal washer for a complete, quick and effective treatment of upper airways (allergic and nonallergic rhinitis, rhinosinusitis, nasal polyps etc.).

- Fill with medication or a saline solution (14) and connect the nasal washer top by rotating clockwise.
- Close the non-treated nostril with your finger.
- Push and hold button (13-A) for nebulisation, and then release the button to stop nebulisation.
- Keep a handkerchief within reach to dry any fluid which is not collected in the tank (13-A).
- We recommend changing from one nostril to the other every 10 seconds.
- Children and people in need of care must be assisted for usage.

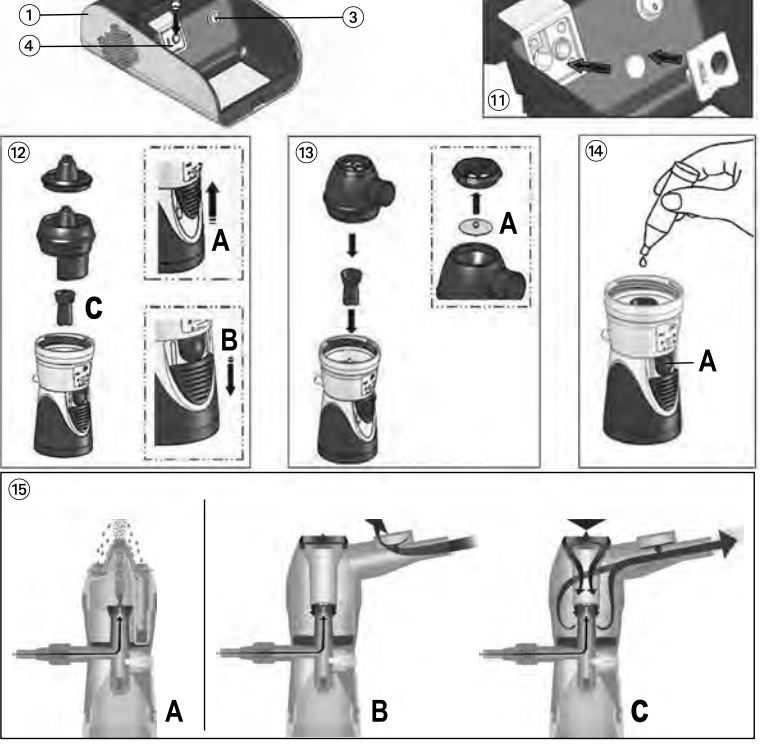
4. NEB 2 in 1 - Aerosol therapy

The advanced «NEB 2 in 1» aerosol therapy allows effective treatments of the medium and lower respiratory tract (asthma, chronic bronchitis, cystic fibrosis etc.).

Advanced treatment
This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (3-C), reducing medication loss during exhalation (3-B). Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.

- Mount the membrane on the valve-holder (3-A) and use the mouthpiece with expiratory valve.
- Fill with medication (14) and connect the nebuliser top to the nebuliser bottom by rotating it clockwise.
- Follow the usage steps as described in «Section 2».
- Only using the mouthpiece with expiratory valve (2) enables the respiratory functioning of the nebuliser.

Short treatment
The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.
A synchronized nebulisation is not possible using this treatment option.

Guarantee Card

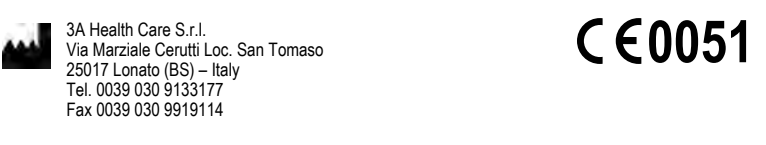
Name of Purchaser / Inköpörens namn / Ostajain nimi / Forhandlers navn / Kjøpers navn / Pircibã värds / Pirkjøp parvards / Ostaj nimi / Ф.И.О. покупателя

Serial Number / Serienummer / Sarjanyumuri / Serienumer / Serienummer / Sarjanyumuri / Серийный номер

Date of Purchase / Inköpsdatum / Ostopäivämäärä / Købsdato / Kjøpsdato / legãdes datums / Pardavimo data / Ostukuupäev / Дата покупки

Special Dealer / Återförsäljare / Alan kauppias / Spezial-forhandler / Specialiat forhandler / Specialists -pärstãvis / Pardavusi įstaiga / Ametik müügiesindaja / Спецпункт/экспертный дилер

3A Health Care S.r.l.
Via Marziale Cerutti Loc. San Tomaso
23017 Lonato (BS) – Italy
Tel. 0039 030 913177
Fax 0039 030 991914

Professional Compressor Nebuliser

1 Piston compressor
2 Power lead
3 ON/OFF Switch
4 Air filter compartment
5 Nebuliser «NEB 2 in 1»
-A: Nasal washer
-B: Nebuliser for aerosol therapy

6 Air hose
7 Mouthpiece with valve
8 Adult face mask
9 Child face mask
10 Nose piece
11 Reaction air filter
12 Assembling the nasal washer
13 Assembling the nebuliser for aerosol therapy
14 Filling the nebuliser
15 Operating diagram of «NEB 2 in 1»

Dear Customer,
This professional nebuliser is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.
Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacist will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – MicroLife AG!

Read the instructions carefully before using this device.
Type BF applied part
IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never use the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nasal washer (2) or the nebuliser kit (3). Ensure that all parts are complete.
- Fill the nebuliser (4) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (23V ~ / 50Hz).
- Switch the ON/OFF switch (3) to position «1» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.
 - Push and hold button (13-A) for nebulisation, and then release the button to stop nebulisation.
- Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

3. NEB 2 in 1 - Nasal washer

This device is equipped with the advanced «NEB 2 in 1» nasal washer for a complete, quick and effective treatment of upper airways (allergic and nonallergic rhinitis, rhinosinusitis, nasal polyps etc.).

- Fill with medication or a saline solution (14) and connect the nasal washer top by rotating clockwise.
- Close the non-treated nostril with your finger.
- Push and hold button (13-A) for nebulisation, and then release the button to stop nebulisation.
- Keep a handkerchief within reach to dry any fluid which is not collected in the tank (13-A).
- We recommend changing from one nostril to the other every 10 seconds.
- Children and people in need of care must be assisted for usage.

4. NEB 2 in 1 - Aerosol therapy

The advanced «NEB 2 in 1» aerosol therapy allows effective treatments of the medium and lower respiratory tract (asthma, chronic bronchitis, cystic fibrosis etc.).

Advanced treatment
This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (3-C), reducing medication loss during exhalation (3-B). Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.

- Mount the membrane on the valve-holder (3-A) and use the mouthpiece with expiratory valve.
- Fill with medication (14) and connect the nebuliser top to the nebuliser bottom by rotating it clockwise.
- Follow the usage steps as described in «Section 2».
- Only using the mouthpiece with expiratory valve (2) enables the respiratory functioning of the nebuliser.

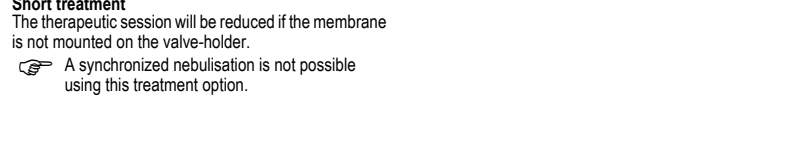
Short treatment
The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.
A synchronized nebulisation is not possible using this treatment option.

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