Description of this Thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor / measuring tip
- ⑤ Cleaning and disinfecting area (thermometer probe only)

EN

Cleaning and disinfecting

For disinfection in home use environment, use a 70% Isopropyl alcohol swab, or a cotton tissue moistened with 70% isopropyl alcohol to wipe surface pollutants off the thermometer probe (note: consider the application and safety instruction of the disinfectant manufacturer). Always start wiping from the end of the thermometer probe (approx. at the middle of the thermometer) towards the thermometer tip. Afterwards the entire thermometer probe (see number ⑤ in the drawing) should be immersed in 70% isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use. Avoid immersing or wiping the display to protect it from fading.

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Battery Replacement
When the  symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover (③) from the thermometer. Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

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Approx. measuring time: 1 minute!

In the anus (rectal) / 36.6 - 37.3 °C

Attention: For the prevention of rectal perforation in children (younger than 3 years), we recommend using another measuring method, or the use of a thermometer with flexible tip.

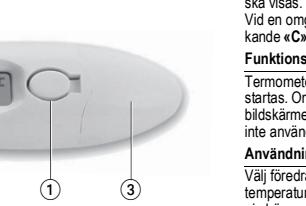
Carefully insert the measuring sensor (④) of the thermometer 2 to 3 cm into the anal aperture.

The use of a probe cover and the use of a lubricant is recommended.

If you are unsure of this measurement method, you should consult a professional for guidance/training.

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Mättid cirka 1 minut.

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För desinfektion i hemmiljö, använd en 70% Isopropylalkohol swab, or a cotton tissue moistened with 70% isopropyl alcohol to wipe surface pollutants off the thermometer probe (note: consider the application and safety instruction of the disinfectant manufacturer). Always start wiping from the end of the thermometer probe (approx. at the middle of the thermometer) towards the thermometer tip. Afterwards the entire thermometer probe (see number ⑤ in the drawing) should be immersed in 70% isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use. Avoid immersing or wiping the display to protect it from fading.

The thermometer is not intended for professional use.

Battery Replacement

When the  symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover (③) from the thermometer. Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

Technical Specifications

Type:	Maximum thermometer
Measurement range:	34.0 °C till 43.9 °C Temp. < 32.0 °C: display «L» lav (for lav) Temp. > 43.9 °C: display «H» hög (for hög)
Measurement accuracy:	± 0.1 °C, 34 - 42 °C
Operating conditions:	± 0.2 °C, 32.0 - 33.9 °C och 42.1 - 42.9 °C 10 - 40 °C, 15 - 95 % relative maximum humidity
Storage conditions:	-25 - +60 °C, 15 - 95 % relative maximum humidity

This device complies with the requirements of the Medical Device Directive 93/42/EEC.
Technical alterations reserved.

Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Battery.

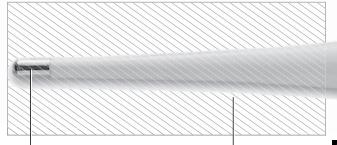
Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

Function Test

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccurate), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

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