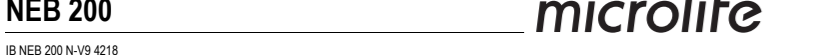
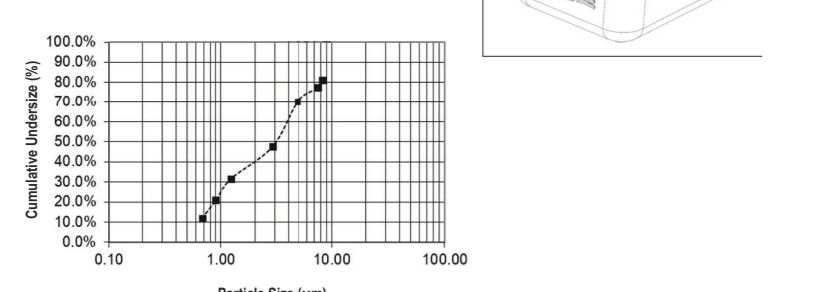
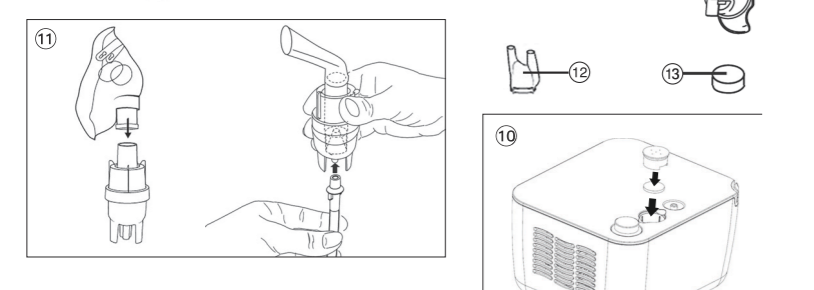
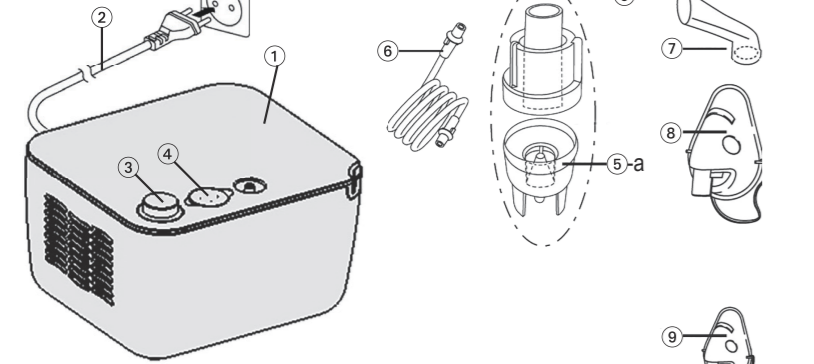
**NEB 200**

IB NEB 200 N-V 4/216

**Guarantee Card**

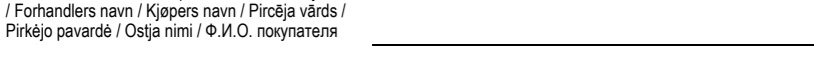
Name of Purchaser / Inköpares namn / Ostajan nimi / Forhandlers navn / Kjöpers navn / Pirçaja vards / Pirkejo varda / Ostja nimi / Ф.И.О. покупателя

Serial Number / Serienummer / Sarjanumero / Serienumer / Serienummer / Sériyas numari / Series numbers / Serianumber / Серийный номер

Date of Purchase / Inköpsdatum / Ostoväpämäärä / Keskbatlo / Kjøpsdato / legdes datums / Parzivimo data / Ostokupäev / Дата покупки

Specialist Dealer / Återförsäljare / Alan kauppias / Special-forhandler / Spetsialist forhandler / Spetsialist -pārdevējs / Parvadaus īpašs / Ametlik müügiesindaja / Спецализированный дилер

Globalcare Medical Technology Co., Ltd  
7th Building, 39 Middle Industrial Main Road, European Industrial Zone, Xiaolan Town, 528415 Zhongshan City, Guangdong Province, PRC

1. Piston compressor  
2. Power lead  
3. ON/OFF Switch  
4. Air filter compartment  
5. Nebuliser  
-a: Vapouriser head  
6. Air hose  
7. Mouthpiece  
8. Adult face mask  
9. Child face mask  
10. Replacing air filter  
11. Assembling nebuliser kit  
12. Nose piece  
13. Air filter

Dear Customer,  
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Stay healthy – Microlife AG!

**1. Explanation of symbols**

⚠ Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

📖 Read the instructions carefully before using this device.

👤 Type BF applied part

📦 Class II equipment

SN Serial number

REF Reference number

🏠 Manufacturer

I ON  
O OFF

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

**CE 0123** CE Marking of Conformity

**2. Important Safety Instructions**

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mixture inflammable with oxygen or nitrogen protoxide. This device is not suitable for anaesthesia and lung ventilation.

This device should only be used with original accessories as shown in these instructions.

Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.

Protect it from:  
- water and moisture  
- extreme temperatures  
- impact and dropping  
- contamination and dust  
- direct sunlight  
- heat and cold

Comply with the safety regulations concerning the electrical devices and in particular:  
- Never touch the device with wet or moist hands.  
- Place the device on a stable and horizontal surface during its operation.  
Do not put the power cord or the device itself to unplug it from the power socket.

The power plug is a separate element from the grid power; keep the plug accessible when the device is in use.

Before plugging in the device, make sure that the electrical rating, shown on the rating plate at the bottom of the unit, corresponds to the mains rating.

In case the power plug provided with the device does not fit your wall socket, contact qualified personnel for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.

Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operating.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, for which the manufacturer cannot be held responsible.

Do not replace the power lead of this device. In case of a power lead damage, contact a technical service center authorized by the manufacturer for its replacement.

The power supply cord should always be fully unwound in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.

Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same accessories for more than one person.

Never bend the nebuliser over 60°.

Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

**3. Preparation and Usage of this Device**

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

1. Assemble the nebuliser kit (1). Ensure that all parts are complete.

2. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.

3. Connect the nebuliser with the air hose (2) to the compressor (3) and plug the power lead (2) into the socket (23V 50 Hz AC).

4. To start the treatment, set ON/OFF switch (3) into the «I» position.

**Compressor Nebuliser**

- 1. Piston compressor
- 2. Power lead
- 3. ON/OFF Switch
- 4. Air filter compartment
- 5. Nebuliser
- a: Vapouriser head
- 6. Air hose
- 7. Mouthpiece
- 8. Adult face mask
- 9. Child face mask
- 10. Replacing air filter
- 11. Assembling nebuliser kit
- 12. Nose piece
- 13. Air filter

Dear Customer,  
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Stay healthy – Microlife AG!

- 1. Explanation of symbols**
- ⚠ Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- 📖 Read the instructions carefully before using this device.
- 👤 Type BF applied part
- 📦 Class II equipment
- SN Serial number
- REF Reference number
- 🏠 Manufacturer
- I ON  
O OFF
- IP21 Protection against solid foreign objects and harmful effects due to the ingress of water
- CE 0123** CE Marking of Conformity
- 2. Important Safety Instructions**
- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mixture inflammable with oxygen or nitrogen protoxide. This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:  
- water and moisture  
- extreme temperatures  
- impact and dropping  
- contamination and dust  
- direct sunlight  
- heat and cold
- Comply with the safety regulations concerning the electrical devices and in particular:  
- Never touch the device with wet or moist hands.  
- Place the device on a stable and horizontal surface during its operation.  
Do not put the power cord or the device itself to unplug it from the power socket.
- The power plug is a separate element from the grid power; keep the plug accessible when the device is in use.
- Before plugging in the device, make sure that the electrical rating, shown on the rating plate at the bottom of the unit, corresponds to the mains rating.
- In case the power plug provided with the device does not fit your wall socket, contact qualified personnel for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.
- Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operating.
- The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, for which the manufacturer cannot be held responsible.
- Do not replace the power lead of this device. In case of a power lead damage, contact a technical service center authorized by the manufacturer for its replacement.
- The power supply cord should always be fully unwound in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Depending on the pathology, only use the treatment that is recommended by your doctor.
- Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.
- Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.
- Do not position the equipment so that it is difficult to operate the disconnection device.
- For greater hygienic safety, do not use the same accessories for more than one person.
- Never bend the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Use of this device is not intended as a substitute for a consultation with your doctor.
- 3. Preparation and Usage of this Device**
- Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».
1. Assemble the nebuliser kit (1). Ensure that all parts are complete.
2. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
3. Connect the nebuliser with the air hose (2) to the compressor (3) and plug the power lead (2) into the socket (23V 50 Hz AC).
4. To start the treatment, set ON/OFF switch (3) into the «I» position.

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