

# microlife®

A partner for people. For life.

Let's work  
together for  
better health.



Microlife **WatchBP**

## Professional blood pressure monitors

---

The professional cardiovascular management solution for better diagnosis of heart diseases and stroke risks

# Table of contents

---

## Introduction

WatchBP – The Cardiovascular Management Solution	1
Advantages of WatchBP	
Accuracy and Validations	3
Special Patient Validations	4
Early Detection and Prevention	7
Advanced Measurement Technology	8
WatchBP Features Overview	10

---

## WatchBP Office

WatchBP Office 2G	14
WatchBP Office Vascular	16

---

## WatchBP 03

WatchBP 03 2G	20
---------------	----

---

## WatchBP Home

WatchBP Home	24
WatchBP Home A	25
WatchBP Home A BT	26

---

## WatchBP Cuffs

WatchBP Cuffs	28
---------------	----

---

## Reference

Reference	38
-----------	----

# WatchBP

The professional cardiovascular management solution for better diagnosis of heart diseases and stroke risks

## Clinical blood pressure measurement has limited accuracy because of:

- ✓ Blood pressure variability
- ✓ White coat effect/hypertension
- ✓ Masked hypertension

## Therefore, guidelines recommend:

- ✓ Ambulatory (24-hour) blood pressure measurement
- ✓ Self-measurement of blood pressure at home
- ✓ Unattended automated office blood pressure measurement

## Additional cardiovascular screening is also recommended for:

- ✓ Atrial Fibrillation (AF)
- ✓ Peripheral arterial disease
  - Ankle Brachial Index (ABI)
  - Inter-Arm Difference (IAD)
  - Pulse wave velocity (PWV)
- ✓ For special patient groups, WatchBP devices are validated and recommended to measure:
  - Children from the age of 3 years
  - Pregnant women with or without pre-eclampsia
  - Patients with end stage renal disease
  - Diabetes patients
  - Elderly
  - Patients with hypotension
  - Patients with atrial fibrillation

**Microlife WatchBP presents a solution that makes screening for these important cardiovascular risks easy, fast and accurate.**



Recommended by NICE



AFIB Detection  
(Microlife Technology)



Protocol Embedded



Protocol Embedded



Protocol Embedded

## Product overview

### WatchBP Products

#### Office Blood Pressure Measurement



WatchBP Office 2G



WatchBP Office Vascular

#### Ambulatory Blood Pressure Measurement



WatchBP O3

#### Self-Measurement of Blood Pressure at Home



WatchBP Home



WatchBP Home A



WatchBP Home A BT

# Accuracy and validations

## Clinically validated blood pressure monitors

All Microlife WatchBP devices are validated according to the established international protocols.

### Clinical devices

WatchBP Device	Validation protocol		Circumstance
	BIHS	ESH / ISO-AAMI	
Office 2G		Pass	At rest/ L-XL Size cuff /children >3 year
Office Vascular	A/A	Pass	At rest/ L-XL Size cuff /children >3 year

\*All devices are validated for L-XL size cuff and for children from the age of 3 years old.

### Ambulatory blood pressure monitoring devices

WatchBP Device	Validation protocol		Circumstance
	BIHS	ESH / ISO-AAMI	
O3 2G		Pass	At rest/ L-XL Size cuff /children >3 year

### Self-measurement devices

WatchBP Device	Validation protocol		Circumstance
	BIHS	ESH / ISO-AAMI	
Home	A/A	Pass	At Rest
	A/A		Pregnancy
	B/A		Pre-eclampsia
			L-XL Size Cuff
Home A	A/A	Pass	At Rest
		Pass	L-XL Size Cuff
Home A BT (Bluetooth®)	A/A	Pass	At Rest
		Pass	L-XL Size Cuff

BIHS, British and Irish Hypertension Society; ESH, European Society of Hypertension; AAMI, Association for the Advancement of Medical Instrumentation; ISO, International Organization for Standardization.



## Special patient validations

### About special patient validations

Because of the technique that is used in automated oscillometric blood pressure monitors, they can be inaccurate when used in so-called special patient groups. This can lead to the fact that a blood pressure monitor that is validated in “regular subjects” may not give accurate readings when used e.g. during pregnancy or when used in children, elderly or diabetes patients. This can have serious clinical consequences.

Therefore, medical standard authorities require that a blood pressure monitor may only be recommended for such a special patient group if it has been validated for this specific group.

Microlife blood pressure monitors have passed many of these challenging validations so that they now may be recommended for almost all special patient groups.

	Stiff arteries						Very low BP values		Cuff fit		AF
	ESRD	Diabetes	Elderly	Dialysis	Pre-eclampsia	Pregnancy alone	Hypotension	Children & adolescents	Obesity	Cuff wide-range	
Microlife	●	●	●	●	●	●	●	●	●	●	●

\*ESRD, end-stage renal disease; AF, atrial fibrillation

### Accurate measurement for End Stage Renal Disease



Microlife WatchBP is accurate for patients with end-stage renal disease.

Patients with moderate to severe renal disease have a very high incidence of hypertension, paired with stiff (calcified) arteries. As automated measurements can be influenced by stiff arteries, a special validation is required before blood pressure monitors can be recommended for use among patients with end stage renal disease.

#### Available models:

WatchBP Office 2G  
WatchBP Office Vascular

WatchBP O3 2G

WatchBP Home  
WatchBP Home A  
WatchBP Home A BT






Early detection of Atrial Fibrillation can reduce the risk of stroke by 68%.



### What is Atrial Fibrillation (AF)?

AF is the most common sustained cardiac arrhythmia occurring in 5% of the population of 65 years and above and in 14% among those older than 85 years old. AF leads to a 5-fold higher risk of stroke and is responsible for 20% of all strokes. Many people have no symptoms from AF and therefore remain undiagnosed, whereas early treatment can reduce the risk of stroke by up to 68%.

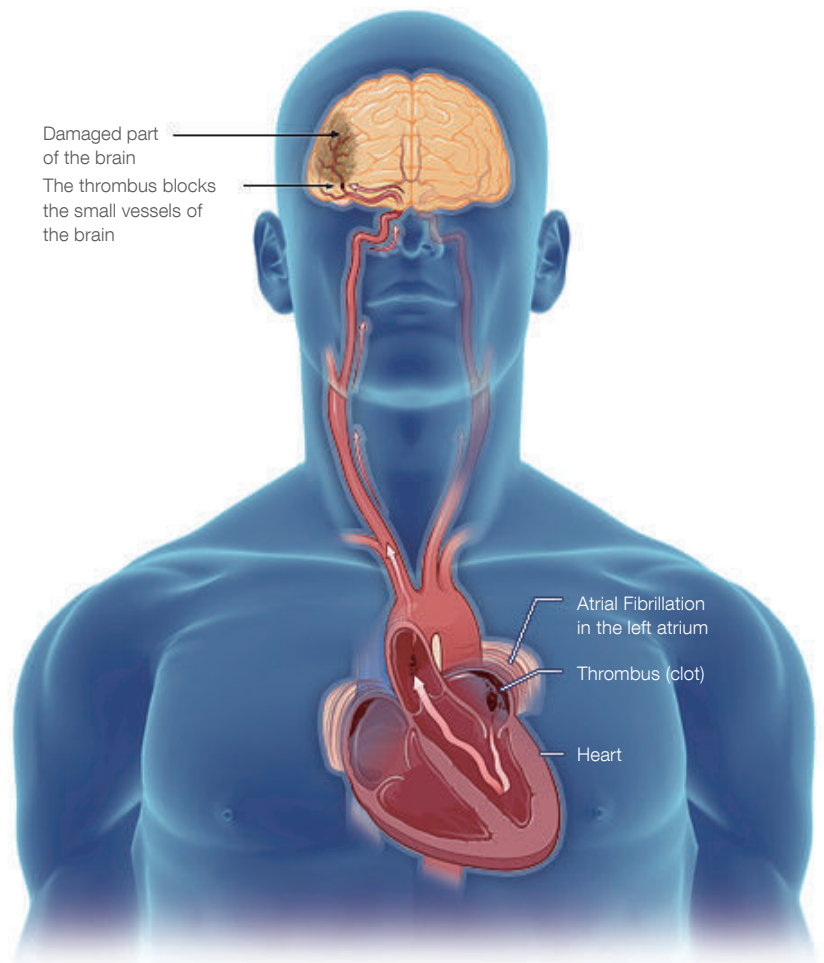

Stroke due to Atrial Fibrillation



# Early detection and prevention

## WatchBP monitors with Atrial Fibrillation (AF) detection system

WatchBP monitors with implemented AF detection system (AFIB) allow patients to be screened for AF during blood pressure measurement. WatchBP AFIB has consistently proven its accuracy and showed that it leads to increased detection of new patients with AF when used in general clinical practice.

## Early detection of AF followed by adequate treatment can reduce the risk of a stroke by up to 68%

### Wrist palpation

Wrist palpation, although commonly performed, has a low reliability.




### Best clinical practice

A symbol appears on the LCD screen when AF is detected.



### “WatchBP Home A should be used for hypertension monitoring in primary care”

Tested and approved for detecting Atrial Fibrillation  
 WatchBP AFIB detects Atrial Fibrillation with high accuracy (sensitivity 98%, specificity 92%) as demonstrated in multiple comparative studies with ECG.

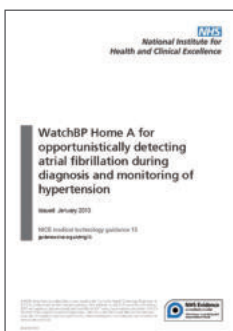

## Recommended by NICE

The National Institute for Health and Care Excellence (NICE) officially recommends using the WatchBP Home A during routine blood pressure measurement for all GPs in the United Kingdom.

[www.nice.org.uk/MTG13](http://www.nice.org.uk/MTG13)


## WatchBP is the only FDA and MDD cleared oscillometric BPM for atrial fibrillation screening

# Advanced measurement technology

## Advanced guideline-based measurements

### The advantages of 3 consecutive measurements

- Superior reliability for blood pressure monitoring.
- Performing three sequential measurements diminishes the influence of an occasional deviating (high) reading.

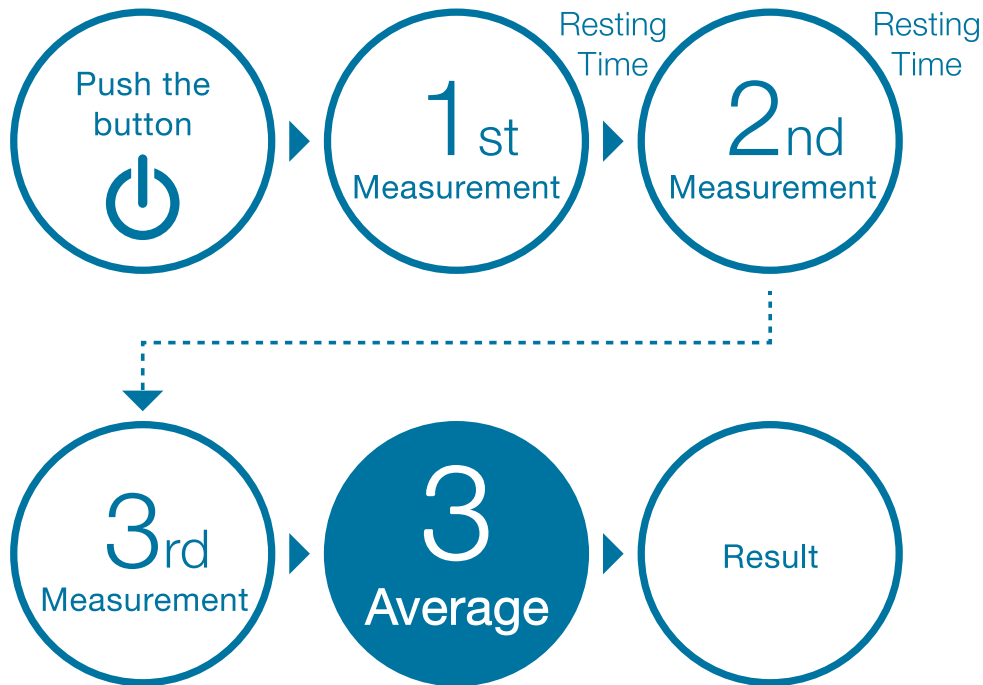

Guidelines recommend taking at least two blood pressure readings each time and averaging the measurements, for a more reliable result.

## Efficient screening for Peripheral Arterial Disease (PAD)

### WatchBP monitors with Ankle Brachial Index (ABI) assessment system


WatchBP Office ABI monitors help physicians to diagnose PAD efficiently by performing ankle-arm blood pressure measurements to assess the Ankle Brachial Index. Accurate and easy to use.



#### What is Peripheral Arterial Disease (PAD)?

PAD is a frequently occurring cardiovascular risk factor that often remains undetected for a long time. In fact, 50% of all patients who have it usually show no symptoms. Patients with PAD have a three-fold higher risk of myocardial infarction, stroke, and death.

### Peripheral Arterial Disease (PAD)

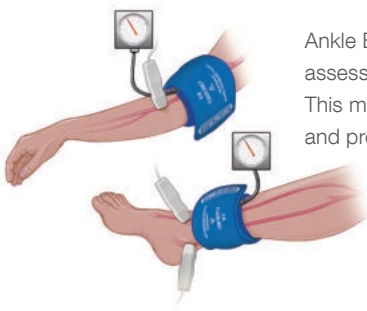

Ankle Brachial Index is commonly assessed with a Doppler device. This method is time consuming and prone to error.



Fatty substances on the arterial wall

WatchBP Office Vascular is easy, fast and accurate for ABI assessment

## Features overview

### WatchBP Office

WatchBP Device	3 Consecutive measurements	Double arm measurement	Auscultatory mode	AFIB detection	ABI assessment	Central BP measurement	AOBPM* (SPRINT algorithm)	PWV Measurement	PC link	Bluetooth® connectivity	Special patient validations			
											End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
Office 2G	●		●	⊙		⊙	●		●	●	●	●	●	●
Office Vascular	⊙	●		●	⊙	⊙	●	⊙	●	●	●	●	●	●

⊙ Optional;

\* Automated Office Blood Pressure Measurement

### WatchBP O3 Ambulatory

WatchBP Device	24-hour ambulatory BP monitoring	AFIB detection	Central BP measurement	Night-time measurement	Pill-button	PC link	Bluetooth® connectivity	End-stage renal disease	Special patient validations		
									Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
O3 2G	●	⊙	⊙	●	●	●	●	●	●	●	●


⊙ Optional;

### WatchBP Home

WatchBP Device	Single measurement	3 Consecutive measurements	AFIB detection	ESH/AHA/BIHS Guidelines embedded	PC link	Bluetooth® connectivity	Special patient validations		
							End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia
Home	●			●	●		●	●	●
Home A		●	●	●	●		●	●	
Home A BT (Bluetooth®)		●	●	●	●	●	●	●	

microlife®

SCREEN

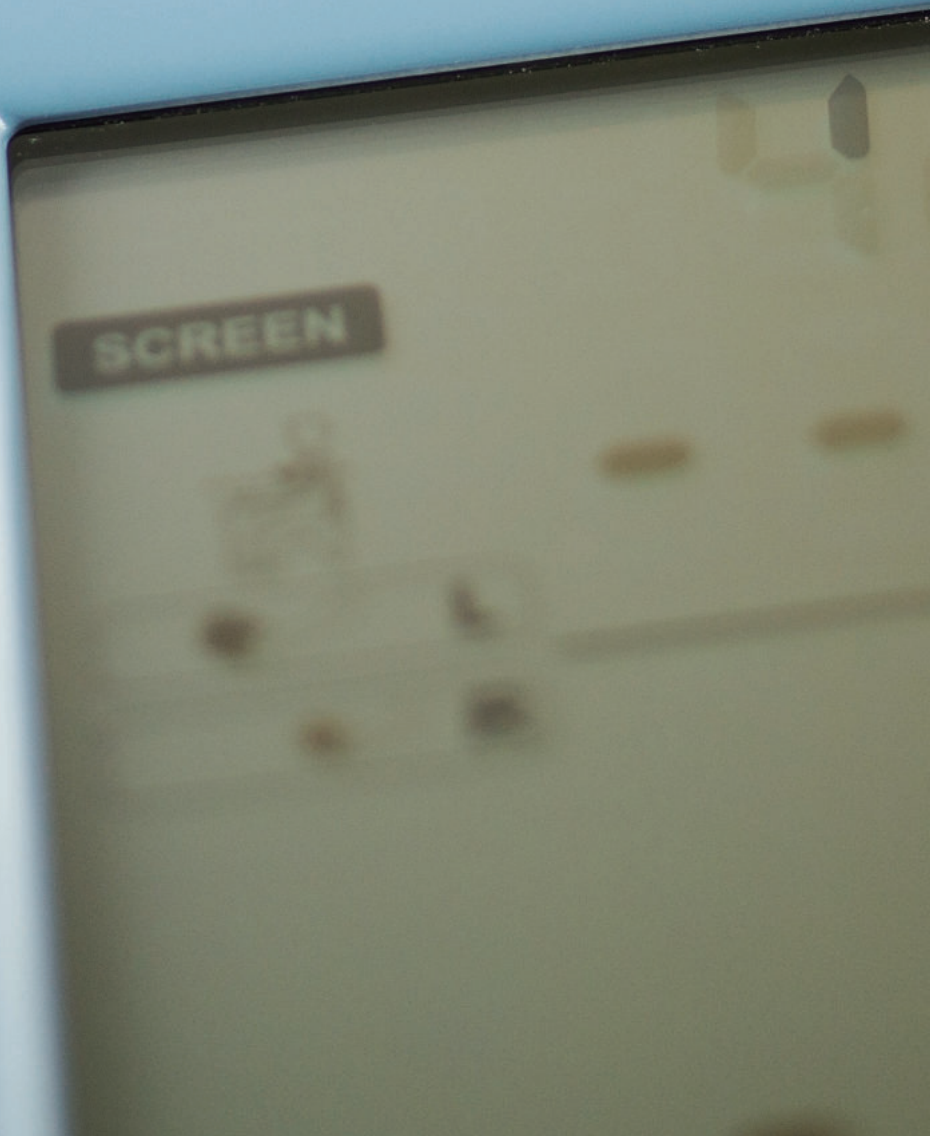



## WatchBP Office

Guideline-Based professional blood pressure monitors



## WatchBP Office



### MicroLife AFIB technology

Screens for Atrial Fibrillation during blood pressure measurement.



### ABI assessment

Fast, easy and reliable simultaneous ankle-arm blood pressure measurement for screening of peripheral arterial disease.




### Double arm measurement

Unique dual-cuff design for assessing inter-arm blood pressure differences.



### Central blood pressure measurement

Non-invasive, reliable method based on brachial pulse volume plethysmography (PVP) waveforms.

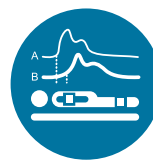

### Brachial-ankle PWV assessment (optional)

User-friendly with high reproducibility cuff-based brachial-ankle PWV measurement method to evaluate arterial stiffness.



### PC link

Transmit all measurement data to PC via USB connection.



### Auscultatory mode

For use in e.g. elderly, obese and those with arrhythmia.



### SPRINT algorithm

Enables Automated Office Blood Pressure Measurement including the "SPRINT Algorithm" (5 min countdown time followed by 3 consecutive measurements with a 1 minute interval between each reading).

Available models



**WatchBP Office 2G**

Accurate automated office blood pressure (AOBP) monitor with flexible measurement schedule options.



**WatchBP Office Vascular**

Accurate blood pressure and cardiovascular screening monitor with multiple features and easy to use.

WatchBP Device	3 Consecutive measurements	Double arm measurement	Auscultatory mode	AFIB detection	ABI assessment	Central BP measurement	AOBPM* (SPRINT Measurement algorithm)	PWV Measurement	PC link	Bluetooth® connectivity	Special patient validations			
											End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
Office (2nd Generation)	●		●	⊙		⊙	●		●	●	●	●	●	●
Office Vascular	⊙	●		●	⊙	⊙	●	⊙	●	●	●	●	●	●

⊙ Optional;


\* Automated Office Blood Pressure Measurement

# WatchBP Office 2G

Professional automated office blood pressure monitor



AFIB

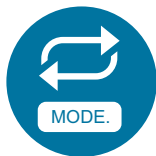

Flexible BPM schedule



Central blood pressure measurement



3 consecutive measurements



SPRINT algorithm



PC link



Auscultatory mode



For use in end stage renal disease



For use in pregnancy




For use in children



For use in diabetes patients



- ✓ Flexible measurement schedule.
- ✓ Possibility to perform (up to 6 cycle measurements) auscultatory blood pressure measurement.
- ✓ Non-invasive central blood pressure measurement (optional).
- ✓ Atrial Fibrillation (AF) detection (optional).
- ✓ Free WatchBP Analyzer software can be downloaded from the website: <https://www.microlife.com/support/software-professional-products>

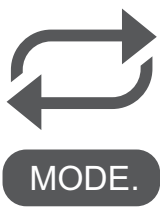

### Flexible BPM schedule

Clinical Blood pressure measurement can be done in many ways. Unfortunately, there is no consensus between different hypertension authorities in how blood pressure should be measured in clinical practice. Recommendations differ with respect to the number of measurements needed from 2 -6 measurements but also for the required interval time between measurements varying from 30 seconds to 5 minutes. Currently, several medical societies also plea for the performances of unattended blood pressure measurement. This means that the healthcare professional must activate the blood pressure monitor before leaving the room so that sequential measurements are taken after a certain countdown period.

To help physicians all over the world adhering to these different measurement recommendations Microlife has developed a professional blood pressure monitor with a flexible measurement schedule. With this unique option the professional is free to select the preferred number of measurements, the interval time between measurement, the count-down period and even the way the average blood pressure value is calculated.



### Central blood pressure

Central blood pressure is the pressure in the aorta, the largest artery that originates from the heart. Central blood pressure measurement may better correlate with cardiovascular risk than upper arm blood pressure measurement because the aorta is closer to the heart and brain, the most vital organs of the body.



### Customisable measurement functions

Customize your ABPM according to your needs with additional, helpful technology features:

- Atrial Fibrillation (AF) detection.
- Central blood pressure measurement.

### Features overview

WatchBP Office	AFIB	Central BP	Flexible BPM schedule	Auscultatory mode	Rechargeable battery	USB connectivity	Bluetooth® connectivity	WatchBP Analyzer	Special patient validations			
									End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
Advanced			●	●	●	●	●	●	●	●	●	●
AFIB	●		●	●	●	●	●	●	●	●	●	●
Central	●	●	●	●	●	●	●	●	●	●	●	●



# WatchBP Office Vascular


Professional automated office blood pressure monitor




Double Arm Measurement



AFIB

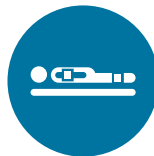


ABI assessment

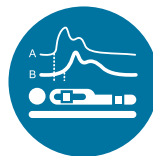

baPWV assessment



Central blood pressure measurement

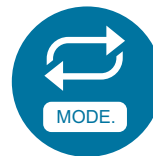

Flexible BPM schedule



SPRINT algorithm



Upgradeable device version



PC link



For use in end stage renal disease



For use in pregnancy



3+ AGES

For use in children



For use in diabetes patients



- ✓ Validated with clinical A/A grade according to the British and Irish Hypertension Society (BIHS), and clinically validated according to AAMI/ ISO standards for many special patient groups including children from the age of 3 and patients with end stage renal disease, diabetes, pregnancy pre-eclampsia and large arm circumference.
- ✓ Mean Arterial Pressure (MAP) and Pulse Pressure (PP).
- ✓ Rechargeable battery, energy-efficient and environmentally friendly.
- ✓ Free WatchBP Analyzer software can be downloaded from the website: <https://www.microlife.com/support/software-professional-products>





**Inter-Arm-Difference (IAD)**

Dual-cuff design for simultaneous measurement on both arms to assess the Inter-Arm-Difference (IAD).



**AFIB**

**Reliable AFIB screening**

Screens for Atrial Fibrillation (AF) with high accuracy (sensitivity 98% - specificity 92%) as demonstrated in multiple comparative studies with 12-lead ECG.



**ABI assessment (optional)**

Fast, easy, and accurate Ankle Brachial Index (ABI) assessment.



**Brachial-ankle PWV assessment (optional)**

User-friendly with high reproducibility cuff-based brachial-ankle PWV measurement method to evaluate arterial stiffness.



**Non-invasive central blood pressure measurement (optional)**

Determines blood pressure of the ascending aorta based on Pulse Volume Plethysmography (PVP) waveforms to help in making a reliable diagnosis of cardiovascular risk and hypertension.



MODE.

**Flexible BPM schedule**

Enables the physicians to adjust the settings of time intervals and numbers of consecutive measurements to comply with different guidelines.



**Sprint algorithm**

Enable the physicians to perform automated office blood pressure measurements (AOBPM).



**Upgradeable (optional)**

Allows on-demand paid upgrades of “ABI” (ABI included) or “PWV” (brachial-ankle pulse wave velocity (baPWV) and central blood pressure indices included).

**Features overview**

	Office Vascular model types	Rechargeable battery	AFIB	IAD	ABI	PWV	CBP	USB connectivity	Bluetooth connectivity	Flexible BPM schedule	WatchBP Analyzer	Sprint Algorithm	End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia	Children
Advanced		●	●	●				●	●	●	●	●	●	●	●	●
ABI		●	●	●	●			●	●	●	●	●	●	●	●	●
PWV		●	●	●	●	●	●	●	●	●	●	●	●	●	●	●



# WatchBP O3

24-hour  
blood pressure  
measurement



## WatchBP O3



### Microlife AFIB technology

Screens for Atrial Fibrillation during blood pressure measurement.



### Pill-button

For recording medication intake.



### Upgradeable device version

Customize your ABPM according to your needs with additional, helpful technology features.



### Night-time measurement

For measuring blood pressure during sleep.



### PC link

Transmit all measurement data to PC via USB connection.



### Bluetooth® connectivity

Results are transferred to smartphone, PC, Hub by Bluetooth.

Available models



**WatchBP O3 2G**

Professional 24-hour ambulatory blood pressure monitor with upgradeable functions for Atrial Fibrillation detection and central blood pressure measurement.

WatchBP Device	24-hour ambulatory BP monitoring	AFIB detection	Central BP measurement	Night-time measurement	Pill-button	PC link	Bluetooth® connectivity	End-stage renal disease	Special patient validations Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
O3 (2nd Generation)	●	⊙	⊙	●	●	●	●	●	●	●	●

⊙ Optional;

# microlife®

## WatchBP O3 2G

Professional 24-hour blood pressure monitor



AFIB



24-hour ABPM



Central blood pressure measurement



Upgradeable device version



Pill button



PC link



For use in end stage renal disease



For use in pregnancy



For use in children



For use in diabetes patients

 **Bluetooth®**

- ✓ Pill/Event button for recording medication intake and occurrence of unusual events.
- ✓ Reliable AFIB screening (optional).
- ✓ Non-invasive central blood pressure measurement (optional).
- ✓ USB and Bluetooth® connectivity.
- ✓ Free WatchBP Analyzer software can be downloaded from the website:  
<https://www.microlife.com/support/software-professional-products>




## Customisable measurement functions

Customize your ABPM according to your needs with additional, helpful technology features:

- Atrial Fibrillation (AF) detection.
- Central blood pressure measurement.


### USB interface

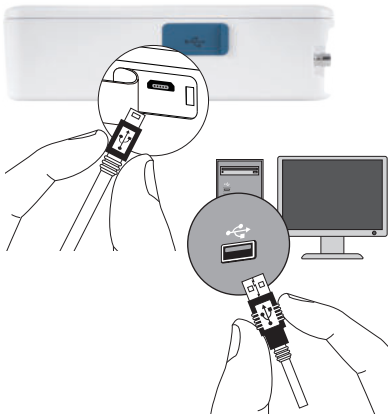

### Bluetooth® connectivity



### Easy to read one page PDF report

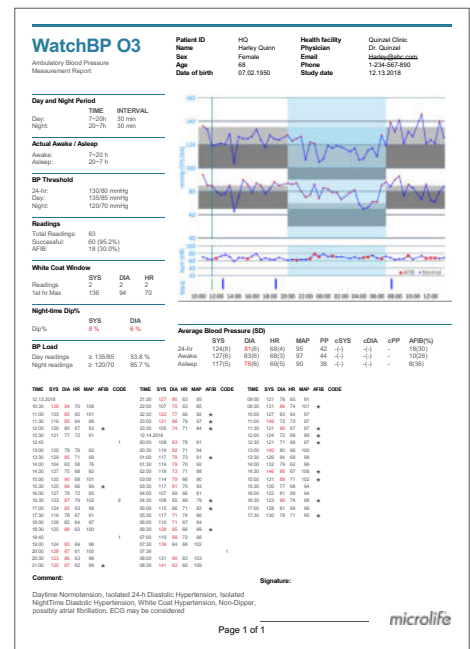

### Pill/Event button

for recording medication intake or to remember certain events.



### WatchBP Analyzer



## Features overview

WatchBP O3	AFIB	Central BP	24-hour ABPM	Pill/Event button	Silent mode	USB connectivity	Bluetooth® connectivity	WatchBP Analyzer	Special patient validations	End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia	Children age 3+
Standard			●	●	●	●	●	●	●	●	●	●	●
AFIB	●		●	●	●	●	●	●	●	●	●	●	●
AFIB & CBP	●	●	●	●	●	●	●	●	●	●	●	●	●



# WatchBP Home

For the best self-measurement of blood pressure at home



## WatchBP Home



**Microlife AFIB technology**  
Screens for Atrial Fibrillation during blood pressure measurement.



**3 consecutive measurements**  
For accurate blood pressure measurement and better Atrial Fibrillation (AF) detection.



**ESH/AHA/BIHS guidelines embedded**  
Helps patients to perform 7-day self-measurement according to ESH/AHA/BIHS guidelines.



**PC link**  
Transmit all measurement data to PC via USB connection.



**Bluetooth® connectivity**  
Results are transferred to smartphone, PC, Hub by Bluetooth.

Available models



**WatchBP Home**

Monitoring hypertension and suitable for use in pregnancy and pre-eclampsia.



**WatchBP Home A**

Monitoring hypertension and Atrial Fibrillation.



**WatchBP Home A BT**

Helps to collect accurate day blood pressure measurements for a good overview of the daily blood pressure pattern.

WatchBP Device	Single measurement	3 Consecutive measurements	AFIB detection	ESH/AHA/BIHS Guidelines embedded	PC link	Bluetooth® connectivity	Special patient validations		
							End-stage renal disease	Diabetes patients	Pregnancy & pre-eclampsia
Home	●			●	●		●	●	●
Home A		●	●	●	●		●	●	
Home A BT (Bluetooth®)		●	●	●	●	●	●	●	

## WatchBP Home



ESH/AHA/BIHS guidelines embedded



PC link



For use in end stage renal disease



For use in pregnancy



For use in diabetes patients



**DIAG.**

### Diagnostic mode

7-day self-measurement program that strictly follows ESH/AHA/BIHS guidelines.

← 7 working days at home →

	1	2	3	4	5	6	7
Morning 06:00 ~ 09:00	<del>✓</del>	✓	✓	✓	✓	✓	✓
Night 18:00 ~ 21:00	<del>✓</del>	✓	✓	✓	✓	✓	✓

1 measurement = 2 readings

Microlife WatchBP Home, also accurate for use in pregnancy and pre-eclampsia



**USUAL**

### Usual mode

For single measurement at any time (250 memory).

# WatchBP Home A

WatchBP Home

Officially recommended as a useful option for **NHS clinicians** by NICE



AFIB



3 consecutive measurements



ESH/AHA/BIHS guidelines embedded



PC link



For use in end stage renal disease



For use in diabetes patients



## Diagnostic mode (with AFIB)

7-day self-measurement program that strictly follows ESH/AHA/BIHS guidelines.



## Usual mode (with AFIB)

3 consecutive measurements for screening for Atrial Fibrillation (250 memory).

## WatchBP Analyzer Home A

Personal information		Average values for day	7th day			
Name	Medical Card	Subtotal 15 readings on average	Time	Sys	Dia	Pulse
11/1/2010	08:36	131	81	73	74	74
11/1/2010	08:36	128	84	71	77	71
11/1/2010	21:44	109	72	68	110	74
11/1/2010	21:47	118	76	69	111	72
7th day						
11/1/2010	08:45	112	73	70	111	72
11/1/2010	08:50	109	72	67	113	74
11/1/2010	20:04	119	72	70	110	72
11/1/2010	20:06	120	71	74	111	72
7th day						
11/1/2010	08:13	112	69	68	111	72
11/1/2010	08:16	108	78	71	110	71
11/1/2010	20:16	105	67	75	110	71
11/1/2010	20:28	114	68	74	111	74

## WatchBP Home A BT

Officially recommended as a useful option for **NHS clinicians** by NICE



AFIB



3 consecutive measurements



ESH/AHA/BIHS guidelines embedded



PC link



For use in end stage renal disease



For use in diabetes patients



**DIAG.**

### Diagnostic mode (with AFIB)

7-day self-measurement program that strictly follows ESH/AHA/BIHS guidelines.



**USUAL**


### Usual mode (with AFIB)

3 consecutive measurements for screening for Atrial Fibrillation (250 memory).

### WatchBP Analyzer Home A BT

Personal Information		Average values for day		20/10/2010			
Name	Client code	Min	Average	Time	Sys	Dia	Pulse
...	...	102	112	08:30	112	87	74
Personal history		Average values for day		21/10/2010		22/10/2010	
...	...	105	115	09:00	115	76	72
Personal history		Average values for day		23/10/2010		24/10/2010	
...	...	108	118	08:00	118	79	71
Personal history		Average values for day		25/10/2010		26/10/2010	
...	...	110	120	08:15	120	80	68

### How does a connected health solution work?

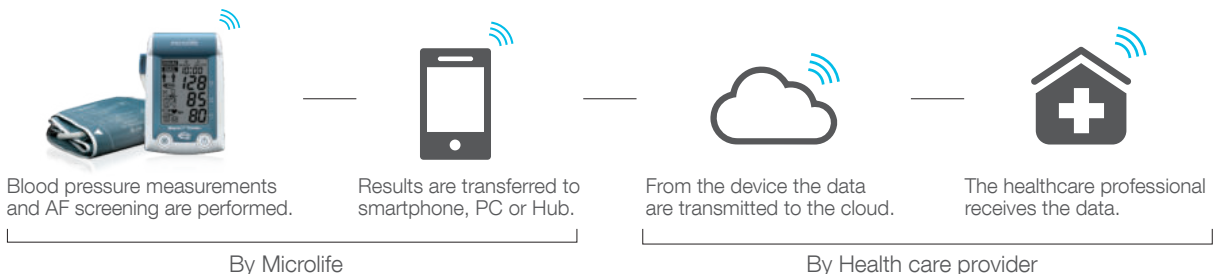





## WatchBP Cuffs

The cuff is as important as the blood pressure monitor itself. Using the right size cuff helps to prevent erroneous blood pressure measurements.




## WatchBP Cuffs


- ✓ Latex-free and PVC-free.
- ✓ Washable cuffs available in both nylon and cotton.
- ✓ Disposable cuffs available (for single patient use).

## WatchBP Disposable cuffs


Upper arm – Single patient use (For WatchBP Office series)

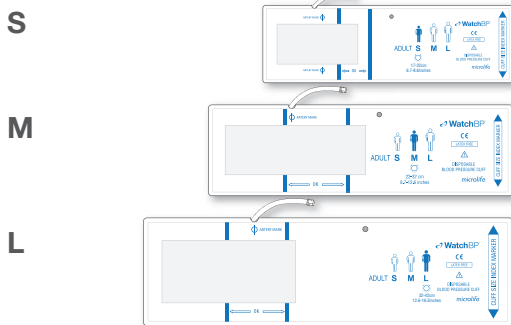

### Tube with cuff connector

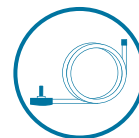


50 cm tube with cuff connector

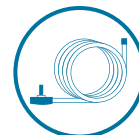


100 cm tube with cuff connector

# WatchBP Office Cuffs

## Upper arm – Soft

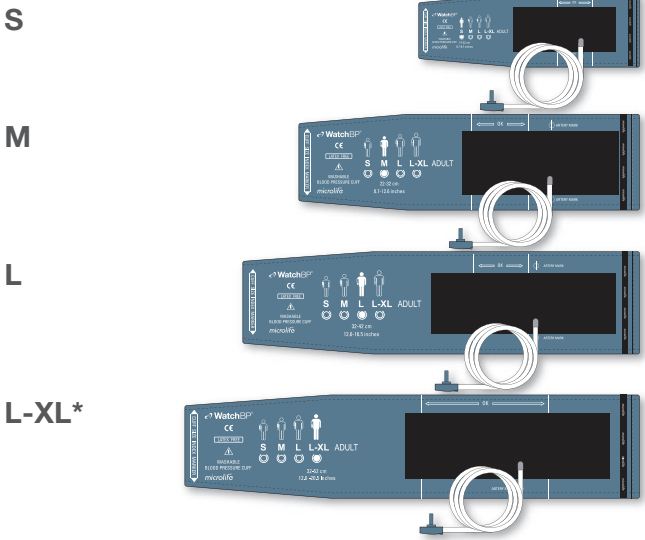


### Cuff connector

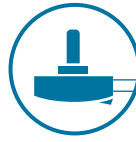


For Office series

## Upper arm – Rigid

**M-L**  
 With air tube 130 cm for  
 Office AFIB, Office ABI

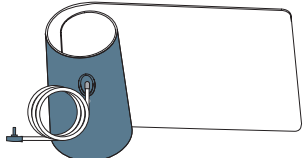


### Cuff connector

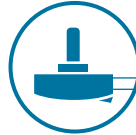


For Office series

## Ankle – Soft

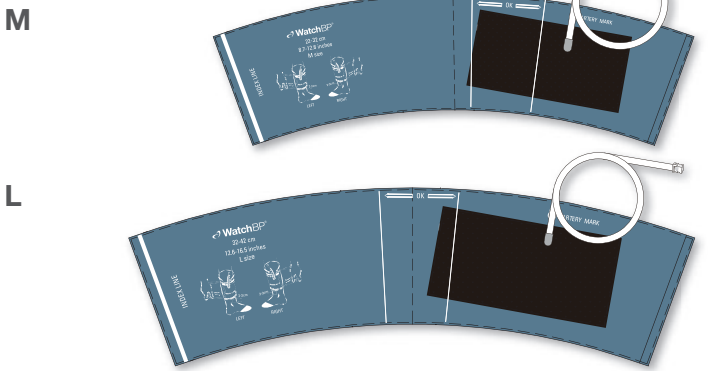

### Tube with cuff connector

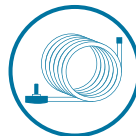


170 cm tube with cuff connector




# WatchBP O3 Ambulatory Cuffs


## Upper arm – Soft

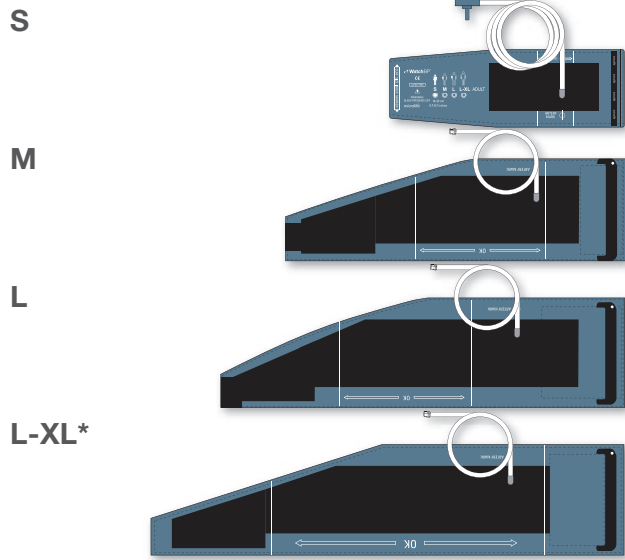

### Tube with cuff connector

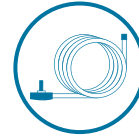


For O3 2G

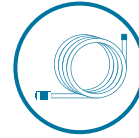


For O3  
100 cm tube with metal connector

# WatchBP Home Cuffs

## Upper arm – Soft

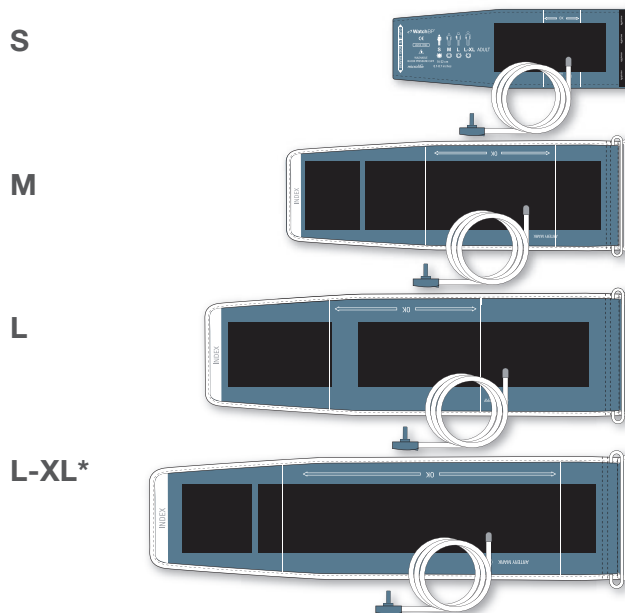

### Tube with cuff connector

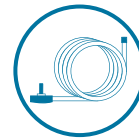

80 cm tube with cuff connector

Cuff	Upper arm cuff					Ankle cuff	
Size	S	M	M-L	L	L-XL	M	L
Measures (cm)	14-22	22-32	22-42	32-42	32-52	22-32	32-42
Measures (inches)	5.5-8.7	8.7-12.6	8.7-16.5	12.6-16.5	12.6-20.5	8.7-12.6	12.6-16.5

\* the L-XL cuff has been validated for use among patients with large arms.



## Reference

### Microlife validation studies

1. Halfon M, et al. Use of oscillometric devices in atrial fibrillation: a comparison of three devices and invasive blood pressure measurement. *Blood pressure*. 2017;1-8.
2. Bing S, et al. Validation of the Microlife BP A200 Comfort and W2 Slim automated blood pressure monitors in a general adult population according to the European Society of Hypertension and the ANSI/AAMI/ISO 81060-2: 2013 protocols. *Blood Press Monit*. 2016;21:118-123.
3. Nathan HL, et al. An accurate semiautomated oscillometric blood pressure device for use in pregnancy (including pre-eclampsia) in a low-income and middle-income country population: the Microlife 3AS1-2. *Blood Press Monit*. 2015;20:52-55.
4. Nathan HL, et al. Accuracy validation of the Microlife 3AS1-2 blood pressure device in a pregnant population with low blood pressure. *Blood Press Monit*. 2015;20:299-302.
5. Gandolfo C, et al. Validation of a simple method for atrial fibrillation screening in patients with stroke. *Neurological sciences : official journal of the Italian Neurological Society and of the Italian Society of Clinical Neurophysiology*. 2015;36:1675-1678.
6. Cheng HM, et al. Measurement accuracy of a stand-alone oscillometric central blood pressure monitor: a validation report for Microlife WatchBP Office Central. *Am J Hypertens*. 2013;26:42-50.
7. Stergiou GS, et al. Accuracy of automated oscillometric blood pressure measurement in patients with atrial fibrillation: 1a.04. *Journal of Hypertension*. 2011;29:e2.
8. Saladini F, et al. Accuracy of Microlife WatchBP Office ABI monitor assessed according to the 2002 European Society of Hypertension protocol and the British Hypertension Society protocol. *Blood Press Monit*. 2011;16:258-261.
9. Masiero S, et al. Accuracy of the Microlife large-extra large-sized cuff (32-52 cm) coupled to an automatic oscillometric device. *Blood Press Monit*. 2011;16:99-102.
10. Kollias A, et al. Automated determination of the ankle-brachial index using an oscillometric blood pressure monitor: validation vs. Doppler measurement and cardiovascular risk factor profile. *Hypertens Res*. 2011;34:825-830.
11. Stergiou GS, et al. Are there really differences between home and daytime ambulatory blood pressure? Comparison using a novel dual-mode ambulatory and home monitor. *J Hum Hypertens*. 2010;24:207-212.
12. Ragazzo F, et al. Validation of the Microlife WatchBP O3 device for clinic, home, and ambulatory blood pressure measurement, according to the International Protocol. *Blood Press Monit*. 2010;15:59-62.
13. Bonso E, et al. Accuracy of a single rigid conical cuff with standard-size bladder coupled to an automatic oscillometric device over a wide range of arm circumferences. *Hypertens Res*. 2010;33:1186-1191.
14. Palatini P, et al. Validation of Microlife BP W100 wrist device assessed according to the European Society of Hypertension and the British Hypertension Society protocols. *Blood Press Monit*. 2009;14:41-44.
15. Chung Y, et al. Validation and compliance of a home monitoring device in pregnancy: microlife WatchBP home. *Hypertens Pregnancy*. 2009;28:348-359.
16. Stergiou GS, et al. Validation of the Microlife Watch BP Office professional device for office blood pressure measurement according to the International protocol. *Blood Press Monit*. 2008;13:299-303.
17. Palatini P, et al. Validation of the Microlife BP W200-1 wrist device for blood pressure measurement. *Blood Press Monit*. 2008;13:295-298.
18. Thompson AM, et al. Validation of an oscillometric home blood pressure monitor in an end-stage renal disease population and the effect of arterial stiffness on its accuracy. *Blood Press Monit*. 2007;12:227-232.
19. Stergiou GS, et al. Validation of the Microlife WatchBP Home device for self home blood pressure measurement according to the International Protocol. *Blood Press Monit*. 2007;12:185-188.
20. Belghazi J, et al. Validation of four automatic devices for self-measurement of blood pressure according to the International Protocol of the European Society of Hypertension. *Vasc Health Risk Manag*. 2007;3:389-400.
21. Stergiou GS, et al. Validation of the Microlife BPA100 Plus device for self-home blood pressure measurement according to the International Protocol. *Blood Press Monit*. 2006;11:157-160.
22. de Greeff A, et al. The Microlife 3AC1: An accurate blood pressure measurement device in pregnancy and pre-eclampsia. *J Hypertens* 2006;24.
23. Topouchian JA, et al. Validation of two devices for self-measurement of brachial blood pressure according to the International Protocol of the European Society of Hypertension: the SEINEX SE-9400 and the Microlife BP 3AC1-1. *Blood Press Monit*. 2005;10:325-331.
24. Reinders A, et al. An accurate automated blood pressure device for use in pregnancy and pre-eclampsia: the Microlife 3BTO-A. *BJOG*. 2005;112:915-920.
25. El Assaad MA, et al. Validation of the Microlife BP 3AC1-1 (R) device for blood pressure measurement according to the International validation Protocol. *American Journal of Hypertension*. 2003;16:47A-47A.
26. Cuckson AC, et al. Validation of the Microlife BP 3BTO-A oscillometric blood pressure monitoring device according to a modified British Hypertension Society protocol. *Blood Press Monit*. 2002;7:319-324.
27. Kollias A., et al. Automated pulse wave velocity assessment using a professional oscillometric office blood pressure monitor. *J Clin Hypertens*. 2020;00:1-7.
28. Palatini P, Fania C, Gasparotti F. Accuracy of the WatchBP Office ABI device for office blood pressure measurement over a wide range of arm sizes. *Blood Press Monit*. 2018 Apr;23(2):117-119.



## Microlife AFIB accuracy and screening studies

1. Wiesel J, et al. Screening for Atrial Fibrillation in Patients  $\geq 65$  Years Using an Automatic Blood Pressure Monitor in a Skilled Nursing Facility. *Am J Cardiol.* 2017;120:1322-1324.
2. Halfon M, et al. Use of oscillometric devices in atrial fibrillation: a comparison of three devices and invasive blood pressure measurement. *Blood pressure.* 2017:1-8.
3. Chan PH, et al. Diagnostic performance of an automatic blood pressure measurement device, Microlife WatchBP Home A, for atrial fibrillation screening in a real-world primary care setting. *BMJ open.* 2017;7:e013685.
4. Chan PH, et al. Head-to-Head Comparison of the AliveCor Heart Monitor and Microlife WatchBP Office AFIB for Atrial Fibrillation Screening in a Primary Care Setting. *Circulation.* 2017;135:110-112.
5. Verberk WJ, et al. Screening for atrial fibrillation with automated blood pressure measurement: Research evidence and practice recommendations. *Int J Cardiol.* 2016;203:465-473.
6. Twigg MJ, et al. Identification of patients with atrial fibrillation in UK community pharmacy: an evaluation of a new service. *Int J Clin Pharm.* 2016;38:784-787.
7. Omboni S, et al. Opportunistic screening of atrial fibrillation by automatic blood pressure measurement in the community. *BMJ open.* 2016;6:e010745.
8. Gandolfo C, et al. Validation of a simple method for atrial fibrillation screening in patients with stroke. *Neurological sciences : official journal of the Italian Neurological Society and of the Italian Society of Clinical Neurophysiology.* 2015;36:1675-1678.
9. Willits I, et al. WatchBP Home A for opportunistically detecting atrial fibrillation during diagnosis and monitoring of hypertension: a NICE Medical Technology Guidance. *Applied health economics and health policy.* 2014;12:255-265.
10. Wiesel J, et al. Comparison of the Microlife blood pressure monitor with the Omron blood pressure monitor for detecting atrial fibrillation. *Am J Cardiol.* 2014;114:1046-1048.
11. Kearley K, et al. Triage tests for identifying atrial fibrillation in primary care: a diagnostic accuracy study comparing single-lead ECG and modified BP monitors. *BMJ open.* 2014;4:e004565.
12. Wiesel J, et al. Screening for asymptomatic atrial fibrillation while monitoring the blood pressure at home: trial of regular versus irregular pulse for prevention of stroke (TRIPPS 2.0). *Am J Cardiol.* 2013;111:1598-1601.
13. NICE. WatchBP Home A for opportunistically detecting atrial fibrillation during diagnosis and monitoring of hypertension <http://guidance.nice.org.uk/MTG13>. 2013;Assessed 18 Aug. 2015.
14. Ermini G, et al. Switching from traditional to automatic sphygmomanometer increases opportunistic detection of atrial fibrillation in hypertensive patients. *BJMP.* 2013;6:a6161.
15. Verberk WJ, et al. Accuracy of oscillometric blood pressure monitors for the detection of atrial fibrillation: a systematic review. *Expert Rev Med Devices.* 2012;9:635-640.
16. Stergiou GS, et al. accuracy of automated oscillometric blood pressure measurement in patients with atrial fibrillation: 1A.04. *Journal of Hypertension.* 2011;29:e2.
17. Wiesel J, et al. Detection of atrial fibrillation using a modified microlife blood pressure monitor. *Am J Hypertens.* 2009;22:848-852.
18. Stergiou GS, et al. Diagnostic accuracy of a home blood pressure monitor to detect atrial fibrillation. *J Hum Hypertens.* 2009;23:654-658.
19. Wiesel J, et al. Home monitoring with a modified automatic sphygmomanometer to detect recurrent atrial fibrillation. *J Stroke Cerebrovasc Dis.* 2007;16:8-13.
20. Wiesel J, et al. The use of a modified sphygmomanometer to detect atrial fibrillation in outpatients. *Pacing Clin Electrophysiol.* 2004;27:639-643.
21. Clark, Katherine, Olivia Snowball, Diane Nzelu, Polly Kay, and Nikos A. Kametas. "Validation of the Microlife WatchBP Home Blood Pressure Device in Pregnancy for Medium and Large Arm Circumferences." *Blood Pressure Monitoring* 23, no. 3 (June 2018): 171–74. <https://doi.org/10.1097/MBP.0000000000000315>.
22. Fania, C., I. Lazzaretto, U. Fontana, and P. Palatini. "Accuracy of the WatchBP O3 Device for Ambulatory Blood Pressure Monitoring According to the New Criteria of the ISO81060-2 2018 Protocol." *Blood Press Monit* 25, no. 5 (October 2020): 285–90. <https://doi.org/10.1097/MBP.0000000000000459>.
23. Kollias, Anastasios, Konstantinos G. Kyriakoulis, Areti Gravvani, Ioannis Anagnostopoulos, and George S. Stergiou. "Automated Pulse Wave Velocity Assessment Using a Professional Oscillometric Office Blood Pressure Monitor." *Journal of Clinical Hypertension (Greenwich, Conn.)* 22, no. 10 (October 2020): 1817–23. <https://doi.org/10.1111/jch.13966>.
24. Halfon M, et al. Use of oscillometric devices in atrial fibrillation: a comparison of three devices and invasive blood pressure measurement. *Blood pressure.* 2017:1-8.
25. Kollias, Anastasios, Angeliki Ntineri, Konstantinos G. Kyriakoulis, Emelina Stambolliu, Styliani Lagou, Nadia Boubouchairopoulou, and George S. Stergiou. "Validation of the Professional Device for Blood Pressure Measurement Microlife WatchBP Office in Adults and Children According to the American National Standards Institute/Association for the Advancement of Medical Instrumentation/International Organization for Standardization Standard." *Blood Pressure Monitoring* 23, no. 2 (April 2018): 112–14. <https://doi.org/10.1097/MBP.0000000000000307>.
26. Palatini, Paolo, Claudio Fania, and Federica Gasparotti. "Accuracy of the WatchBP Office ABI Device for Office Blood Pressure Measurement over a Wide Range of Arm Sizes." *Blood Pressure Monitoring* 23, no. 2 (April 2018): 117–19. <https://doi.org/10.1097/MBP.0000000000000312>.
27. Palatini, Paolo, Ilaria Lazzaretto, Umberto Fontana, and Claudio Fania. "Accuracy of the Oscillometric Method for the Measurement of Heart Rate at Rest and during Mild Exercise." *Journal of Hypertension* 40, no. 2 (February 1, 2022): 240–44. <https://doi.org/10.1097/HJH.0000000000002998>.
28. Stergiou, George S., Konstantinos G. Kyriakoulis, Ioanna Bountzona, Ariadni Menti, Antonios Destounis, Petros Kalogeropoulos, and Anastasios Kollias. "Automated Blood Pressure Measurement in Atrial Fibrillation: Validation Process Modification and Evaluation of a Novel Professional Device Which Detects Atrial Fibrillation and Adapts Its Blood Pressure Measurement Algorithm." *Journal of Hypertension* 39, no. 4 (April 1, 2021): 614–20. <https://doi.org/10.1097/HJH.0000000000002684>.
29. Anastasios Kollias, Konstantinos G. Kyriakoulis, Areti Gravvani, Ioannis Anagnostopoulos, George S. Stergiou. "Automated pulse wave velocity assessment using a professional oscillometric office blood pressure monitor". *Wiley Online Library Journal* (6 June 2020): DOI: 10.1111/jch.13966



**Europe / Middle-East / Africa**

Microlife WatchBP AG  
Eспенstrasse 139  
9443 Widnau  
Switzerland  
Tel: + 41 71 727 7000  
Fax: + 41 71 727 7011  
watchbp@microlife.ch  
www.watchbp.com

**United Kingdom**

Microlife Health Management Ltd.  
St. John's Innovation Centre  
Cowley Road  
Cambridge  
CB4 0WS  
United Kingdom  
Tel: + 44 (0) 1223 422 444  
Fax: + 44 (0) 1223 420 844  
info@watchbp.co.uk  
service@watchbp.co.uk  
www.watchbp.co.uk

**Asia**

Microlife Corporation  
9F, 431, RuiGang Road, NeiHu Taipei, 114,  
Taiwan, R.O.C.  
Tel: + 886 2 8797 1288  
Fax: + 886 2 8797 1283  
watchbp@microlife.com.tw  
www.watchbp.com

**WatchBP Microlife solutions B.V.**

Mercator I  
Toernooiveld 220  
6525 EC Nijmegen  
The Netherlands  
watchbp@microlife.ch  
www.watchbp.nl

**United States / Canada / Latin America**

Microlife USA Inc.,  
1617 Gulf to Bay Blvd, 2nd Floor  
Clearwater, FL 33755  
USA  
Tel: +1-727-4425353  
Fax: +1-727-4425331  
msa@microlifeusa.com  
www.watchbp.com

The logo for microlife, featuring the word "microlife" in a lowercase, bold, sans-serif font. A registered trademark symbol (®) is positioned at the top right of the letter "e".

**For more information, please visit: [www.watchbp.com](http://www.watchbp.com)**

Microlife Corporation is a global corporation working closely with medical societies, specialists and primary care physicians to create tools and solutions that advance healthcare for the benefit of both physicians and patients.