microlife®

BP B2 Easy Blood Pressure Monitor

EN → 1	CZ → 25	HU → 49
RU → 7	SK → 31	HR → 55
BG → 13	SL → 37	PL → 61
RO → 19	SR → 43	

IB_BP_B2_Easy_E-V11_1321
Revision Date: 2021-03-19

Microlife AG
Espenstrasse 139
9443 Widnau / Switzerland
www.microlife.com

Microlife UAB
P. Lukšio g. 32,
08222 Vilnius
Lithuania

CE0044

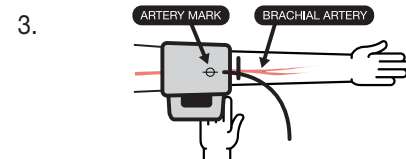
Preparation

Sit on a back-supported chair and keep your legs uncrossed. / Сядьте, не скрещивая ноги, на стул со спинкой. / Сядьте на стол с облегалка и не кръстосайте краката си. / Așezați-vă într-un scaun cu spatar și nu încrucișați picioarele / Sediñete si na židli s opěrkou a nepřekřížujte nohy. / Sadnite si na stoličku s opierkou a neprekrížujte si nohy. / Ušedite se na stol s podprtim hrbtom in nogami na tleh (ne prekrížajte jih). / Sedite na stolicu koja podupire leđa i nemojte prekrštatí noge. / Üljon egy háttámlás székre és lábait ne rakja keresztbe! / Sjednite na stolicu koja podupire leđa i neka Vam noge ne budu prekrížene. / Usiadź na krzesle wspieranym plecami i nie rozstawiaj nóg.

Avoid thick or close-fitting garments on the upper arm. / Освободите плечо от плотной или плотно облегающей одежды. / Избягвайте дебели или плтно прилепващи дрехи върху горната част на ръката. / Evitaiți îmbracaminte groasa sau stâmta pe braț / Vyhněte se těsnému nebo přiléhavému oděvu na horní části paže. / Vyhnite sa tesnému alebo priliehavému oděvu na hornej časti ramena. / Izogibajte se tesnim oblačilom, ki stisnejo nadlaket. / Izbegavajte usku odeću na nadlaktici. / Kerülje a vastag vagy szoros ruhákat a felkaron! / Izbjegavajte usku odjeću na nadlaktici. / Unikaj grubej lub dopasowanej odzieży na ramieniu.

Place the artery-mark on the cuff over your artery. / Поместите манжету так, чтобы значок артерии на манжете находился над артерией руки в сгибе локтя. / Поставете маркера на маншета върху артерията. / Plasați semnul artera al manșetei peste artera dumneavoastră / Umistěte manžetu se značkou tepny nad Vaši tepnu. / Umistite znak za arterijo, ki je na manšeti, na vašo arterijo. / Postavite oznaku za arteriju na manžetni preko Vaše arterije. / Helyezze a mandzsetta artériajelzőjét az artériája fölé! / Postavite oznaku arterije na manžeti preko Vaše arterije. / Umieść mankiecie na tętnicy.

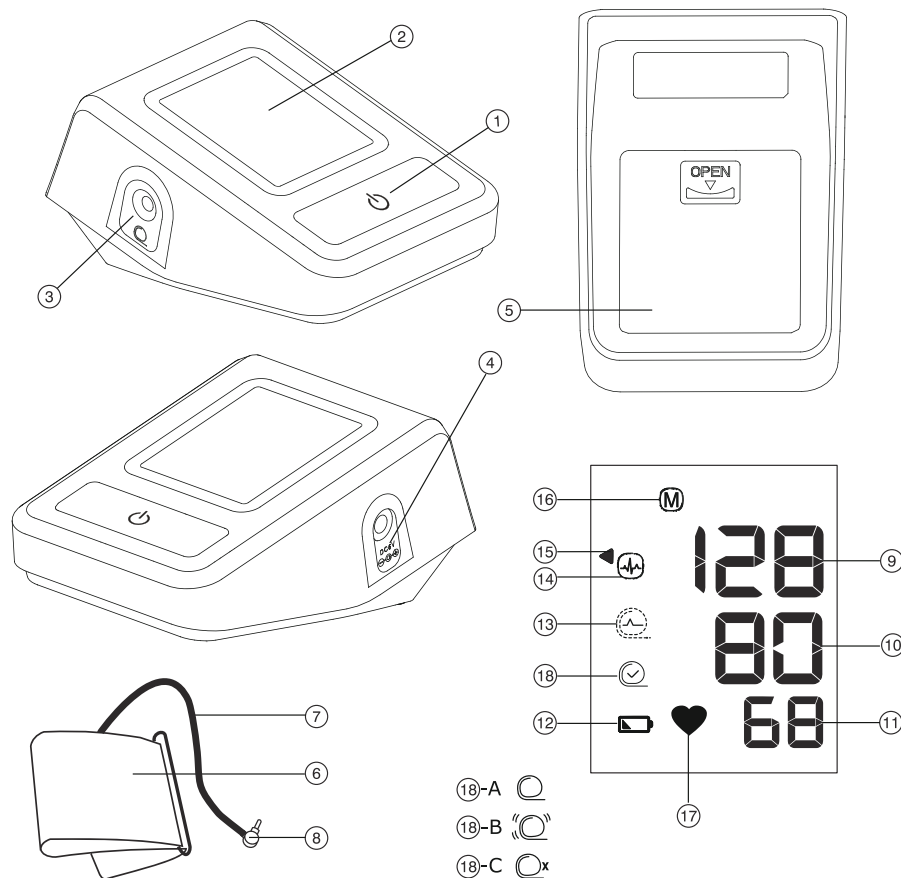
Fit the cuff closely, but not too tight. / Поместите манжету так, чтобы она плотно, но не туго прилежала. / Стегнете добре маншета, но не твърде много. / Fixați manșeta ferm dar nu prea stransă / Nasadite manžetu těsně, ale ne příliš těsně. / Nasadite manžetu tesne, ale nie příliš tesne. / Manšeto dobro nameštite, vendar ne pretensno. / Dobro zategnite manžetnu, ali ne previše stegnuto. / Rögzítse a mandzsettát szorosan, de ne feszítse meg túl erősen! / Postavite manžetu blizu, ali ne pretjensno. / Zaciñij mankiet dokladnie, jednak niezbyt silnie.

Position the cuff 1-2 cm above your elbow. / Расположите манжету на 1-2 см выше локтя. / Поставете маншета 1-2 см над лакътя. / Plasați manșeta cu 1-2 cm deasupra articulației / Manžetu umistěte 1-2 cm nad loket. / Manžetu umiestnite 1-2 cm nad laket. / Nameštite manšeto 1-2 cm nad komolcem. / Postavite manžetnu 1-2 cm iznad Vašeg lakta. / A mandzsetta 1-2 centiméterrel legyen a könyökétől feljebb! / Postavite manžetu 1-2 cm iznad Vašeg lakta. / Ustaw mankiet na 1-2 cm powyżej łokcia.

Keep your arm still and do not speak during the measurement. / Держите руку неподвижно и не разговаривайте во время измерения. / Дръжте ръката си неподвижна и не говорете по време на измерването. / Nu mișcați mâna și nu vorbiți în timpul măsurării tensiunii arteriale. / Držte ruku uvolněnou a během měření nemlve. / Držte ruku uvolněnou a počas merania nerozprávjajte. / Med merjenjem ne premikajte roke in ne govorite. / Držite ruku mirno i nemojte pričati tokom merenja. / Ne mozgassa a karját és ne beszéljen mérés közben! / Držite ruku mirno i nemojte pričati tijekom mjerenja. / Trzymaj rękę nieruchomo i nie mów podczas pomiaru.

1.

Avoid eating, bathing, smoking or caffeine (approx. 30 min).
Избегайте приема пищи, купания, курения или приема кофеиносодержащих продуктов (прибл. за 30 минут до измерения).
Избягайте ядене, къпане, пушене или кофеин (около 30 минути).
Evitați sa mâncați, sa faceți baie, sa fumați, sau sa beți cafea (aproximativ 30 min).
Vyhňte se jídlu, koupání, kouření nebo kofeinu (přibližně 30 minut).
Vyhňte sa jedlu, kúpaniu, fajčeniu alebo kofeínu (približne 30 minút).
Izogibajte se hrani, kopanju in kajenju (približno 30 minut).
Izbegavajte hranu, kupanje, pušenje i kofein (približno 30 minuta).
Kerülje az evést, fürdést, dohányzást vagy koffeinfogyasztást (a megelőző kb. 30 percben)!
Izbjegavajte jesti, kupati se, pušiti i kofein (oko 30 min.)
Unikaj jedzenia, kąpieli, palenia lub kofeiny (około 30 minut).

2.

Avoid activity and relax for 5-10 min.
В течение 5-10 мин. оставайтесь в расслабленном состоянии и избегайте физической нагрузки.
Избягвайте активности и се отпуснете в продължение на 5-10 минути.
Evitați orice activitate și relaxați-vă 5-10 min.
Vyhňte se fyzické aktivity a odpočívajte 5-10 minút.
Vyhňte sa fyzickej aktivity a odpočívajte 5-10 minút.
Izogibajte se fizični aktivnosti in počivajte 5-10 minut.
Izbegavajte fizičku aktivnost i opustite se tokom 5-10 minuta.
Kerülje a fizikai tevékenységet, pihenjen 5-10 percig!
Izbjegavajte aktivnost i opustite se 5-10 min.
Unikaj aktywności i zrelaksuj się przez 5-10 minut.

3.

Measure before medication intake.
Измеряйте до приема лекарства.
Измерете преди приема на лекарството ви.
Měření proveďte před přijetím léku.
Měření uskutočnite pred užitím liekov.
Merjenje izvedite pred zaužitjem zdravil.
Obavite merenje pre uzimanja leka.
Mérjen a gyógyszer bevétele előtt!
Izvedite mjerjenje prije nego što uzmete lijek.
Zmierz przed przyjęciem leku.

Name of Purchaser / Ф.И.О. покупателя /
Име на купувача / Numele cumpărătorului /
Jméno kupujícího / Meno zákazníka / Ime in
priimek kupca / Ime i prezime kupca / Vásárló
neve / Ime i prezime kupca / Imię i nazwisko
nabywcy

Serial Number / Серийный номер / Серийен
номер / Număr de serie / Výrobní číslo /
Výrobné číslo / Serijska številka / Serijski broj /
Sorozatszám / Serijski broj / Numer seryjny

Date of Purchase / Дата покупки / Дата на
закупуване / Data cumpărării / Datum nákupu /
Dátum kúpy / Datum nakupa / Datum kupovine /
Vásártás dátuma / Datum kupovine / Data zakupu

Specialist Dealer / Специализированный дилер /
Специалист дистрибутор / Distribuitor de spe-
cialitate / Specializovaný dealer / Špecializovaný
predajca / Spezializirani trgovec / Ovlašćeni diler /
Forgalmazó / Ovlašteni prodavač / Przedstawiciel

- ① ON/OFF button
- ② Display
- ③ Cuff Socket
- ④ Mains Adapter Socket
- ⑤ Battery Compartment
- ⑥ Cuff
- ⑦ Cuff Tube
- ⑧ Cuff Connector

Display

- ⑨ Systolic Value
- ⑩ Diastolic Value
- ⑪ Pulse Rate
- ⑫ Battery Display
- ⑬ Cuff Signal Indicator «Err 1»
- ⑭ Irregular Heartbeat (IHB) Symbol
- ⑮ Traffic Light Indicator
- ⑯ Stored Value
- ⑰ Pulse Indicator
- ⑱ Cuff Fit Check
 - A: Suboptimal Cuff Fit
 - B: Arm Movement Indicator «Err 2»
 - C: Cuff Pressure Check «Err 3»

Read the instructions carefully before using this device.

Type BF applied part

Keep dry

Intended use:

This oscillometric blood pressure monitor is intended for measuring non-invasive blood pressure in people aged 12 years or older.

It is clinically validated in patients with hypertension, hypotension, diabetes, pregnancy, pre-eclampsia, atherosclerosis, end-stage renal disease, obesity and the elderly.

Dear Customer,

This device was developed in collaboration with physicians and clinical tests carried out prove its measurement accuracy to be of a very high standard.*

If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Stay healthy – Microlife AG!

** This device uses the same measuring technology as the award winning «BP 3BTO-A» model tested according to the British and Irish Hypertension Society (BIHS) protocol.*

Table of Contents

- 1. Using the Device for the First Time**
 - Inserting the batteries
 - Selecting the correct cuff
- 2. Checklist for Taking a Reliable Measurement**
- 3. Taking a Blood Pressure Measurement**
 - Manual inflation
 - How do I evaluate my blood pressure?
 - Appearance of the Irregular Heartbeat (IHB) Symbol
- 4. Data Memory**
- 5. Battery Indicator and Battery change**
 - Low battery
 - Flat battery – replacement
 - Which batteries and which procedure?
 - Using rechargeable batteries
- 6. Using a Mains Adapter**
- 7. Error Messages**

8. Safety, Care, Accuracy Test and Disposal

- Safety and protection
- Device care
- Cleaning the cuff
- Accuracy test
- Disposal

9. Guarantee

10. Technical Specifications

Guarantee Card (see Back Cover)

1. Using the Device for the First Time

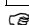
Inserting the batteries

After you have unpacked your device, first insert the batteries. The battery compartment (5) is on the bottom of the device. Insert the batteries (4 x 1.5 V, size AA), thereby observing the indicated polarity.

Selecting the correct cuff

Microlife offers different cuff sizes. Select the cuff size to match the circumference of your upper arms (measured by close fitting in the centre of the upper arm).

Cuff size	for circumference of upper arm
S	17 - 22 cm
M	22 - 32 cm
M - L	22 - 42 cm
L	32 - 42 cm
L - XL	32 - 52 cm

 Pre-shaped cuffs are optionally available.

 Only use Microlife cuffs.

- ▶ Contact your local Microlife Service if the enclosed cuff (6) does not fit.
- ▶ Connect the cuff to the device by inserting the cuff connector (8) into the cuff socket (3) as far as it will go.

2. Checklist for Taking a Reliable Measurement

- ▶ Avoid activity, eating or smoking immediately before the measurement.
- ▶ Sit down on a back-supported chair and relax for 5 minutes. Keep the feet flat on the floor and do not cross your legs.
- ▶ **Always measure on the same arm** (normally left). It is recommended that doctors perform double arm measurements on a

patients first visit in order to determine which arm to measure in the future. The arm with the higher blood pressure should be measured.

- ▶ Remove close-fitting garments from the upper arm. To avoid constriction, shirt sleeves should not be rolled up - they do not interfere with the cuff if they are laid flat.

- ▶ Always ensure that the correct cuff size is used (marking on the cuff).
 - Fit the cuff closely, but not too tight.
 - Make sure that the cuff is positioned 1-2 cm above the elbow.
 - The **artery mark** on the cuff (ca. 3 cm long bar) must lie over the artery which runs down the inner side of the arm.
 - Support your arm so it is relaxed.
 - Ensure that the cuff is at the same height as your heart.

3. Taking a Blood Pressure Measurement

1. Press the ON/OFF button (1) to start the measurement.
2. The cuff will now pump up automatically. Relax, do not move and do not tense your arm muscles until the measurement result is displayed. Breathe normally and do not talk.
3. The cuff fit check (13) on the display indicates that the cuff is perfectly placed. If the icon (13)-A appears, the cuff is fitted suboptimally, but it is still ok to measure.
4. When the correct pressure is reached, the pumping stops and the pressure falls gradually. If the required pressure was not reached, the device will automatically pump some more air into the cuff.
5. During the measurement, the pulse indicator (17) flashes in the display.
6. The result, comprising the systolic (9) and the diastolic (10) blood pressure and the pulse rate (11) is displayed. Note also the explanations on further display symbols in this booklet.
7. When the device has finished measuring, remove the cuff.
8. Switch off the device. (The monitor does switch off automatically after approx. 1 min.).

 You can stop the measurement at any time by pressing the ON/OFF button (e.g. if you feel uneasy or an unpleasant pressure sensation).

 This monitor is specially tested for use in pregnancy and pre-eclampsia. When you detect unusual high readings in pregnancy, you should measure after a short while again (eg. 1 hour). If the reading is still too high, consult your doctor or gynecologist.

Manual inflation

In case of high systolic blood pressure (e.g. above 135 mmHg), it can be an advantage to set the pressure individually. Press the ON/OFF button after the monitor has been pumped up to a level of approx. 30 mmHg (shown on the display). Keep the button pressed until the pressure is about 40 mmHg above the expected systolic value – then release the button.

How do I evaluate my blood pressure?

The triangle on the left-hand edge of the display (15) points at the range within which the measured blood pressure value lies. The value is either within the optimum (white), elevated (hatched gray) or high (black) range. The classification corresponds to the following ranges defined by international guidelines (ESH, ESC, JSH). Data in mmHg.

Range	Systolic	Diastolic	Recommendation
1. blood pressure too high	≥135	≥85	Seek medical advice
2. blood pressure elevated	130 - 134	80 - 84	Self-check
3. blood pressure normal	<130	<80	Self-check

The higher value is the one that determines the evaluation. Example: a blood pressure value of **140/80** mmHg or a value of **130/90** mmHg indicates «blood pressure too high».

Appearance of the Irregular Heartbeat (IHB) Symbol

This symbol (14) indicates that an irregular heartbeat was detected. In this case, the measured blood pressure may deviate from your actual blood pressure values. It is recommended to repeat the measurement.

Information for the doctor in case of repeated appearance of the IHB symbol:

This device is an oscillometric blood pressure monitor that also measures the pulse during blood pressure measurement and indicates when the heart rate is irregular.

4. Data Memory

This device always stores the last result at the end of the measurement. To recall the reading, press and hold the ON/OFF button (1) while the device is switched off. Release the button as soon as all display elements are shown. The stored result will then be shown, indicated with the letter «M» (16).

5. Battery Indicator and Battery change

Low battery

When the batteries are approximately ¾ empty the battery symbol (12) will flash as soon as the device is switched on (partly filled battery displayed). Although the device will continue to measure reliably, you should obtain replacement batteries.

Flat battery – replacement

When the batteries are flat, the battery symbol (12) will flash as soon as the device is switched on (flat battery displayed). You cannot take any further measurements and must replace the batteries.

1. Open the battery compartment (5) at the back of the device.
2. Replace the batteries – ensure correct polarity as shown by the symbols in the compartment.

Which batteries and which procedure?

- ☞ Use 4 new, long-life 1.5 V, size AA alkaline batteries.
- ☞ Do not use batteries beyond their date of expiry.
- ☞ Remove batteries if the device is not going to be used for a prolonged period.

Using rechargeable batteries

You can also operate this device using rechargeable batteries.

- ☞ Only use «NiMH» type reusable batteries.
- ☞ Batteries must be removed and recharged when the flat battery symbol appears. They should not remain inside the device as they may become damaged (total discharge as a result of low use of the device, even when switched off).
- ☞ Always remove the rechargeable batteries if you do not intend to use the device for a week or more.
- ☞ Batteries cannot be charged in the blood pressure monitor. Recharge batteries in an external charger and observe the information regarding charging, care and durability.

6. Using a Mains Adapter

You can operate this device using the Microlife mains adapter (DC 6V, 600 mA).

- ☞ Only use the Microlife mains adapter available as an original accessory appropriate for your supply voltage.
- ☞ Ensure that neither the mains adapter nor the cable are damaged.

1. Plug the adapter cable into the mains adapter socket (4) in the blood pressure monitor.
2. Plug the adapter plug into the wall socket.

When the mains adapter is connected, no battery current is consumed.

7. Error Messages

If an error occurs during the measurement, the measurement is interrupted and an error message, e.g. «Err 3», is displayed.

Error	Description	Potential cause and remedy
«Err 1» ⑬	Signal too weak	The pulse signals on the cuff are too weak. Re-position the cuff and repeat the measurement.*
«Err 2» ⑱-B	Error signal	During the measurement, error signals were detected by the cuff, caused for instance by movement or muscle tension. Repeat the measurement, keeping your arm still.
«Err 3» ⑱-C	Abnormal cuff pressure	An adequate pressure cannot be generated in the cuff. A leak may have occurred. Check that the cuff is correctly connected and is not too loose. Replace the batteries if necessary. Repeat the measurement.
«Err 5»	Abnormal result	The measuring signals are inaccurate and no result can therefore be displayed. Read through the checklist for performing reliable measurements and then repeat the measurement.*
«HI»	Pulse or cuff pressure too high	The pressure in the cuff is too high (over 299 mmHg) OR the pulse is too high (over 200 beats per minute). Relax for 5 minutes and repeat the measurement.*
«LO»	Pulse too low	The pulse is too low (less than 40 beats per minute). Repeat the measurement.*

* Please immediately consult your doctor, if this or any other problem occurs repeatedly.

8. Safety, Care, Accuracy Test and Disposal

Safety and protection

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- The cuffs are sensitive and must be handled with care.
- Do not exchange or use any other kind of cuff or cuff connector for measuring with this device.
- Only pump up the cuff once fitted.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- If the device is not going to be used for a prolonged period the batteries should be removed.
- Read the additional safety information provided within the individual sections of this instruction manual.
- If you suffer from a cardiac arrhythmia consult with your doctor before using the device. See also chapter «Appearance of the Irregular Heartbeat (IHB) Symbol» of this user manual.
- The measurement results given by this device is not a diagnosis. It is not replacing the need for the consultation of a physician, especially if not matching the patient's symptoms. Do not rely on the measurement result only, always consider other potentially occurring symptoms and the patient's feedback. Calling a doctor or an ambulance is advised if needed.
- Permanently high blood pressure values can damage your health and must be treated by your doctor!**
- Always discuss your values with your doctor and tell him/her if you have noticed anything unusual or feel unsure. **Never rely on single blood pressure readings.**

- Under no circumstances should you alter the dosages of drugs or initiate a treatment without consulting your doctor.
- Deviations between measurements taken by your doctor or in the pharmacy and those taken at home are quite normal, as these situations are completely different.
- The pulse display is not suitable for checking the frequency of heart pacemakers!
- If you are **pregnant**, you should monitor your blood pressure regularly as it can change drastically during this time.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Device care

Clean the device only with a soft, dry cloth.

Cleaning the cuff

The cuff delivered with this device is washable.

1. Remove the cuff connector ⑧ from the cuff tube ⑦ and carefully pull the bladder through the opening at the edge of the cuff cover.
2. Hand wash the cuff cover in soapsuds: not hotter than 30 °C.
3. Completely dry the cuff cover by linen drying.
4. Loop the cuff tube back through its opening and carefully place the bladder flat in the cuff cover.
5. Reattach the cuff connector on the cuff tube.

- ☞ The bladder must lay straight in the cuff cover, not folded.
- ☞ Do not use fabric softener.

WARNING: Do not wash the cuff in a washing machine or dishwasher!

WARNING: Do not dry the cuff cover in a tumble dryer!

WARNING: Under no circumstances may you wash the inner bladder!

Accuracy test

We recommend this device is tested for accuracy every 2 years or after mechanical impact (e.g. being dropped). Please contact your local Microlife-Service to arrange the test (see foreword).

Disposal

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

9. Guarantee

This device is covered by a **5 year guarantee** from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Batteries, power adapter (optional).

The cuff is covered by a functional guarantee (bladder tightness) for 2 years.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website:

www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

10. Technical Specifications

Operating conditions: 10 - 40 °C / 50 - 104 °F

15 - 95 % relative maximum humidity

Storage conditions: -20 - +55 °C / -4 - +131 °F

15 - 95 % relative maximum humidity

Weight: 277 g (including batteries)

Dimensions: 131 x 90 x 60.5 mm

Measuring procedure: oscillometric, corresponding to Korotkoff method: Phase I systolic, Phase V diastolic

Measurement range: 20 - 280 mmHg – blood pressure
40 - 200 beats per minute – pulse

Cuff pressure display range:	0 - 299 mmHg
Resolution:	1 mmHg
Static accuracy:	pressure within ± 3 mmHg
Pulse accuracy:	± 5 % of the readout value
Voltage source:	4 x 1.5 V alkaline batteries; size AA Mains adapter DC 6V, 600 mA (optional)
Battery lifetime:	approx. 920 measurements (using new batteries)
IP Class:	IP20
Reference to standards:	EN 1060-1 /-3 /-4; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11
Expected service life:	Device: 5 years or 10000 measurements Accessories: 2 years

This device complies with the requirements of the Medical Device Directive 93/42/EEC.

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