## Description of this Thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor / measuring tip
- ⑤ Cleaning and disinfecting area (entire thermometer surface)

EN

## Battery Replacement

When the «▼» symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover ③ from the thermometer. Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

## Technical Specifications

Type:	Maximum thermometer
Measurement range:	32.0 °C to 43.9 °C Temp. < 32.0 °C: display «L» for low (too low) Temp. > 43.9 °C: display «H» for high (too high)
Measurement accuracy:	± 0.1 °C; 34 °C - 42 °C
Operating conditions:	10 - 40 °C; 15-95 % relative maximum humidity
Storage conditions:	-25 - +60 °C; 32.0 - 33.9 °C and 42.1 - 43.9 °C
Battery:	3V; CR1632
Battery lifetime:	approx. 3200 measurements (using a new battery)
IP Class:	IP67
Reference to standards:	EN 12470-3, clinical thermometers; ASTM E112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11
Expected service life:	5 years or 10000 measurements

This device complies with the requirements of the Medical Device Directive 93/42/EEC.

## Technical alterations reserved.

## Guarantee

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

The following items are excluded from the guarantee:

- Transport costs and risks of transport.

- Damage caused by incorrect application or non-compliance with the instructions for use.

- Damage caused by leaking batteries.

- Damage caused by accident or misuse.

- Packaging/storage material and instructions for use.

- Regular checks and maintenance (calibration).

- Accessories and wearing parts: Battery.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: [www.microlife.com/support](http://www.microlife.com/support)

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

**WARNING:** The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only.

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Read the instructions carefully before using this device.

Type BF applied part

## Turning on the Thermometer

To turn on the thermometer, press the ON/OFF button ①; a short beep signals «Thermometer ON». A display test is performed. All segments should be displayed.

The last measurement reading will be shown on the display ② automatically for 2 seconds with the «M» icon.

Then at an ambient temperature of less than 32 °C, an «L» and a flashing «C» or «F» appear at the display field ②. The thermometer is ready for use now.

## Function Test

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

## Using the Thermometer

Choose the preferred measuring method. When taking a measurement, the current temperature is continuously displayed and the «C» symbol flashes. If the beep is heard 5 times and the «C» is no longer flashing, the predictive end-temperature has been determined and the thermometer can be read now.

40 short beeps and a red LCD backlight alert the patient that he/she may have a temperature equal to or higher than 37.5 °C. Reference: Oral temperature.

To achieve comparable results allow a 1 minute interval time between measurements.

To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button ①. Otherwise the thermometer will automatically turn off after about 10 minutes.

## Changing from °C to °F

With the thermometer turned off, press and hold the ON/OFF button ① for 5 seconds until «M» disappears. Shortly press the ON/OFF button to switch between °C or °F. Wait 3 seconds till «C» or «F» starts blinking and press ON/OFF button again to confirm your selection. The thermometer is set to your selection for future use.

## Measuring methods / Normal body temperature

In the armpit (axillary) / 34.7 - 37.3 °C  
Wipe the underarm with a dry towel. Place the measuring sensor ④ under the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait at least 5 minutes, regardless of the beep sound.

In the mouth (oral) / 35.5 - 37.5 °C  
Do not eat or drink anything hot or cold 10 minutes before the measurement. The mouth should remain closed up to 2 minutes before starting a reading.

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor ④ must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

If this is not possible due to blocked airways, another method for measuring should be used.

## Approx. measuring time: 10 seconds!

In the anus (rectal) / 36.6 - 38.0 °C  
Carefully insert the measuring sensor ④ of the thermometer 2 to 3 cm into the anal aperture.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

## Approx. measuring time: 10 seconds!

## Cleaning and Disinfecting

For disinfection in home use environment, use a 70% Isopropyl alcohol swab or a cotton tissue moistened with 70% Isopropyl alcohol to wipe surface pollutants off the thermometer probe. Always start wiping from the end of the thermometer probe (approx. at the middle of the thermometer) towards the thermometer tip. Afterwards the entire thermometer (see number ⑤ in the drawing) should be immersed in 70% Isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use.

For professional use: You may apply probe covers (available as accessory). Contact Microlife for further details concerning suitable disinfectants for your thermometer model.

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