

Portable Compressor Nebuliser

- 1 Piston compressor
- 2 ON/OFF button
- 3 Power indicator
- 4 USB Port
- 5 Air filter compartment
- 6 Air outlet
- 7 Nebuliser
 - a- Vapouriser head
- 8 Air hose
- 9 Mouthpiece
- 10 Nose piece
- 11 Adult face mask
- 12 Child face mask
- 13 Air filter
- 14 Micro USB cable
- 15 AC adapter
- 16 Assembling nebuliser kit
- 17 Replacing air filter

Dear Customer,
The nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract.

- Explanation of Symbols**
 - Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
 - Read the instructions carefully before using this device.
 - Type BF applied part
 - Class II equipment
 - Serial number
 - Reference number
 - Manufacturer
 - ON
 - OFF

CE 0123 CE Marking of Conformity
2. Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- Device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect use.
- Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mist or inflammable with oxygen or nitrous oxide, and do not use for anesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.
- Water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Comply with the safety regulations concerning the electrical devices and in particular:
 - Never touch the device with wet or moist hands.
 - Place the device on a stable and horizontal surface during its operation.
 - Do not pull the power cord or the device itself to unplug it from the power socket.
- Water plugging in the device, make sure that the electrical rating shown on the rating plate on the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit the wall socket, contact a qualified electrician to replace a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended, if it is not indispensable. It is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, and that they are certified and qualified products.
- Do not leave the unit plugged in when not in use, unplug the device from the wall socket when it is not operating.
- The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, of which the manufacturer cannot be held responsible. Do not replace the USB cable nor the AC adapter of this device. In case of damage, have them replaced only in a service centre authorised by the manufacturer.
- The power supply cord should always be fully ungrounded in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
- Only use the medication prescribed for you by your doctor and follow the instructions with regard to dosage, duration and frequency of the therapy.
- Depending on the pathology, only use the treatment that is recommended by your doctor.
- Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the burblings into the nose, but only positioning them as close as possible to the nostrils.
- Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.
- Do not position the equipment so that it is difficult to operate the disconnection device.
- For greater hygienic safety, do not use the same nebuliser for more than one patient.
- Never bend the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile phones or Wi-Fi routers.
- Keep a minimum distance of 3.3 m from such devices when using this device.

3. Preparation and Usage of this Device
Prior to using the device for the first time, we recommend cleaning it as described in the section "Cleaning and Disinfecting".

1. Assemble the nebuliser kit (9). Ensure that all parts are correctly assembled and in good working order.

2. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in children. Use of this device is not intended as a substitute for a consultation with your doctor.

4. Cleaning and Disinfecting
Thoroughly clean all components to remove medication residues and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the equipment.

5. Maintenance, Care, and Service
Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see forward).

Replace the nebuliser (2) if the long period of inactivity, in cases where it shows deformities, breakage, or when the vapouriser head (7-a) is obstructed by the nebuliser dust, etc. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters! Do not use the device without filter!

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Use the disinfecting and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each use, wash the nebuliser (2) and the mouthpiece (9) and the nose piece (10) by using tap water for 30 minutes. Do not use hot water for longer than 30 minutes, after your usual treatment.

The device requires no calibration.

No modification to the device is permitted.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Read the instructions carefully before using this device.

Type BF applied part

Class II equipment

Serial number

Reference number

Manufacturer

ON

OFF

Protection against small foreign objects and harmful effects due to the ingress of water

CE Marking of Conformity

Important Safety Instructions

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

Device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect use.

Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mist or inflammable with oxygen or nitrous oxide, and do not use for anesthesia and lung ventilation.

This device should only be used with original accessories as shown in these instructions.

Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.

Water and moisture

extreme temperatures

impact and dropping

contamination and dust

direct sunlight

heat and cold

Comply with the safety regulations concerning the electrical devices and in particular:

Never touch the device with wet or moist hands.

Place the device on a stable and horizontal surface during its operation.

Do not pull the power cord or the device itself to unplug it from the power socket.

Water plugging in the device, make sure that the electrical rating shown on the rating plate on the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit the wall socket, contact a qualified electrician to replace a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended, if it is not indispensable. It is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, and that they are certified and qualified products.

Do not leave the unit plugged in when not in use, unplug the device from the wall socket when it is not operating.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, of which the manufacturer cannot be held responsible. Do not replace the USB cable nor the AC adapter of this device. In case of damage, have them replaced only in a service centre authorised by the manufacturer.

The power supply cord should always be fully ungrounded in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed for you by your doctor and follow the instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the burblings into the nose, but only positioning them as close as possible to the nostrils.

Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same nebuliser for more than one patient.

Never bend the nebuliser over 60°.

Do not use this device close to strong electromagnetic fields such as mobile phones or Wi-Fi routers.

Keep a minimum distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in children. Use of this device is not intended as a substitute for a consultation with your doctor.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residues and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the equipment.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see forward).

Replace the nebuliser (2) if the long period of inactivity, in cases where it shows deformities, breakage, or when the vapouriser head (7-a) is obstructed by the nebuliser dust, etc. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters! Do not use the device without filter!

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Use the disinfecting and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each use, wash the nebuliser (2) and the mouthpiece (9) and the nose piece (10) by using tap water for 30 minutes. Do not use hot water for longer than 30 minutes, after your usual treatment.

The device requires no calibration.

No modification to the device is permitted.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Read the instructions carefully before using this device.

Type BF applied part

Class II equipment

Serial number

Reference number

Manufacturer

ON

OFF

Protection against small foreign objects and harmful effects due to the ingress of water

CE Marking of Conformity

Important Safety Instructions

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

Device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect use.

Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mist or inflammable with oxygen or nitrous oxide, and do not use for anesthesia and lung ventilation.

This device should only be used with original accessories as shown in these instructions.

Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.

Water and moisture

extreme temperatures

impact and dropping

contamination and dust

direct sunlight

heat and cold

Comply with the safety regulations concerning the electrical devices and in particular:

Never touch the device with wet or moist hands.

Place the device on a stable and horizontal surface during its operation.

Do not pull the power cord or the device itself to unplug it from the power socket.

Water plugging in the device, make sure that the electrical rating shown on the rating plate on the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit the wall socket, contact a qualified electrician to replace a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended, if it is not indispensable. It is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, and that they are certified and qualified products.

Do not leave the unit plugged in when not in use, unplug the device from the wall socket when it is not operating.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, of which the manufacturer cannot be held responsible. Do not replace the USB cable nor the AC adapter of this device. In case of damage, have them replaced only in a service centre authorised by the manufacturer.

The power supply cord should always be fully ungrounded in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed for you by your doctor and follow the instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the burblings into the nose, but only positioning them as close as possible to the nostrils.

Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same nebuliser for more than one patient.

Never bend the nebuliser over 60°.

Do not use this device close to strong electromagnetic fields such as mobile phones or Wi-Fi routers.

Keep a minimum distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in children. Use of this device is not intended as a substitute for a consultation with your doctor.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residues and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the equipment.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see forward).

Replace the nebuliser (2) if the long period of inactivity, in cases where it shows deformities, breakage, or when the vapouriser head (7-a) is obstructed by the nebuliser dust, etc. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters! Do not use the device without filter!

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Use the disinfecting and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each use, wash the nebuliser (2) and the mouthpiece (9) and the nose piece (10) by using tap water for 30 minutes. Do not use hot water for longer than 30 minutes, after your usual treatment.

The device requires no calibration.

No modification to the device is permitted.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Read the instructions carefully before using this device.

Type BF applied part

Class II equipment

Serial number

Reference number

Manufacturer

ON

OFF

Protection against small foreign objects and harmful effects due to the ingress of water

CE Marking of Conformity

Important Safety Instructions

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

Device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect use.

Retain instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mist or inflammable with oxygen or nitrous oxide, and do not use for anesthesia and lung ventilation.

This device should only be used with original accessories as shown in these instructions.

Do not use this device if you think it is damaged or notice anything unusual.

Never open this device.

This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.

Water and moisture

extreme temperatures

impact and dropping

contamination and dust

direct sunlight

heat and cold

Comply with the safety regulations concerning the electrical devices and in particular:

Never touch the device with wet or moist hands.

Place the device on a stable and horizontal surface during its operation.

Do not pull the power cord or the device itself to unplug it from the power socket.

Water plugging in the device, make sure that the electrical rating shown on the rating plate on the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit the wall socket, contact a qualified electrician to replace a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended, if it is not indispensable. It is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits, and that they are certified and qualified products.

Do not leave the unit plugged in when not in use, unplug the device from the wall socket when it is not operating.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, of which the manufacturer cannot be held responsible. Do not replace the USB cable nor the AC adapter of this device. In case of damage, have them replaced only in a service centre authorised by the manufacturer.

The power supply cord should always be fully ungrounded in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed for you by your doctor and follow the instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the burblings into the nose, but only positioning them as close as possible to the nostrils.

Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same nebuliser for more than one patient.

Never bend the nebuliser over 60°.

Do not use this device close to strong electromagnetic fields such as mobile phones or Wi-Fi routers.

Keep a minimum distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in children. Use of this device is not intended as a substitute for a consultation with your doctor.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residues and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the equipment.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see forward).

Replace the nebuliser (2) if the long period of inactivity, in cases where it shows deformities, breakage, or when the vapouriser head (7-a) is obstructed by the nebuliser dust, etc. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters! Do not use the device without filter!

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Use the disinfecting and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each use, wash the nebuliser (2) and the mouthpiece (9) and the nose piece (10) by using tap water for 30 minutes. Do not use hot water for longer than 30 minutes, after your usual treatment.

The device requires no calibration.

No modification to the device is permitted.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Read the instructions carefully before using this device.

Type BF applied part

Class II equipment

Serial number

Reference number

Manufacturer

ON

OFF

Protection against small foreign objects and harmful effects due to the ingress of water

CE Marking of Conformity

Important Safety Instructions

Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future

