
RU
NO LV LT EE
EN SV FI DE

Compressor Nebuliser
NEB 50B

microlife

NEB 50B

IS 008 N-09-0316

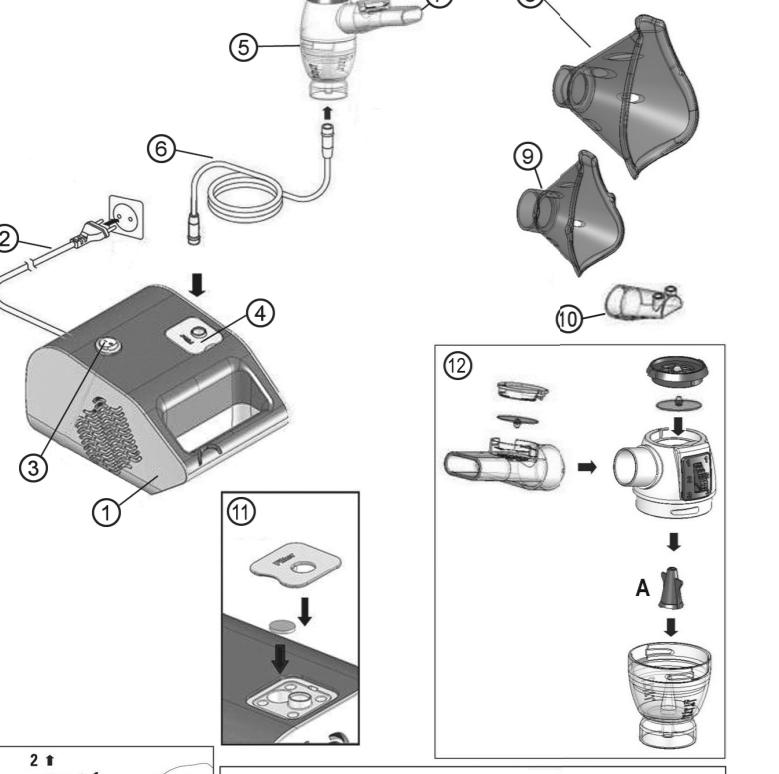Guarantee Card

Name of Purchaser / Inköparens namn / Ostajan nimi
/ Forhandlars navn / Kjøpers navn / Příroda výrobce /
Pirkijo pavardé / Ostja nimi / ФИО покупателя

Serial Number / Serienummer / Sarjanumero / Serien-
nummer / Serienummer / Séríjus nummus / Serios
numeris / Serienummer / Серийный номер

Date of Purchase / Inköpsdatum / Ostapäivämäärä /
Kehstato / Kjørstato / Ilegades datums / Pardavimo
data / Ostukupiéra / Дата покупки

Specialist Doctor / Aterforrådare / Alten kauppias /
Special-forhandler / Specialist-forhandler / Specialists
praktis / Pardavimo lääkäri / Ametlik müügiseisnaja /
Специалист-лекарь

3A Health Care S.r.l.
Via Marziale Cerutti Loc. San Tomaso
25017 Lornato (BS) - Italy
Tel. 0039 030 913377
Fax 0039 030 991914

CE0051

Advanced Compressor Nebuliser

- ① Piston compressor
- ② Power lead
- ③ ON/OFF switch
- ④ Air filter compartment
- ⑤ Nebuliser with switchable nebulisation rate
- ⑥ Air hose
- ⑦ Mouthpiece
- ⑧ Adult face mask
- ⑨ Child face mask
- ⑩ Nose piece
- ⑪ Replacing air filter
- ⑫ Assembly nebuliser kit

Customer.

This professional nebuliser is a high-quality device for inhalation therapy for the treatment and care of upper and medium/lower respiratory tract.

The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.

Please read through these instructions carefully so that you understand all functions and safety information. We would like to be happy to help you if you have any problems or want to order spare parts please contact your local Microlife Customer Service.

Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – Microlife AG!

Type BF applied part

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer must be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- This device comprises sensitive components and must be treated with caution. Observe the storage and handling instructions described in the « Technical Specification » section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never bend the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 cm from such devices when using this device.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case the device is supplied with cables or tubes.
- Electronic accessories must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- ⚠ Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section « Cleaning and Disinfecting ».

1. Assemble the nebuliser unit (2). Ensure that all parts are complete. Use the wheel on the upper part of the nebuliser to select the position most suitable for the therapy to be performed (see chapter 3).

2. Fill the nebuliser with the therapeutic solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.

3. Connect the nebuliser with the air hose (6) to the compressor (1) and plug the power lead (2) into the socket (230V - 50Hz).

4. Switch the ON/OFF switch (3) to position «I» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over the mouth and nose.

- This device gives you a better drug delivery to the lungs.

- Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.

- Use all accessories including the nose piece (10) as prescribed by your doctor.

5. Breathe in and out calmly during therapy. Sit in a relaxed position with the upper body upright. Do not lean forward while inhaling. Stop inhalation if you feel unwell.

6. After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «O» to turn off the device and unplug it from the socket.

7. Empty the remaining medication from the nebuliser and clean the device as described in the section « Cleaning and Disinfecting ».

- This device was designed for intermittent use for 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

3. Choosing between 3 different Nebulisation Rates

The nebuliser generates an aerosol with variable characteristics to adapt to the patient's respiratory pathology. It can produce an aerosol with the granulometric characteristics that will deposit the maximum amount of the drug in the respiratory tract being treated.

Choose one of the 3 following therapeutic treatment settings by regulating the wheel on the nebuliser's (5) upper part:

- **Upper air passages** (nose, throat, etc.): Position the wheel towards the bottom at number 3: the aerodynamic diameter of the aerosol particles will be from 10.33 - 14.56 micron.
- **Tracheal / bronchial tract**: Position the wheel in the center at number 2: the aerodynamic diameter of the aerosol particles will be from 6.21 - 9.45 micron.
- **Deep air passages**: Position the wheel towards the top, at number 1: the aerodynamic diameter of the aerosol particles will be from 1.14 - 5.32 micron.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (6) should be cleaned with a clean, moist cloth.

• Always wash your hands well before cleaning and disinfecting the accessories.

• Do not expose the compressor to water or heat.

• Replace the air hose for each treatment with a new patient or in case of impurities.

⚠ Unplug the device before cleaning.

Cleaning with water

• Wash all nebuliser components (except air hose) with warm water (max. 60 °C) for about 5 minutes either (if necessary) a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.

• Rinse thoroughly making sure that all deposits are washed away and leave to dry.

Disinfecting

• All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.

• Disinfectants are usually available at pharmacies.

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Ta vare på helsen – Microlife AG!