### Portable Compressor Nebuliser

- 1) Piston compressor
- 2) ON/OFF button
- 3) Power indicator
- 4) USB Port
- 5) Air filter compartment
- 6) Air outlet
- 7) Nebuliser
- 8) Vapouriser head
- 9) Air hose
- 10) Mouthpiece
- 11) Nose piece
- 12) Adult face mask
- 13) Child face mask
- 14) Air filter
- 15) Micro USB cable
- 16) AC adapter
- 17) Assembling nebuliser kit
- 18) Replacing air filter

Dear Customer, This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacist will be able to give you the correct instructions for use in your country. Alternatively, visit the internet at [www.microlife.com](http://www.microlife.com) where you will find a wealth of available information on our products. Use a soft and dry cloth with non-abrasive cleaners to clean the compressor.

- Explanation of Symbols**
  - Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
  - Read the instructions carefully before using this device.
  - Type BF applied part
  - Class II equipment
  - Serial number
  - Reference number
  - Manufacturer
  - ON
  - OFF
  - IP22 Protection against solid foreign objects and harmful effects due to the ingress of water

### Compressor Nebuliser

### NEB NANO basic

### CE0123

- Important Safety Instructions**
  - Follow instructions for use. This document provides important product operation and safety information regarding this device. Read the instructions carefully before using the device and keep for future reference.
  - This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
  - Use the nebuliser in a safe place for future reference.
  - Do not operate the unit in presence of any anesthetic mist inflammable with oxygen or nitrous gas and long ventilation.
  - This device is not suitable for anaesthesia and lung ventilation.
  - This device should only be used with original accessories as shown in these instructions.
  - Do not use this device if you think it is damaged or notice anything unusual.
  - Never open this device.
  - This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.

### 2. Important Safety Instructions

- water and moisture
- extreme temperatures
- impact and dropping
- contamination and dust
- direct sunlight
- heat and cold
- Comply with the safety regulations concerning the electrical devices and in particular:
  - Never touch the device with wet or moist hands.
  - Place the device on a stable and horizontal surface during operation.
  - Do not pull the power cord or the device itself to unplug it from the power socket.
  - The power plug is a separate element from the grip power; keep the plug accessible when the device is in use.

- Before plugging in the device, make sure that the electrical ratings shown on the rating plate on the bottom of the unit, corresponds to the mains rating.
- In case the power plug provided with the device does not fit your wall socket, contact a qualified person for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention that they do not exceed the maximum power limits.
- Do not use the device if you are using a power extension cable.
- Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operating.
- The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, for which the manufacturer cannot be held responsible. Do not replace the USB cable nor the AC adapter of this device. In case of damage, have them replaced by a service centre authorised by the manufacturer.
- The power supply cord should always be fully unwound in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
- Only use the device and medication prescribed for you by your doctor and follow the instructions with regard to dosage, duration and frequency of the therapy.
- Depending on the pathology, only use the treatment type as recommended by your doctor.
- Only use the nose piece expressly indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.
- Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.
- Do not position the equipment so that it is difficult to operate the disconnection device.
- For greater hygienic safety, do not use the same accessories for more than one person.
- Never bend the nebuliser over 60°.
- Do not use the device close to strong electromagnetic fields such as mobile phones.
- Keep a minimum distance of 3.3 m from such devices when using this device.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case the device is supplied with cables or tubes.
- Use of this device is not intended as a substitute for a consultation with your doctor.

### 3. Preparation and Usage of This Device

- Specialized Dealer / Rivenditore / Categoria rivenditori / Fachändler / Упознат Салар / Vendedor especializado / Revendedor autorizado / Specialist Dealer / Ekipovnikovozvrhovnoopromotno / ايجير الاحصائى / ابيجير الاحصائى
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case the device is supplied with cables or tubes.
- Use of this device is not intended as a substitute for a consultation with your doctor.

### 4. Cleaning and Disinfecting

- Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the compressor.
- Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected from the power outlet.
- No modification to the device is permitted.
- The device requires no calibration.
- No modification to the device is permitted.

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