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North / Central / South America  
Microlife USA, Inc.  
1617 Gulf to Bay Blvd., 2nd Floor Ste A  
Clearwater, FL 33755 / USA  
Tel. +1 727 442 5353  
Fax +1 727 442 5377  
Email msa@microlifeusa.com  
www.microlife.com

IB FH 600 V25-1 2020, Revision Date 2020-05-06

Microlife Foot Warmer FH 600

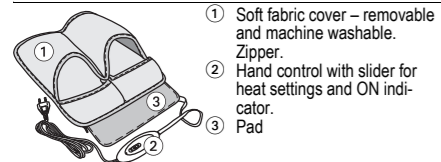
EN	→	2
FR	→	3
ES	→	5
PT	→	7
PL	→	9
HU	→	11
DE	→	13
SV	→	15
FI	→	16
TR	→	17
GR	→	19
IT	→	21
DA	→	23
NL	→	24
LT	→	26
LV	→	27
EE	→	28
BG	→	29
RO	→	31
CZ	→	33
SK	→	34
SR	→	35
HR	→	36
AR	→	37
FA	→	38

- Please read this manual carefully before using this device.
- This device is not intended for medical use in hospitals: For household use only.
- To avoid any risk of electric shock when using the pad please observe the following:
  - Do not use if wet. If the pad has become damp or wet, completely dry the pad before use.
  - Only use the pad in dry rooms, not e.g. in the bathroom.
  - Check that the pad is undamaged each time before use, i.e. do not use it if there is visible damage to the hand control, switches, plug or the pad itself.
  - Do not insert pins or other sharp objects.
- To avoid overheating:
  - Do not leave the pad switched on unsupervised; switch it off when you leave the room.
  - Do not use the pad while folded.
  - Avoid creasing the pad.
  - **Do not cover the hand control!** To ensure that the hand control functions correctly, do not place it on or under the heating pad or cover it with other objects.
  - The cable should not be wound around the hand control.
- Do not use on a person incapable of movement, a baby, a person insensitive to heat, or animals. Do not let children play with the device.
- Ensure that children do not use this device unsupervised. Children under 8 years of age must not use this device unless a parent or guardian has set the control to the lowest temperature setting 1.
- This device may be used by children aged 8 or over and by people who have reduced physical, sensory or mental abilities or inadequate experience and/or knowledge, provided they are supervised or have been instructed on how to use the device safely and have understood the resulting dangers.
- Excessive application at a high setting can result in skin burns.
- When the device is likely to be used for an extended period, i.e. by the user falling asleep, controls should be adjusted to a setting recommended for continuous use.
- Examine the device frequently for signs of wear or damage. If there are signs of wear or damage or the device has been misused, please contact the manufacturer or authorised agent prior to any further use.
- If the power cable is damaged, it must be replaced by the manufacturer, an authorised agent or similarly qualified persons in order to avoid any hazards.
- Do not attempt to open or repair the device yourself. Special tools are required to re-assemble the device.
- This device emits very weak electrical and magnetic fields which may disturb the function of your cardiac pacemaker. Therefore, we recommend you to consult your physician and the manufacturer of your cardiac pacemaker before using this device.
- Allow the device to cool down before folding and storing it in a dry place. Do not place heavy objects on it to prevent buckling.

- ⚠ **Not suitable for children between 0-3 years!** Children under 3 years of age are unable to respond to overheating.
- ♻ Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

## 1. How to use a Microlife foot warmer

- ① Soft fabric cover – removable and machine washable. Zipper.
- ② Hand control with slider for heat settings and ON indicator.
- ③ Pad

⚠ **WARNING:** Never use the pad without the cloth cover. Allow some air circulation between the pad and skin during use.

- Do not use with shoes.
  - The device is not to be used for warming animals.
1. Put the cloth cover ① on a flat surface. Open the zipper.
  2. Insert the pad ③ completely into the cover, fitting one side at a time. Make sure the cable and hand control ② remain outside the cover.
  3. The pad will lie flat, filling the entire cover. Close zipper.
  4. Plug into a 220-240 volt AC mains socket and set the hand control to the desired heat setting using the slider.
    - Select the temperature setting on the slider – this switches on the heat pad. An indicator will illuminate, showing that the appliance is connected to the power supply and is switched on.
    - Setting 1 is recommended for continuous use.
    - The pad is switched off by moving the slider to the «0» position. The indicator light will then turn off.
  5. **The hand control will automatically turn off the heater after approximately 90 minutes**, however, the indicator light will remain illuminated. To reset the timer and start a further heating period, move the slider to the off «0» position and then back to the desired heat setting.
  6. Unplug when finished.


## 2. Cleaning Instructions

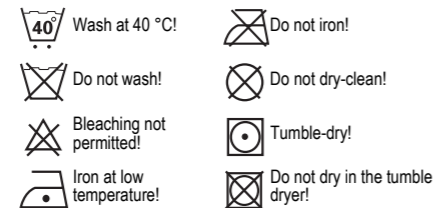⚠ **WARNING:** Always unplug the power cable from the mains socket before undertaking any cleaning of the product.

1. The cover can be machine washed after removal from the pad according to the washing instructions given on the cover.
2. The pad must **not** be machine washed or dry-cleaned. It can be cleaned with a brush when dry.
3. Do not allow the hand control to get wet. Do not use any detergents or solvents as they may damage the device.
4. Never hold the hand control under water. Otherwise moisture may enter the hand control and cause damage.
5. Check the hand control for signs of damage. Dry the pad and its fabric cover completely before switching it on again.

🧼 Cleaning and user maintenance shall not be made by children without supervision.

## 3. Explanation of symbols

## 4. Guarantee

This product is guaranteed for **2 years** from the date of purchase. This guarantee covers the hand control, cable, and the pad itself. This guarantee does not cover incorrect use, accidents or damage resulting from failure to follow instructions, or modifications to the device not carried out by the manufacturer. **The guarantee is only valid upon presentation of the guarantee card completed by the dealer.**

## 5. Technical Specifications

**Model:** FH 600  
**Voltage:** 220 - 240 V   
**Power:** 50 W  
**Frequency:** 50 Hz

Technical alterations reserved.

## 6. www.microlife.com

Detailed user information about our products and about our services can be found at [www.microlife.com](http://www.microlife.com).

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