Description of this Thermometer

- 1 ON/OFF button
2 Display
3 Battery compartment cover
4 Measuring sensor / measuring tip
5 Cleaning and disinfecting area (entire thermometer surface)

Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device.
This device is only to be used for measuring human body temperature through oral, rectal or axillary.
Do not use this device if you think it is damaged or notice anything unusual.
We recommend cleaning this device according to the cleaning instructions before first use for personal hygiene.
The minimum measurement time until the beep is heard must be maintained without exception.
Consider that different measurement locations may require continued measuring even after the beep, see section «Measuring methods / Normal body temperature».
Do not attempt rectal measurements on persons with rectal disorders.
Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed.
Do not use this device close to strong electromagnetic fields.
Protect the device from impact and dropping!
Avoid bending the thermometer probe more than 45°.
Use only the commercial disinfectants listed in the section «Cleaning and Disinfecting» to clean the device to avoid damage to the device.
We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped).

- WARNING: The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only.
Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
Read the instructions carefully before using this device.
Type BF applied part

Turning on the Thermometer

To turn on the thermometer, press the ON/OFF button 1; a short beep signals «thermometer ON». A display test is performed. All segments should be displayed.
Then at an ambient temperature of less than 32 °C, an «--» and a flashing «°C» appear at the display field 2. The thermometer is now ready for use.

Function Test

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

Using the Thermometer

Choose the preferred measuring method. When taking a measurement, the current temperature is continuously displayed and the «°C» symbol flashes. If the beep is heard 10 times and the «°C» is no longer flashing, this means that the measured increase in temperature is less than 0.1 °C in 8 seconds and that the thermometer can now be read.
To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button 1. Otherwise the thermometer will automatically turn off after about 10 minutes.

Storage of Measured Values

Press ON/OFF button 1 for 3 seconds, when turning on the thermometer. The last measurement will be displayed together with a «M» for memory. After 2 seconds the thermometer will automatically switch off.

Measuring methods / Normal body temperature

- In the armpit (axillary) / 34.7 - 37.3 °C
Wipe the underarm with a dry towel. Place the measuring sensor 4 under the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body.
In the mouth (oral) / 35.5 - 37.5 °C
Do not eat or drink anything hot or cold 10 minutes before the measurement.
In the anus (rectal) / 36.6 - 38.0 °C
Carefully insert the measuring sensor 4 of the thermometer 2 to 3 cm into the anal aperture.
Cleaning and Disinfecting

Battery Replacement

When the «▼» symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover 3 from the thermometer. Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand.

Technical Specifications

Table with 2 columns: Property and Value. Includes Type, Measurement range, Measurement accuracy, Operating conditions, Storage conditions, Battery, Battery lifetime, IP Class, Reference to standards, and Expected service life.

This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

Guarantee

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee: Transport costs and risks of transport, Damage caused by incorrect application or non-compliance with the instructions for use, Damage caused by leaking batteries, etc.

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Microlife AG, Espenstrasse 139 9443 Widnau / Switzerland