## Description of this Thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor / measuring tip
- ⑤ Cleaning and disinfecting area (thermometer probe only)

EN

## Battery Replacement

When the  symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover ③ from the thermometer. Insert the new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

## Technical Specifications

Type:	Maximum thermometer
Measurement range:	32.0 °C to 43.9 °C Temp. < 32.0 °C: display  for low (too low) Temp. > 43.9 °C: display  for high (too high)
Measurement accuracy:	± 0.1 °C between 34 °C and 42 °C
Operating conditions:	10 - 40 °C; 15 - 95 % relative maximum humidity
Storage conditions:	-25 - +60 °C; 15 - 95 % relative maximum humidity
Battery:	LR41 (1.5V) / SR41 (1.55V) approx. 4500 measurements (using a new battery) IP22
IP Class:	EN 12470-3, clinical thermometers; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11
Reference to standards:	5 years or 10000 measurements

## Expected service life:

This device complies with the requirements of the Medical Device Directive 93/42/EEC.  
Technical alterations reserved.

## Guarantee

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Battery.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: [www.microlife.com/support](http://www.microlife.com/support)

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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ES

## Turning on the Thermometer

To turn on the thermometer, press the ON/OFF button ①; a long beep signals «thermometer ON». A display test is performed. All segments should be displayed.

Then  and a flashing  appear at the display field ②. The thermometer is now ready for use.

## Function Test

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by  on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

## Using the Thermometer

Choose the preferred measuring method. When taking a measurement, the current temperature is continuously displayed and the  symbol flashes. If the beep is heard 10 times and the  is no longer flashing, this means that the measured increase in temperature is less than 0.1 °C in 5 seconds.

To prolong the battery life, turn off the thermometer by briefly pressing the ON/OFF button ①. Otherwise the thermometer will automatically turn off after about 10 minutes.

## Reviewing the last Measurement

When turning on the thermometer, the last measurement will be displayed with a  for memory.

## Measuring methods / Normal body temperature

## In the armpit (axillary) / 34.7 - 37.3 °C

Wipe the underarm with a dry towel. Place the measuring sensor ④ under the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait at least 5 minutes, regardless of the beep sound.

In the mouth (oral) / 35.5 - 37.5 °C

Do not eat or drink anything hot or cold 10 minutes before the measurement. The mouth should remain closed up to 2 minutes before starting a reading.

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor ④ must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

If this is not possible due to blocked airways, another method for measuring should be used.

Approx. measuring time: 30 seconds!

In the anus (rectal) / 36.6 - 38.0 °C

Carefully insert the measuring sensor ④ of the thermometer 2 to 3 cm into the anal aperture.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

Approx. measuring time: 30 seconds!

## Cleaning and disinfecting

For disinfection in home use environment, use a 70% Isopropyl alcohol swab, or a cotton tissue moistened with 70% Isopropyl alcohol to wipe surface pollutants off the thermometer probe (note: consider the application and safety instruction of the disinfectant manufacturer). Always start wiping from the end of the thermometer probe (approx. at the middle of the thermometer) towards the thermometer tip. Afterwards the entire thermometer probe (see number ⑤ in the drawing) should be immersed in 70% Isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use. Avoid immersing or wiping the display to protect it from fading.

The thermometer is not intended for professional use.

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