Dear Customer, This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts please contact your local MicroLife Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information and product literature. Refer to the relevant instructions in a safe place for future reference. Stay healthy – MicroLife AG!

- Read the instructions carefully before using this device.
- Type BF applied part

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.

This device is not suitable for anaesthesia and lung ventilation. This device should only be used with original accessories as shown in these instructions.

Do not use this device and the AC adapter if you think they are damaged or notice anything unusual. Never open this device and the AC adapter.

This device comprises sensitive components and should be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.

- Protect it from:
  - water and moisture
  - extreme temperatures
  - impact and dropping
  - contamination and dust
  - direct sunlight
  - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Only use the AC adapter model SJ-1220-E (cod. 3A3309). Do not use other models of AC adapters. Keep the cord away from hot surfaces.
- Never plug in or unplug the AC adapter with wet hands. Never use this device (with the AC adapter inserted) close to water. Do not get the device wet or immerse it in any liquids. If by accident it falls into water, pull it out of the wall socket immediately before recovering it.
- Never bend the nebuliser over 90°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes. Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

## 2. Preparation and Usage of This Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nebuliser kit (13). Ensure that all parts are complete.
- Fill the nebuliser (12) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser (12) with the air hose (2) to the air hose connector (4) of the compressor (1), as shown in figure 14.
- Connect the AC adapter (12) to the AC adapter socket (6) of the device and plug the power plug of AC adapter into the wall socket.
- Switch the ON/OFF switch (5) to position «1» to turn on the device and place the mouthpiece (8) in the mouth or fit one of the face masks over mouth and nose.

- The mouthpiece gives you a better drug delivery to the lungs.
- Choose between adult (10) or child face mask (11) and make sure that it encloses the mouth and nose area completely.
- Use all accessories including the nose piece (9) as prescribed by your doctor.

Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. Do not lie down while inhaling. Stop inhalation if you feel unwell.

- After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (5) to position «0» to turn off the device. Disconnect the AC adapter (12) from the wall socket and from the device.
- Disconnect the air hose (2) from the nebuliser (12) and from the air hose connector (4) of the device.
- Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

Name of Purchaser / Inköparens namn / Ostajnar nimi / Forhandlers navn / Kjøpers navn / Pircbja vards / Pirkje pavarde / Ostja nimi / Ф.И.О. покупателя

Serial Number / Serienummer / Sarjainumero / Serienummer / Серийный номер

Date of Purchase / Inköpsdatum / Ostöpäivämäärä / Købsdato / Kjøpsdato / Iegades datums / Pardavimo data / Ostukuapevi / Дата покупки

Specialist Dealer / Återförärlare / Alan kauppias / Special-forhandler / Spetsialist forhandler / Spetsialist -pärstavis / Pardavus istaiga / Аmetik mühesindaja / Специализированный дилер

3A Health Care S.r.l. Via Salsola Centro, Loc. San Tomaso 25017 Lonate (BS) – Italy Tel. 0039 030 9133177 Fax 0039 030 9919114

CE 0051

## Portable Compressor Nebuliser

- Piston compressor
- Air hose
- Air filter compartment
- Air hose connector
- ON/OFF Switch
- AC adapter socket
- Nebuliser «NEB PRO»
- Vapouriser head
- Membrane
- Mouthpiece with valve
- Nose piece
- Adult face mask
- Child face mask
- AC adapter
- Assembling nebuliser kit
- Assembling the device
- Filtering the nebuliser
- Breath-enhanced nebulisation
- Opening the air filter compartment
- Replacing air filter
- Air filter

A synchronized nebulisation is not possible using this treatment option.

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (2) should be cleaned with a clean, moist cloth.

Always wash your hands well before cleaning and disinfecting the accessories. Do not expose the compressor to water or heat.

Replace the air hose for each treatment with a new patient or in case of impurities.

Always unplug the AC adapter from the power supply before cleaning.

Wash all nebuliser components (except air hose) under warm tap water (max. 60 °C) for about 5 minutes adding if necessary a small amount of detergent following dosage and use limitations as provided by detergent manufacturer.

Rinse thoroughly making sure that all deposits are washed away and leave to dry.

All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer. Disinfectants are usually available at pharmacies.

Wash all nebuliser components (except air hose and mask) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). EN554/ISO 1134.

The sterilisation package conforms to EN868/ISO 11607 and be suitable for steam sterilisation.

After sterilisation always let all components cool down to ambient temperature before further use.

Do not repeat sterilization cycle when components are still warm.

## 3. Nebuliser «NEB PRO» treatment options

This device is equipped with the innovative nebuliser «NEB PRO» (7) which includes the breath-enhanced valve system (8). All common medication for inhalation therapy can be nebulised effectively.

Advanced treatment This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (16-A), reducing medication loss during exhalation (16-B). Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.

Mount the membrane on the valve-holder and use the mouthpiece with expiratory valve (9).

Follow the usage steps as described in «Section 2.»

Only using the mouthpiece with expiratory valve (9) enables the synchronized functioning of the nebuliser.

Short treatment The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.

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Saglabājiet lietotāšanas pamācību drošā vietā turpmākai uzziņai. Lai Jums laika vesellība – Microfile AG!