microlife®

WatchBP® Single-Patient Cuff

 Microlife Corporation
9F, No. 431, RuiGuang Road, NeiHu
Taipei, 114, Taiwan, China
www.microlife.com

 Microlife UAB
P. Lukšio g. 32
08222 Vilnius
Lithuania
eurep@microlife.lt

 Microlife AG
Espanstrasse 139
9443 Widnau
Switzerland

WBP Single-Patient cuff V32 3325
Revision Date: 2025-07-02
Issuing Date: July 2025

EN → 1	EE → 17	SL → 33	DE → 49
SV → 3	RU → 19	SR → 35	ES → 51
FI → 5	UA → 21	PL → 37	PT → 53
DA → 7	KZ → 23	HU → 39	GR → 55
NO → 9	BG → 25	HR → 41	TR → 57
IS → 11	RO → 27	NL → 43	HE → 59
LV → 13	CZ → 29	FR → 45	AR → 61
LT → 15	SK → 31	IT → 47	FA → 63

1. Introduction

Document scope

Read the instructions carefully before using the cuff.

Refer to this document and the instruction for use of your blood pressure monitor for important product operation and safety information about the cuff. Keep these documents for future reference.

2. Disclaimers

MicroLife® is a registered trademark of MicroLife Corporation.
WatchBP® is a registered trademark of MicroLife Corporation.

3. Important information

Device description

This arm cuff is an accessory of upper arm oscillometric digital non-invasive blood pressure monitors.

WARNING

- Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.
- Keep the cuff and its parts away from children and people incapable of using the device and cuff. Beware of the risks of accidental ingestion of small parts and strangulation with the cuff tube.

CAUTION

- Caution items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.
- Use this cuff only with compatible devices and connectors from WatchBP. Using non-compatible accessories may compromise the safety and performance of the device.
- Always use the arm cuff of range appropriate for the mid arm circumference of the patient. Using a cuff of incorrect size may cause measurement error and/or incorrect readings.
- Stop using this device and cuff and consult with your doctor if you experience skin irritation or discomfort.
- DO NOT use this device, cuff, or parts after the expiration of its stated service life.

Adverse events and reporting

Please report any serious incident, injury or adverse event that has occurred in relation to the device to the manufacturer/ European authorized representative (EU REP), and to the competent authority.

4. Device information

Package contents

20 x WatchBP Single-Patient cuff
50 cm tube with connectors
170 cm tube with connectors
Instruction Manual

Inspect the device, cuff and other parts for damage. DO NOT USE the device, cuff or parts if they appear damaged or operating abnormally.

5. Device Setup

Place and connect the cuff

- Place cuff to upper arm such that the lower edge of the cuff is 1-2 cm from the elbow hinge line. Align artery mark for positioning brachial artery.
- Wrap the cuff snugly around upper arm.
- Connect the tube to blood pressure monitor.

A loose connection will result in inaccurate readings, and/or error messages.

To change the connector, pull the connector out of the cuff tube and insert the required connector into the cuff tube as far as it will go.

The tube on the WatchBP device part is reusable. Please change the cuff via the flying connector.

6. Maintenance

Additional information about maintenance and disposal of your WatchBP blood pressure cuff: www.microlife.com/watchbp-cuff

Storage

When not in use:

Keep the cuff in a dry, cool place away from sunlight, with ambient conditions within the temperature and humidity ranges described in the «Specifications and compliance» section.

Do not bend or crease the cuff tube excessively.

Do not immerse in any liquids. Fluid in the tubing or air bladder will damage monitoring equipment.

7. Specifications and compliance

Technical specifications

Technical specifications subjected to change without notice.

Reference number	Model number	Dimensions
WatchBP Single-Patient cuff S	CA0-S0-514	≤407 × ≤123 mm
WatchBP Single-Patient cuff M	CA0-M0-513	≤500 × ≤145 mm
WatchBP Single-Patient cuff L	CA0-L0-512	≤600 × ≤175 mm

Product description:

Single-Patient cuff for digital non-invasive blood pressure monitor

Operating conditions:

10 - 40 °C / 50 - 104 °F
15 - 90 % relative maximum humidity
700 hPa - 1060 hPa

Storage and transport conditions:

-20 - +55 °C / -4 - +131 °F
15 - 90 % relative maximum humidity

Expected service life:

100 measurements (one measurement cycle consists of a full inflation and deflation of the cuff)

Compliance information

This device complies with the requirements of the Medical Device Regulation (EU)2017/745.

8. Guarantee

The cuff has a functional guarantee (bladder tightness) for 90 days.

- Not included are sleeve and cuff enclosure due to wear.
- The guarantee is only valid on presentation of the purchase receipt.
- The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

9. Symbols and definitions

Medical device

CE Marking of Conformity

Importer

Authorized representative
in the European Union

Authorized representative in Switzerland

Manufacturer

Country of manufacture
(Date of manufacture if date printed next to symbol)

Model number

Lot number (YYYY-MM-DD; year-month-day)

Unique Device Identifier

Caution

General warning sign

Temperature limitation for operating **or** storage

Humidity limitation for operating **and** storage

Atmospheric pressure limitation

Read instructions for use before operating the device.

Patient information website

Reminder/Note

Not made with natural rubber latex

Cuff size and circumference

Acceptable arm circumference range (cuff folding crease shall fall within range)

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