microlife®

WatchBP® Cuff

 Microlife Corporation
9F, No. 431, RuiGuang Road, NeiHu
Taipei, 114, Taiwan, China
www.microlife.com

 Microlife UAB
P. Lukšio g. 32
08222 Vilnius
Lithuania
eurep@microlife.lt

 Microlife AG
Espanstrasse 139
9443 Widnau
Switzerland

WBP reusable cuff V32 2026-08-10
Issuing Date: July 2025

EN → 1	EE → 17	SL → 37	DE → 54
SV → 3	RU → 19	SR → 39	ES → 56
FI → 5	UA → 22	PL → 41	PT → 58
DA → 7	KZ → 25	HU → 44	GR → 60
NO → 9	BG → 28	HR → 46	TR → 63
IS → 11	RO → 31	NL → 48	HE → 65
LV → 13	CZ → 33	FR → 50	AR → 67
LT → 15	SK → 35	IT → 52	FA → 69

1. Introduction

Document scope

Read the instructions carefully before using the cuff.

Refer to this document and the instruction for use of your blood pressure monitor for important product operation and safety information about the cuff. Keep these documents for future reference.

2. Disclaimers

Microlife® is a registered trademark of Microlife Corporation.
WatchBP® is a registered trademark of Microlife Corporation.

3. Important information

Device description

This arm or ankle cuff is an accessory of oscillometric digital non-invasive blood pressure monitors.

WARNING

Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.

- Keep the cuff and its parts away from children and people incapable of using the device and cuff. Beware of the risks of accidental ingestion of small parts and of strangulation with the cuff tube.

CAUTION

Caution items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

- Use this cuff only with compatible devices and connectors from Microlife. Using non-compatible accessories may compromise the safety and performance of the device.
- Use only with Microlife WatchBP blood pressure monitors.
- Always use an arm cuff sized for the patient's mid-arm circumference or an ankle cuff sized for the patient's upper-ankle circumference.
Using a cuff of incorrect size may cause measurement error and/or incorrect readings.
- Stop using this device and cuff and consult with your doctor if you experience skin irritation or discomfort.
- DO NOT use this device, cuff, or parts after the expiration of its stated service life.

Adverse events and reporting

Please report any serious incident, injury or adverse event that has occurred in relation to the device to the manufacturer/ European authorized representative (EU REP), and to the competent authority.

4. Device information

Package contents

Microlife WatchBP Cuff

Cuff tube

Various connectors

Instruction Manual

Inspect the device, cuff and other parts for damage. DO NOT USE the device, cuff or parts if they appear damaged or operating abnormally.

5. Device Setup

Choosing the connector

Check your blood pressure monitor to select the compatible cuff connector correctly! Using an incompatible connector may cause air leakage, measurement error, incorrect readings and/or prevent proper assembly.

Additional information about selecting a compatible cuff connector for your Microlife WatchBP blood pressure monitor:
<http://www.microlife.com/watchbp-connectors>

A loose connection will result in inaccurate readings, and/or error messages.

Note: To change the connector, pull the connector out of the cuff tube and insert the required connector into the cuff tube as far as it will go.

To replace the cuff with a different size, detach it from the flying connector (if applicable) and attach the new cuff.

6. Maintenance

Additional information about maintenance and disposal of your WatchBP blood pressure cuff: www.microlife.com/watchbp-reusable-cuff

The bladder must lay straight in the cuff sleeve, not folded.

Do not use fabric softener.

Do not dry the cuff sleeve in a tumble dryer!

WARNING: Under no circumstances may you wash the inner bladder!

Cleaning and disinfecting

Cuffs with a side pocket that allows removal of the bladder for washing

Cleaning the cuff

- Remove the cuff connector from the cuff tube and carefully pull the bladder through the opening at the edge of the cuff sleeve.
- Fold and place the cuff sleeve inside a washing bag.
- Wash the cuff sleeve with warm water (43°C; 110°F) and a mild detergent in the washing machine.
- Completely air dry the cuff sleeve.
- Loop the cuff tube back through its opening and carefully place the bladder flat in the cuff sleeve.
- Reattach the cuff connector to the cuff tube.

Disinfecting the cuff

- Remove the cuff connector from the cuff tube and carefully pull the bladder through the opening at the edge of the cuff sleeve.
- Fold and place the cuff sleeve inside a washing bag.
- Soak the cuff sleeve in 75°C (167°F) hot water for 30 minutes.
- Completely air dry the cuff sleeve.
- Loop the cuff tube back through its opening and carefully place the bladder flat in the cuff sleeve.
- Reattach the cuff connector to the cuff tube.

Cuffs without a side pocket for removal of the bladder for washing

Cleaning and disinfecting the cuff

- Use a soft cloth with one of the following recommended cleaning solutions to wipe the cuff, cuff connector and the hose: With 70% ethyl or isopropyl alcohol.
- Do not immerse cuff directly.
- Completely air dry the cuff sleeve.

Storage

When not in use:

Keep the cuff in a dry, cool place away from sunlight, with ambient conditions within the temperature and humidity ranges described in the «Specifications and compliance» section.

 Do not bend or crease the cuff tube excessively.

7. Specifications and compliance

Technical specifications

 Technical specifications subjected to change without notice.

Reference number	Model number	Dimensions	Washability
WatchBP Home Cuff S	CA1-S0-D52	≤350 x ≤115 mm	non-washable
WatchBP Home Cuff M	CA1-M0-D51	≤495 x ≤150 mm	non-washable
WatchBP Home Cuff L	CA1-L0-D50	≤620 x ≤170 mm	non-washable
WatchBP Home Cuff L-XL	CA1-Z0-D49	≤690 x ≤170 mm	non-washable
WatchBP O3 Cuff S	CA0-C0-C69	≤350 x ≤125 mm	washable
WatchBP O3 Cotton Cuff S	CA0-C0-D62	≤350 x ≤125 mm	washable
WatchBP O3 Cuff M	CA1-M0-C68	≤500 x ≤155 mm	washable
WatchBP O3 Cotton Cuff M	CA1-M0-D61	≤500 x ≤155 mm	washable
WatchBP O3 Cuff L	CA1-L0-C67	≤620 x ≤175 mm	washable
WatchBP O3 Cotton Cuff L	CA1-L0-D60	≤620 x ≤175 mm	washable
WatchBP O3 Cuff L-XL	CA1-Z0-C66	≤755 x ≤175 mm	washable
WatchBP O3 Cotton Cuff L-XL	CA1-Z0-D59	≤755 x ≤175 mm	washable
WatchBP Office Cuff S	CA0-C0-D30	≤350 x ≤125 mm	washable
WatchBP Office Cuff M	CA0-M0-D29	≤540 x ≤150 mm	washable
WatchBP Office Cuff L	CA0-L0-D28	≤630 x ≤165 mm	washable
WatchBP Office Cuff L-XL	CA0-M0-D27	≤720 x ≤165 mm	washable
WatchBP Office Ankle Cuff M	CK2-M0-D32	≤620 x ≤150 mm	washable
WatchBP Office Ankle Cuff L	CK2-L0-D31	≤685 x ≤170 mm	washable

Product description: Cuff for digital non-invasive blood pressure monitor

Operating conditions:
10 – 40 °C / 50 – 104 °F
15 - 90 % relative maximum humidity
700 hPa – 1060 hPa

Storage and transport conditions:
-20 - +55 °C / -4 - +131 °F
15 - 90 % relative maximum humidity

Expected service life: 1 year or 5000 measurements, whichever comes first

Compliance information

This device complies with the requirements of the Medical Device Regulation (EU)2017/745.

8. Guarantee

The cuff has a functional guarantee (bladder tightness) for 1 year.

- Not included are sleeve and cuff enclosure due to wear.
- The guarantee is only valid on presentation of the purchase receipt.
- The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

9. Symbols and definitions

Medical device

CE Marking of Conformity

Importer

Authorized representative in the European Union

Authorized representative in Switzerland

Manufacturer

Country of manufacture
(Date of manufacture if date printed next to symbol)

Model number

Reference number

Lot number (YYYY-MM-DD; YYYY-MM-DD = Date of manufacture)

Unique Device Identifier

Caution

General warning sign

Temperature limitation for operating **or** storage

Humidity limitation for operating **and** storage

Atmospheric pressure limitation

Read instructions for use before operating the device.

Patient information website

Reminder/Note

Not made with natural rubber latex

Cuff size and circumference

Acceptable limb circumference range (cuff folding crease shall fall within range)

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