microlife®


WS150

Weight Scale

EN → 1	RU → 11	RO → 21	TR → 31
FR → 3	UA → 13	DE → 23	GR → 33
IT → 5	PL → 15	NL → 25	AR → 35
ES → 7	HU → 17	SV → 27	FA → 37
PT → 9	CZ → 19	FI → 29	


Preparation

Microlife WS 150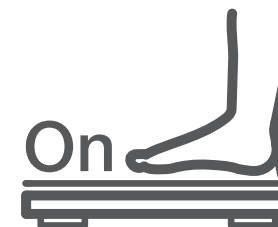

Easy Short Instructions

1. In order to get accurate measurements, place the scale on a firm level floor.
2. Step onto the scale with your feet parallel and weight evenly distributed. Stand still while the scale measures your weight. After a few seconds, the display will show your weight.
3. Do not lean on anything while being weighed, as this could affect the accuracy of the reading.
4. Step off the scale once the measurement is complete. The display will continue to show your weight for a few seconds, then turn off automatically.

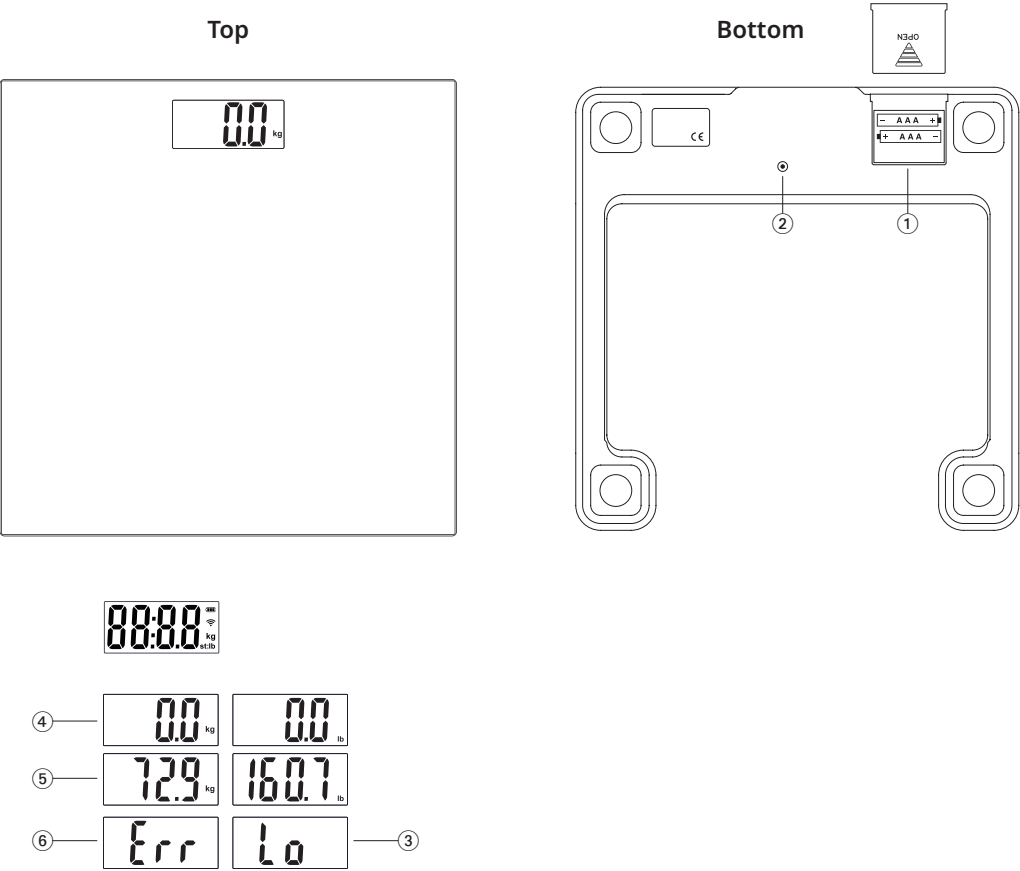Name of Purchaser / Nom de l'acheteur / Nome del rivenditore /
Nombre del comprador / Nome do comprador / Ф.И.О. покупателя
/ П.І.Б. покупця / Imię i nazwisko nabywcy / Vásárló neve /
Jméno kupujícího / Numele cumpărătorului / Name des Käufers
/ Naam koper / Inköparens namn / Ostajan nimi / Alicinin adi /
Όνοματεπώνυμο αγοραστή / اسم المشتري / نام خریدار

Serial Number / Numéro de série / Numero di serie / Número de
serie / Número de série / Серийный номер / Серійний номер
/ Numer seryjny / Sorozatszám / Výrobní číslo / Număr de serie
/ Serienummer / Serienummer / Sarjanumero / Seri numarası /
Αριθμός σειράς / رقم التسلسل / مدل

Date of Purchase / Date d'achat / Data d'acquisto / Fecha de
compra / Data da compra / Дата покупки / Дата покупки / Data
zakupu / Vásárlás dátuma / Datum nákupu / Data cumpărării /
Kaufdatum / Datum van aankoop / Inköpsdatum / Ostapäivämäärä
/ Satın alma tarihi / Ημερομηνία αγοράς / تاريخ الشراء
/ شماره سریال

Specialist Dealer / Revendeur / Categoria rivenditore / Distribuidor
especializado / Revendedor autorizado / Специализированный
дилер / Специализований дилер / Przedstawiciel / Forgalmazó /
Specializovaný dealer / Distributor de specialitate / Fachhändler
/ Specialist Dealer / Aterförsäljare / Alan kauppias / Uzman
satici / Εξειδικευμένος αντιπρόσωπος / التاجر المختص
/ تاريخ خرید

- ① Battery compartment
- ② KG / LB / ST Switch

Display

- ③ Low battery indication
- ④ Zero Setting
- ⑤ Weight Reading
- ⑥ Error (Err)

Dear Customer,

This new electronic personal scale allows you to evaluate your body weight. Capacity 180 kilos, divided into 100 grams. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your Microlife product. If you have any questions or problems, please contact Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the Internet at www.microlife.com where you will find a wealth of invaluable information on our products.

Stay healthy – Microlife Corporation!

Read the instructions carefully before using this device.

1. Suggestions for Use

- Place the scale on a flat, hard surface. Soft, inaccurate flooring (e.g. rugs, carpets, linoleum) can cause the scale to give uneven reading of your weight.
- Step onto the scale with your feet parallel and weight equally distributed.
- Stand still while the scale measures your weight.
- Always weigh yourself on the same scale each day at the same time, preferably undressed and before breakfast. To get the best results from your scale, weigh yourself twice, and if the two weights are different from each other, your weight is between the two readings.
- The surface of the scale can be slippery, if wet.

2. Using the device for the first time

How to insert the battery

This scale operates with two AAA batteries. When the battery is dead the display shows «LO» ③ or the battery symbol. Insert two new AAA batteries into the compartment ①, ensuring the poles match the (+) and (-) symbols as indicated.

KG / LB / ST Switch

Located at the bottom of the scale, this switch ② allows you to choose the unit of measurement for weighing: kg, lb or st. Select your desired unit of measurement, always when the scale is turned on and «0.0» is displayed (otherwise the unit of measure cannot be changed).

Use as simple scale

1. In order to get accurate measurements, place the scale on a firm level floor.
2. Step onto the scale and wait without moving; after a few seconds, the display screen will show your weight ⑤. Do not lean on anything while being weighed as this could alter the reading.
3. Step off the scale, the display screen will continue to show your weight for a few seconds then it will turn off automatically.
4. If the «Err» symbol is displayed, it means the weight limit has been exceeded. Please step off the scale immediately to prevent damage to the product. The maximum load capacity is 3 – 180 kg.

3. Safety, care, accuracy test and disposal

Safety and protection

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures

- impact and dropping
- contamination and dust
- direct sunlight
- heat and cold
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- If the device will not be used for a prolonged period, remove the batteries and ensure nothing is left on the scale.
- Do not mix old and new batteries.
- Do not mix different types of batteries (e.g., alkaline, standard carbon-zinc, or rechargeable nickel-cadmium).
- Read the additional safety information provided within the individual sections of this instruction manual.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Device care

- Clean the device only with a soft, dry cloth.
- Never immerse the device in water or rinse it under running water.

Disposal

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

4. Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Batteries.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

5. Technical specifications

- Equipped with high-precision strain-gauge sensors
- Capacity: 180 kg / 396 lb / 28.8 st
- Resolution: 0.1 kg / 0.2 lb / 1/4 st
- Battery: 3 V (1.5 V AAA x 2)
- Display: LCD
- Functions:
 - Automatic zero resetting
 - Automatic switch-off
 - Overload indication
 - Low battery indication
- Technical alterations reserved.

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