
MT 850

IB MT 850 EN-PT-ES-AR 2026-04-06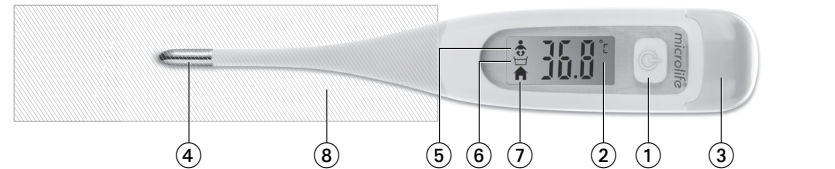

Description of this thermometer

- 1 Multifunction button
- 2 Display
- 3 Battery compartment cover
- 4 Measuring sensor
- 5 Body mode
- 6 Water bath mode
- 7 Ambient mode
- 8 Cleaning and disinfecting area (thermometer probe only)

Optional accessories

Single use probe cover for hygienic measurement: Microlife probe covers for digital thermometer are available as an accessory. Use probe covers compatible with the device for measurements to avoid cross contamination between measurements and patients. Using a non-compatible probe cover with the device may affect measurement accuracy. Optional probe cover installation: Insert the thermometer thoroughly into the probe cover and remove the probe cover protection layer. After measurement completed, remove used probe cover and dispose it in accordance with applicable local regulations.

Caution: Do not re-use the probe cover. The probe cover is intended only for single-use to prevent cross-contamination.

Device installation and setup

A battery is pre-installed in the device. When the low battery symbol appears in the display, the battery is flat and needs replacing. Remove the battery compartment cover (3) from the thermometer by using proper instrument. Replace the new battery with the + at the top.

NOTE: Make sure you have a new battery on hand. For the correct battery type, please refer to the specifications. Keep the battery away from children/infants.

This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

Important information

Device description: A digital thermometer is a medical device that utilizes the principles of conductive thermal equivalent between device-body contact and digital signal processing to compute and provide a body temperature measurement.

Intended purposes:

The device is intended to measure body temperature.

Intended user:

The device is intended to be operated by adults and adolescents with adequate vision, motor functions, and education, capable of understanding the instructions for use and operating general household electrical appliances.

Intended patient:

The intended patients are the general population of all ages.

Intended use environment and conditions:

The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a care giver, and in a professional healthcare facility (e.g. physician office).

Indications:

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contra-indications:

- The device is not suitable for use on sites with any breached or compromised skin.
- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!

NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.

- This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be held liable for damage caused by incorrect use.
- The measurement result of this device is not a medical diagnosis and not intended to substitute consultation and diagnosis by a qualified professional healthcare provider (e.g., physician, pharmacist, or other licensed health-care professionals). Do not use this device for self-diagnosis or for self-treatment of a medical condition. Seek advice from a health-care professional immediately if the patient is clearly unwell and/or having physiological or medical symptoms.
- This product contains a button cell battery. A swallowed button cell battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children/infants. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.
- Do not attempt to measure temperature of the body sites that are not specified in this instruction.
- Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed.
- Do not walk or run when taking temperature.
- For safety reasons (risk of rectal perforation) rectal measurement in children younger than 3 years must be performed only by trained health care personnel (professional user). Use another measurement method instead. For rectal fever measurement in small children younger than 3 years, thermometers with a flexible probe are available.
- Professional users must follow the country specific applicable medical devices ordinance.

CAUTION

NOTE: Caution items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

- If the patient temperature is influenced (e.g. incubator, heating blanket) the measurement result provided by this device shall not be used to determine the presence/absence of fever.

Device maintenance and disposal

Cleaning and disinfecting

Cleaning the thermometer is recommended before and after use. (Remove the probe cover before cleaning and disinfection, if applicable) To clean and disinfect the thermometer:

1. Select swab or cotton tissue and moisten it with 70% Isopropyl alcohol.
2. Wipe the surface of the device from the end of the thermometer probe (6) towards the measuring sensor (4) on both front and back side thrice per swab. Repeat this step to both clean and disinfect the thermometer (use at least two swabs).

Caution: Avoid wiping and immersion of the display area to protect it from fading.

NOTE: Consider the application and safety instruction of the disinfectant manufacturer.

Caution: Do not boil, clean in dishwasher or clean in autoclave units. Other cleaning agents or cleaning methods may cause malfunction or damage to the product and will void the warranty.

Storage

When not in use:

Keep the device and accessories in a dry, cool place away from sunlight, with ambient conditions within the temperature and humidity ranges described in the «Technical specifications» section.

Warning: Storing the device **unused** for an extended period without removing batteries increases the chance of battery fluid leakage, which may lead to device damage and skin irritation when in contact. If your eye or skin is exposed to battery fluid, wash the exposed part immediately with ample clean water. Consult a doctor if irritation or discomfort persists.

Calibration and support

The device is calibrated during manufacturing. In general, it is recommended to have the device verified by your local designated Microlife device distributor every two years, or after mechanical impact, liquid ingress, and/or device malfunctions. For questions related to device measurement accuracy, please contact your local designated Microlife device distributor.

Caution: Do not attempt to service or calibrate the device and accessories yourself.

Disposal

This device is medical electrical equipment. Dispose this device and batteries in accordance with the Waste Electrical and Electronic Equipment (WEEE) directive and applicable local regulations. Do not dispose of the device and batteries with domestic or commercial waste.

Guarantee

This device is covered by a **5 year guarantee** from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

- Transport costs and risks of transport.
 - Damage caused by incorrect application or non-compliance with the instructions for use.
 - Damage caused by leaking batteries.
 - Damage caused by accident or misuse.
 - Packaging/storage material and instructions for use.
 - Regular checks and maintenance (calibration).
 - Accessories and wearing parts: Battery.
- Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: www.microlife.com/support.

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

Specifications and compliance

Technical specifications

Device name: Digital thermometer
Catalogue number: MT 850
Model number: MT1PG1
Operating mode: Adjusted mode
Measurement site: Sublingual, rectal or axillary
Reference body site: Sublingual, rectal or axillary
Measurement range: Body mode: 34.0 – 43.9 °C
Temp. < 34.0 °C: display «--» for low (too low)
Temp. > 43.9 °C: display «H» for high (too high)
Water bath mode: 5 – 50 °C
Temp. < 5 °C: display «L» for low (too low)
Temp. > 50 °C: display «H» for high (too high)
Ambient mode: -10 – +50 °C
Temp. < -10 °C: display «ERR» for low (too low)
Temp. > 50 °C: display «H» for high (too high)

Measurement accuracy (Laboratory): Body mode: ± 0.1 °C between 34 – 43.9 °C
Water bath mode: ± 1 °C between 5 – 50 °C
Ambient mode: ± 1 °C between -10 – +50 °C
Clinical accuracy: Clinical bias: 0.08 °C
Limits of agreement: 0.54 °C
Clinical repeatability: 0.21 °C
10 – 40 °C:
15 – 90 % relative maximum humidity
700 – 1060 hPa Atmospheric pressure
-20 – +55 °C:
15 – 90 % relative maximum humidity
700 – 1060 hPa Atmospheric pressure
3V; CR1632
approx. 2000 measurements (using a new battery)

Ingress protection (IP) rating: IP67: Dust-tight. Protected against the effects of temporary immersion in water.

Dimensions: 126x0.5 x 18.6x0.2 x 11.2x0.2 mm

Weight: Approx. 11x0.5 g (including batteries)

Expected service life: 5 years or 10000 measurements, whichever comes first

Compliance information

This device complies with the requirements of the Medical Device Regulation (EU)2017/745.

Compliant standards: EN 60601-1; EN 60601-1-2; EN 60601-1-11; EN ISO 80601-2-56

NOTE: Technical alterations reserved.

Symbols and definitions

- Read the instructions carefully before using this device.
- Keep out of reach of children
- Type BF applied part
- Caution
- Medical device
- Catalogue number
- Model number
- Lot number (YYYY-MM-DD; YYYY-MM-DD = Date of manufacture)
- Manufacturer
- Country of manufacture (Date of manufacture if date printed next to symbol)
- CE Marking of Conformity
- Authorized representative in the European Union
- Temperature limitation for operating or storage
- Humidity limitation for operating and storage
- Dispose in accordance with waste electrical and electronic equipment (WEEE) directive.
- Importer
- Unique Device Identifier
- Keep away from sunlight
- Single use only
- Patient information website
- Not made with natural rubber latex
- Keep dry

IP67 IP67: Dust-tight. Protected against the effects of temporary immersion in water.

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