
MT 500

IB MT 500 EN-DE-FR-IT 2026-04-06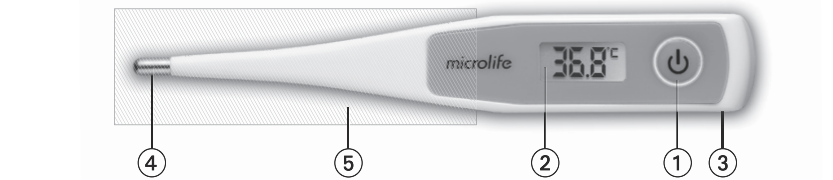

Description of this thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor
- ⑤ Cleaning and disinfecting area (thermometer probe only)

Optional accessories

Single use probe cover for hygienic measurement: Microlife probe covers for digital thermometer are available as an accessory. Use probe covers compatible with the device for measurements to avoid cross contamination between measurements and patients. Using a non-compatible probe cover with the device may affect measurement accuracy.

Optional probe cover installation: Insert the thermometer thoroughly into the probe cover and remove the probe cover protection layer. After measurement completed, remove used probe cover and dispose it in accordance with applicable local regulations.

Caution: Do not re-use the probe cover. The probe cover is intended only for single-use to prevent cross-contamination.

Device installation and setup

A battery is pre-installed in the device. When the low battery symbol **▼** appears in the display, the battery is flat and needs replacing. Remove the battery compartment cover ③ from the thermometer by using proper instrument. Replace the new battery with the + at the top.

NOTE: Make sure you have a new battery on hand. For the correct battery type, please refer to the specifications. Keep the battery away from children/infants.

This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.

Important information

Device description

A digital thermometer is a medical device that utilizes the principles of conductive thermal equivalent between device-body contact and digital signal processing to compute and provide a body temperature measurement.

Intended purposes

The device is intended to measure body temperature.

Intended user

The device is intended to be operated by adults and adolescents with adequate vision, motor functions, and education, capable of understanding the instructions for use and operating general household electrical appliances.

Intended patient

The intended patients are the general population of all ages.

Intended use environment and conditions

The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a care giver, and in a professional healthcare facility (e.g. physician office).

Indications

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications

- The device is not suitable for use on sites with any breached or compromised skin.
- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!

NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.

- This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be held liable for damage caused by incorrect use.
- The measurement result of this device is not a medical diagnosis and not intended to substitute consultation and diagnosis by a qualified professional healthcare provider (e.g., physician, pharmacist, or other licensed health-care professionals). Do not use this device for self-diagnosis or for self-treatment of a medical condition. Seek advice from a healthcare professional immediately if the patient is clearly unwell and/or having physiological or medical symptoms.

- This product contains a button cell battery. A swallowed button cell battery can cause internal chemical burns in as little as two hours and lead to death. Dispose of used batteries immediately. Keep new and used batteries away from children/infants. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.

- Do not attempt to measure temperature of the body sites that are not specified in this instruction.
- Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed.
- Do not walk or run when taking temperature.
- For safety reasons (risk of rectal perforation) rectal measurement in children younger than 3 years must be performed only by trained health care personnel (professional user). Use another measurement method instead. For rectal fever measurement in small children younger than 3 years, thermometers with a flexible probe are available.
- Professional users must follow the country specific applicable medical devices ordinance.

CAUTION

NOTE: Caution items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

- If the patient temperature is influenced (e.g. incubator, heating blanket) the measurement result provided by this device shall not be used to determine the presence/absence of fever.

- Use of compatible probe cover, or cleaning and/or disinfection of device is recommended before / after measurement to avoid cross contamination. Refer to «Cleaning and disinfection» section for details.

- Do not bite, chew, or bend the device probe excessively.
- Do not use this device if you think it is damaged or notice anything unusual.

Electromagnetic Compatibility Information

- This device is compliant with EN 60601-1-2 Electromagnetic Disturbances standard.

Further documentation in compliance with this EMC standard is available from Microlife on www.microlife.com/electro-magnetic-compatibility.

- Do not use this device close to strong electromagnetic fields and portable radio frequency communication devices (for example microwave oven and mobile devices). Keep a minimum distance of 0.3 m from such devices when using this device.

- This device is not certified to be used in the vicinity of medical equipment including high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) and computerized tomography (CT) instruments.

Adverse events and reporting

Please report any serious incident that has occurred in relation to the device, injury or adverse event to the local competent authority and to the manufacturer or to the european authorised representative (EU REP).

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Battery.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: www.microlife.com/support. Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

Specifications and compliance

Technical specifications

Device name: Digital thermometer
Catalogue number: MT 500
Model number: MT1PM1
Operating mode: Direct mode
Measurement site: Sublingual, rectal or axillary
Measurement range: 32.0 °C to 43.9 °C
Temp. < 32.0 °C: display **←--→** for low (too low)
Temp. > 43.9 °C: display **«H»** for high (too high)

Measurement accuracy (Laboratory): ± 0.1 °C; 34 °C – 42 °C
± 0.2 °C; 32.0 – 33.9 °C and 42.1 – 43.9 °C

Operating conditions: 15 – 90 % relative maximum humidity
700 – 1060 hPa Atmospheric pressure
-20 – +55 °C

Storage and transport conditions: 15 – 90 % relative maximum humidity
700 – 1060 hPa Atmospheric pressure

Battery: LR41 (1.5V) / SR41 (1.55V)
Battery lifetime: approx. 4500 measurements (using a new battery)

Ingress protection (IP) rating: IP67: Dust-tight. Protected against the effects of temporary immersion in water.

Dimensions: 122.6±0.5 x 18.9±0.2 x 9.5±0.2 mm
Weight: Approx. 19.3±0.5 g (including batteries)

Expected service life: 5 years or 10000 measurements, whichever comes first

Compliance information

This device complies with the requirements of the Medical Device Regulation (EU)2017/745.

Compliant standards: EN 60601-1; EN 60601-1-2; EN 60601-1-11; EN ISO 80601-2-56

Symbols and definitions

Read the instructions carefully before using this device.

Keep out of reach of children

Type BF applied part

Caution

Medical device

Catalogue number

Model number

Lot number (YYYY-MM-DD; YYYY-MM-DD = Date of manufacture)

Manufacturer

Country of manufacture (Date of manufacture if date printed next to symbol)

CE Marking of Conformity

Authorized representative in the European Union

Authorized representative in Switzerland

Temperature limitation for operating or storage

Humidity limitation for operating and storage

Dispose in accordance with waste electrical and electronic equipment (WEEE) directive.

Importer

Unique Device Identifier

Keep away from sunlight

Single use only

Patient information website

Not made with natural rubber latex

Keep dry

IP67: Dust-tight. Protected against the effects of temporary immersion in water.

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