
MT 720

IB MT 720 5+V-B 2026-06-29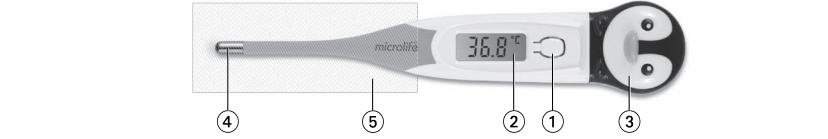

Description of this thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor
- ⑤ Cleaning and disinfecting area (thermometer probe only)

Optional accessories
Microprobe probe cover for hygienic measurement: Microprobe probe covers for digital thermometer are available as an accessory. Use probe covers compatible with the device for hygienic measurement to avoid cross contamination between measurements and patients. Using a non-compatible probe cover with the device may affect measurement accuracy.

Measuring methods / Normal body temperature
Insert the thermometer thoroughly into the probe cover and remove the probe cover protection layer. After measurement completed, remove used probe cover and dispose it in accordance with local disposal regulations.

Caution: Do not re-use the probe cover. The probe cover is intended only for single-use to prevent cross-contamination.

Device installation and setup

A battery is pre-installed in the device. When the low battery symbol ∇ appears in the display, the battery is flat and needs replacing. Remove the battery compartment cover (3) from the thermometer by using proper instrument. Replace the new battery with the + at the top.

NOTE: Make sure you have a new battery on hand. For the correct battery type, please refer to the specifications. Do not use old or damaged batteries. Do not use the thermometer by using proper instrument. Replace the new battery with the + at the top.

This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future use as a reference.

Important information

Device description
A digital thermometer is a medical device which utilizes the principles of conductive thermal equivalent between device-body contact and digital signal processing to compute and provide a digital temperature measurement.

Intended purposes
The device is intended to measure body temperature.

The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Intended patient
The intended patients are the general population of all ages.

Intended use environment and conditions
The device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications
The device is not suitable for use on sites with any breached or compromised skin.

- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!
NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in death, critical or serious injury to the user or patient.

This device may only be used for the purposes described in this instruction manual. The manufacturer cannot be held liable for damage caused by incorrect use.

The measurement result of this device is not a medical diagnosis. It is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications
The device is not suitable for use on sites with any breached or compromised skin.

- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!
NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

This device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications
The device is not suitable for use on sites with any breached or compromised skin.

- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!
NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

This device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications
The device is not suitable for use on sites with any breached or compromised skin.

- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!
NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

This device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Contraindications
The device is not suitable for use on sites with any breached or compromised skin.

- Do not attempt rectal measurements on persons with rectal disorders.

WARNING!
NOTE: Warning items indicate potentially hazardous situations, if not avoided, may result in minor or negligible injury to the user or patient, or damage to the property or environment.

This device is intended for use in a home healthcare environment (e.g. general household without medically trained personnel) by patients (e.g. for self-measurement) or by a caregiver, and in a professional healthcare facility (e.g. physician office).

This device measures body temperature for indications of:

- Screening for illness and conditions related to high or low body temperature.

Storage and transport conditions:	-20 ... +55 °C
Battery:	120 ... 90 % relative maximum humidity
Battery lifetime:	120 ... 90 % relative maximum humidity
Integress protection (IP):	IP22: Protection against solid foreign objects of 12.5 mm and greater. Protected against vertically falling water drops when enclosure tilted up to 15°
Dimensions:	126,6x44 x 24,8x2,1 x 19x2,2 mm
Weight:	Approx. 11,5 g (including batteries)
Expected service life:	5 years or 10000 measurements, whichever comes first

NOTE: Technical information reserved. **Compliant standards:** EN 60601-1; EN 60601-1-1; EN ISO 80601-2-56

Symbols and definitions

Read the instructions carefully before using this device.

Keep out of reach of children

Type BF applied part

Caution

Medical device

Catalogue number

Model number

Lot number (YYYY-MM-DD; YYYY-MM-DD = Date of manufacture)

Country of manufacture

Date of manufacture (if date printed next to symbol)

CE Marking of Conformity

Authorized representative in the European Union

Authorized representative in Switzerland

Temperature limitation for operating or storage

Humidity limitation for operating and storage

Disposal in accordance with waste electrical and electronic equipment (WEEE) directive.

Importer

Unique Device Identifier

Keep away from sunlight

Single use only

Patient information website

Not made with natural rubber latex

Keep dry

IP22: Protected against solid foreign objects of 12.5 mm Ø and greater. Protected against vertically falling water drops when enclosure tilted up to 15°

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