



## Description of this Thermometer

- ① ON/OFF button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor / measuring tip
- ⑤ Cleaning and disinfecting area (thermometer probe only)

EN

new battery with the + at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

## Technical Specifications

- Type: Maximum thermometer  
Measurement range: 32.0 °C to 42.9 °C  
Temp. < 32.0 °C: display „L“ for low (low low)  
Temp. > 42.9 °C: display „H“ for high (high high)

Measurement accuracy: ± 0.1 °C between 34 °C and 42 °C  
± 0.2 °C, 32.0 - 33.9 °C and 42.1 - 42.9 °C

Operating conditions: 10 - 40 °C, 15-95 % relative maximum humidity

Storage conditions: -25 - +60 °C, 15-95 % relative maximum humidity

Battery: LR41 (1.5V) / SR41 (1.55V)

Battery lifetime: approx. 2700 measurements (using a new battery)

IP Class: IP22

Reference to standards: EN 12470-3, clinical thermometers; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11

Expected service life: 5 years or 10000 measurements

The device complies with the requirements of the Medical Device Directive 93/42/EEC.

Technical alterations reserved.

## Guarantee

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

Protect this device from impact and dropping!

Avoid bending the device more than 45°!

Opening or altering the device invalidates the guarantee.

Use only the commercial disinfectants listed in the section Cleaning and Disinfecting.

To clean the device, avoid damage to the device.

We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact your local Microlife Service to arrange the test.

**WARNING:** The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only. Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Read the instructions carefully before using this device.

Type BF applied part

## Turning on the Thermometer

To turn on the thermometer, press the On/Off button (1); a short beep signal (thermometer On). A display test is performed. All sensors should be disinfected.

The last measurement reading will be shown on the display (2) automatically for 2 seconds with an „L“ icon and a flashing „C“ appear at the display field (2). The thermometer is now set at an ambient temperature of less than 32 °C, an „L“ and a flashing „C“ appear at the display field (2).

The thermometer is now ready for use.

Protect this device from impact and dropping!

Avoid bending the device more than 45°!

Opening or altering the device invalidates the guarantee.

Use only the commercial disinfectants listed in the section Cleaning and Disinfecting.

To clean the device, avoid damage to the device.

We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact your local Microlife Service to arrange the test.

**WARNING:** The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only. Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Read the instructions carefully before using this device.

Type BF applied part

## Technical characteristics

Type: Maximum thermometer

Dial range: 32.0 °C to 42.9 °C

Temp. &lt; 32.0 °C: displays "L" (low)

Temp. &gt; 42.9 °C: displays "H" (high)

Accuracy: ± 0.1 °C between 34 °C and 42 °C

± 0.2 °C, 32.0 - 33.9 °C and 42.1 - 42.9 °C

Operating conditions: 10 - 40 °C, 15-95 % relative maximum humidity

Storage conditions: -25 - +60 °C, 15-95 % relative maximum humidity

Battery: LR41 (1.5V) / SR41 (1.55V)

Battery life: approx. 2700 measurements (using a new battery)

IP class: IP22

Reference to standards: EN 12470-3, medical thermometers; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11

Expected service life: 5 years or 10000 measurements

The device complies with the requirements of the Medical Device Directive 93/42/EEC.

Technical alterations reserved.

## Guarantee

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

Use only the commercial disinfectants listed in the section Cleaning and Disinfecting.

To clean the device, avoid damage to the device.

We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact your local Microlife Service to arrange the test.

**WARNING:** The measurement result given by this device is not a diagnosis! Do not rely on the measurement result only. Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Read the instructions carefully before using this device.

Type BF applied part

## Measuring methods / Normal body temperature

In the armpit (axillary) | 34.7 - 37.3 °C

Wipe the underarm with a dry towel. Place the measuring sensor (4) on the armpit until the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait at least 5 minutes, regardless of the beeps sound.

In the mouth (oral) | 35.5 - 37.5 °C

Do not eat or drink anything hot or cold 10 minutes before the measurement. The thermometer should remain closed up to 2 minutes before starting a reading.

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor (4) must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

If this is not possible due to blocked airways, another method for measurement should be used.

Approx. measuring time: 10 seconds!

In the anus (rectal) | 36.6 - 38.0 °C

Carefully insert the measuring sensor (4) of the thermometer 2 to 3 cm into the anal aperture.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

Approx. measuring time: 10 seconds!

Cleaning and Disinfecting

For disinfection in home environment, use a 70% Isopropyl alcohol swab, or a cotton tissue moistened with 70% Isopropyl alcohol to wipe surface pollutants off the thermometer probe (note: consider the application and safety instruction of the disinfectant manufacturer). Always start wiping from the side of the thermometer probe (apex), towards the entire thermometer probe (see number 5 in the drawing). Afterwards the entire thermometer probe (see number 5 in the drawing) should be immersed in 70% Isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before next use. Avoid immersing or wiping the display to protect it from fading.

The thermometer is not intended for professional use.

Battery Replacement

When the ▼ symbol (up-side-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover (3) from the thermometer. Insert the

## Microlife®

microlife

