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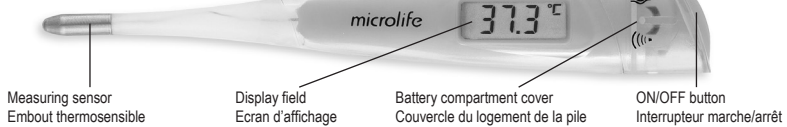
IMPORTANT SAFETY INSTRUCTIONS

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- WARNING:** The measurement result given by this device is not a diagnosis. Do not rely on the measurement result only.
- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.
- Protect the thermometer from hitting and striking!
- Avoid bending the tip more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- The manufacturer recommends verifying the accuracy by an authorised laboratory every 2 years.

EN

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button on the top; a short beep signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, «--» appears and a «°C» flashes at the upper right of the display field. The thermometer is now ready for use.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 8 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. About 2 seconds after release the button, this value will disappear and the thermometer will be turned off.

MEASURING METHODS / Normal body temperature

In the mouth (oral) / 35.5 - 37.5 °C

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air. Approx. measuring time: 20 seconds

In the anus (rectal) / 36.6 - 38.0 °C

This is the most reliable measuring method, and is especially suitable for infants and small children. Carefully insert the measuring sensor of the thermometer 2 - 3 cm into the anal aperture. Approx. measuring time: 10 seconds

In the armpit (axillary) / 34.7 - 37.3 °C

To receive more reliable results we recommend measuring temperature orally or rectally. Approx. measuring time: 20 - 90 seconds!

CLEANING AND DISINFECTING

Name: Isopropyl alcohol 70 %; immerse: max. 24 hours.

BATTERY REPLACEMENT

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions»

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This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

GUARANTEE

We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. The battery and packaging are excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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