<b>Name of disinfectant</b> <b>Name des Mittels</b> <b>Nom du désinfectant</b> <b>Naam van desinfectieoplossing</b>	Isopropyl alcohol 70 % Isopropyl Alkohol 70 % Alcool isopropylique 70 % Isopropyl alcohol 70 %	<b>Max. duration &amp; Method</b> <b>Max. Dauer &amp; Methode</b> <b>Méthode et durée max.</b> <b>Maximale onderdompeling</b>	Immerse for 24 hours 24 St. eintauchen Immersion pendant 24 h maximaal 24 uur
--	---	--	--

### MEASURING METHODS

#### In the anus (rectal)

Carefully insert the tip of the thermometer 2 to 3 cm into the anal aperture.  
Approx. measuring time: 10 seconds!

#### BATTERY REPLACEMENT

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions».

#### GUARANTEE

This device is covered by a lifetime guarantee from the date of purchase. During this guarantee period, at our discretion, MicroLife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee.

The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Battery. Should guarantee service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website:

#### www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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MicroLife AG