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IMPORTANT SAFETY INSTRUCTIONS

- The pacifier thermometer is suitable exclusively for oral measurement of the body temperature of children up to three years of age.
- The minimum measurement time until the signalling tone [beep] must be maintained without exception.
- The readings displayed by the pacifier thermometer may be too low under some conditions (e.g. if the child is crying or persistently breathing through its mouth). Therefore, in case of doubt, or if there are signs of fever (reddening of skin, sweating, agitation, unwillingness to drink), please **always perform a follow-up rectal measurement.**
- This thermometer is not suitable for use as a pacifier.
- Protect the thermometer from hitting and striking.
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer.
- The thermometer can/must not be opened! The batteries cannot be changed for safety reasons.
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- Warning:** The thermometer contains a long-life silver oxide battery (not replaceable). For disposal, please comply with local regulations for special waste.
- It is not necessary to calibrate the measurement accuracy within the limited 3-years battery life time.

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON.» A display test is performed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

The pacifier thermometer is suitable only for oral measurement of the body temperature of children, and it must not be used as a pacifier. Place the thermometer in the child's mouth like a normal baby pacifier. Be sure the child keeps the thermometer in his mouth during the measurement, and keep the child's mouth closed as much as possible. The measurement time signalled is normally about 5 minutes. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 32 seconds and that the thermometer is ready for reading.

Warning! Certain external factors can lead to a certain measurement inaccuracy in any measurement of body temperature, especially on small children. Therefore, please note the instruction given in the section «Safety instructions». After a measurement, the maximum temperature measured is shown on the display until the thermometer shuts off. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display.

CLEANING AND DISINFECTING

Name: Isopropyl alcohol 70 %; immerse: max. 24 hours.

BATTERY EXHAUSTION

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and thermometer needs to be replaced. For battery disposal please refer to «safety instructions».

Measuring sensor
Embout thermosensible

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This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

GUARANTEE

We grant you a lifetime guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the guarantee. Battery and packaging are also excluded from the guarantee. All other damage claims excluded. A guarantee claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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