

Guarantee Card

NC 100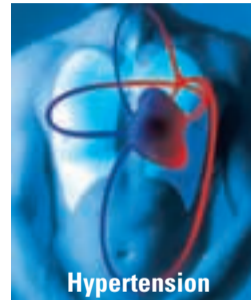


Hypertension

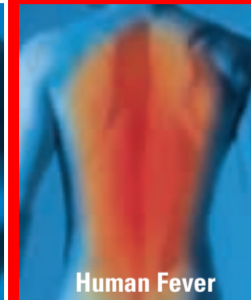Human Fever

Asthma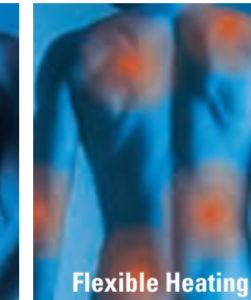

Flexible Heating

Europe / Middle-East / Africa

Microlife AG
Esenstrasse 139
9443 Widnau / Switzerland
Tel. +41 / 71 727 70 30
Fax +41 / 71 727 70 39
Email admin@microlife.ch
www.microlife.com

Asia

Microlife Corporation.
9F, 431, RuiGang Road, NeiHu
Taipei, 11492, Taiwan, R.O.C.
Tel. 886 2 8797-1288
Fax 886 2 8797-1283
Email service@microlife.com.tw
www.microlife.com

North / Central / South America

Microlife USA, Inc.
1617 Gulf to Bay Blvd., 2nd Floor Ste A
Clearwater, FL 33755 / USA
Tel. +1 727 442 5353
Fax +1 727 442 5377
Email msa@microlifeusa.com
www.microlife.com

CE0044

IB NC 100 EN-PL 2313


Microlife NC 100

EN → 2
PL → 8

microlife®



Microlife NC 100

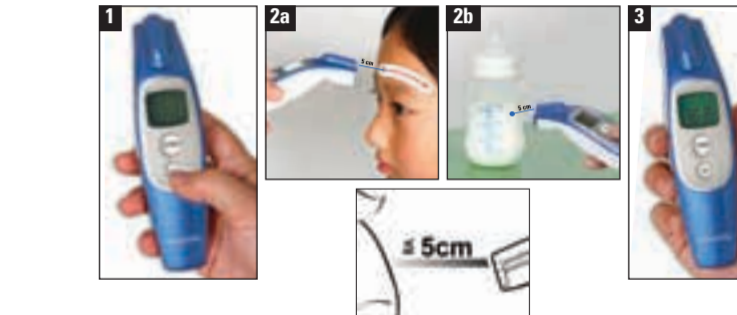microlife®

Microlife NC 100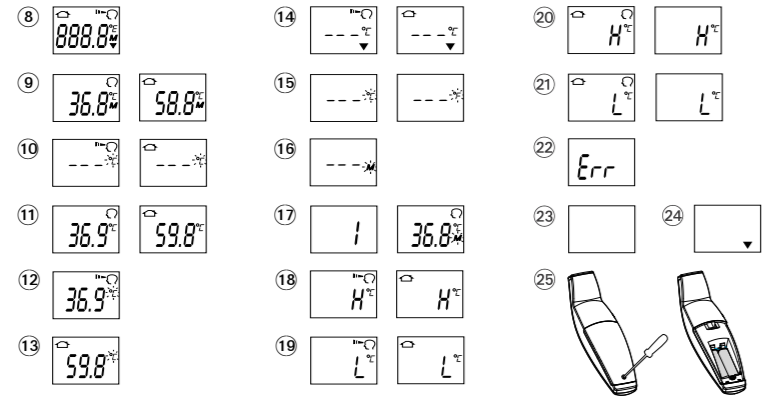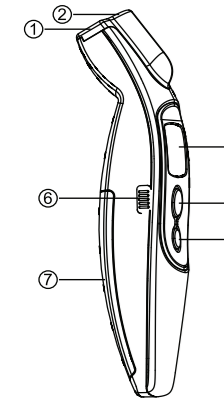


Microlife Non Contact Thermometer NC 100

EN

- ① Measuring sensor
- ② Tracking light
- ③ START button
- ④ Display
- ⑤ ON/OFF button
- ⑥ Mode switch
- ⑦ Battery compartment cover
- ⑧ All segments displayed
- ⑨ Memory
- ⑩ Ready for measuring
- ⑪ Measurement complete
- ⑫ Body mode
- ⑬ Object mode
- ⑭ Low battery indicator
- ⑮ Changing between Celsius and Fahrenheit
- ⑯ Recall mode
- ⑰ Recall the last 30 readings
- ⑱ Measured temperature too high

- ⑲ Measured temperature too low
- ⑳ Ambient temperature too high
- ㉑ Ambient temperature too low
- ㉒ Error function display
- ㉓ Blank display
- ㉔ Flat battery
- ㉕ Replacing the battery

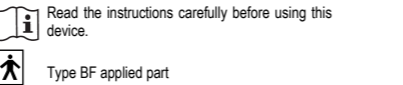This Microlife thermometer is a high quality product incorporating the latest technology and tested in accordance with international standards. With its unique technology, this device can provide a stable, heat-interference-free reading with each measurement. The device performs a self-test every time it is switched on to always guarantee the specified accuracy of any measurement.

This Microlife thermometer is intended for the periodic measurement and monitoring of human body temperature. **This thermometer has been clinically tested and proven to be safe and accurate when used in accordance to the operating instruction manual.**

Please read through these instructions carefully in order for you to understand all functions and safety information.

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vated blue tracking light will indicate the area being measured. After 3 seconds a long beep will verify the completion of measurement. If the temple area hasn't been reached before the long beep is heard, repeat the measurement as described above, but move the thermometer a little faster.

6. Read the recorded temperature from the LCD display.

Measuring in object mode

1. Follow steps 1-3 above, then aim the thermometer at the center of the object you want to measure with a distance of no more than 5 cm. Press the START button (3). After 3 seconds a long beep will verify the completion of measurement.

2. Read the recorded temperature from the LCD display.

NOTE:

- **Patients and thermometer should stay in similar room condition for at least 30 minutes.**
- Don't take a measurement while or immediately after nursing a baby.
- Don't use the thermometer in high humidity environments.
- Patients should not drink, eat or exercise before/while taking the measurement.
- Don't move the measurement device from the measuring area before hearing the termination beep.

- Use an alcohol swab to carefully clean the probe and wait for 15 minutes before taking a measurement on another patient.
- 10 short beeps and a red LCD backlight alerts the patient that he/she may have a temperature equal to or higher than 37.5 °C.
- Always take the temperature in the same location, since temperature readings may vary according to locations.
- Doctors recommend rectal measurement for newborn infants within the first 6 months, as all other measuring methods might lead to ambiguous results. If using a non contact thermometer on those infants, we always recommend verifying the readings with a rectal measurement.
- In the following situations it is recommended that three temperatures are taken with the highest one taken as the reading:
 1. Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
 2. When the user is learning how to use the thermometer for the first time until he/she has familiarized himself/herself with the device and obtains consistent readings.
 3. If the measurement is surprisingly low.
- **Readings from different measuring sites should not be compared as the normal body temperature varies**

by measuring site and time of day, being highest in the evening and lowest about one hour before waking up.

Normal body temperature ranges:

- Axillar: 34.7 - 37.3 °C / 94.5 - 99.1 °F
- Oral: 35.5 - 37.5 °C / 95.9 - 99.5 °F
- Rectal: 36.6 - 38.0 °C / 97.9 - 100.4 °F
- Microlife NC 100: 35.4 - 37.4 °C / 95.7 - 99.3 °F

7. Changing between Celsius and Fahrenheit

This thermometer can display temperature readings in either Fahrenheit or Celsius. To switch the display between °C and °F, simply turn OFF the unit, **press and hold** the START button (3) for 5 seconds; after 5 seconds, the current measurement scale («°C» or «°F» icon) will flash on the display (15). Change the measurement scale between °C and °F by pressing the START button (3). When the measurement scale has been chosen, wait for 5 seconds and the unit will automatically enter the «ready for measuring» mode.

8. How to recall 30 readings in Memory Mode

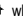
This thermometer can recall the last 30 readings.

- **Recall mode (16):** Press the START button (3) to enter Recall mode when power is off. The memory icon «M» will flash.

- **Reading 1 - the last reading** ⑰: Press and release the START button ③ to recall the last reading. Display 1 alone with memory icon.
- **Reading 30 - readings in succession:** Press and release the START button ③ consecutively to recall the last 30 readings in succession.

Pressing and releasing the START button ③ after the last 30 readings have been recalled will resume the above sequence from reading 1.

9. Error Messages

- **Measured temperature too high** ⑱: Displays «H» when measured temperature is higher than 42.2 °C / 108.0 °F in body mode or 100 °C / 212 °F in object mode.
- **Measured temperature too low** ⑲: Displays «L» when measured temperature is lower than 34.0 °C / 93.2 °F in body mode or 0 °C / 32 °F in object mode.
- **Ambient temperature too high** ⑳: Displays «H» and  when ambient temperature is higher than 40.0 °C / 104.0 °F.
- **Ambient temperature too low** ㉑: Displays «L» and  when ambient temperature is lower than 16.0 °C / 60.8 °F in body mode or lower than 5.0 °C / 41.0 °F in object mode.

- **Error function display** ㉒: The system has a malfunction.
- **Blank display** ㉓: Check if the batteries have been inserted correctly. Also check polarity (<+> and <->) of the batteries.
- **Flat battery indicator** ㉔: If only «▼» icon is shown on the display, the batteries should be replaced immediately.

10. Cleaning and Disinfecting

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer casing and the measuring sensor. Ensure that no liquid enters the interior of the device. Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the device in water or other cleaning liquids. Take care not to scratch the surface of the sensor lens and the display.

11. Battery Replacement

This device is supplied with 2 new, long-life 1.5V, size AAA batteries. Batteries need replacing when this icon «▼» ㉔ is the only symbol shown on the display.

Use a screwdriver to open the battery cover ㉕. Replace the batteries – ensure correct polarity as shown by the symbols in the compartment.

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

12. Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers the device. Batteries and packaging are not included.
 - Opening or altering the device invalidates the guarantee.
 - The guarantee does not cover damage caused by improper handling, discharged batteries, accidents or non-compliance with the operating instructions.
- Please contact Microlife-service.

13. Technical Specifications

Type:	Non Contact Thermometer NC 100
Measurement range:	Body mode: 34.0-42.2 °C / 93.2-108.0 °F Object mode: 0-100.0 °C / 32-212.0 °F
Resolution:	0.1 °C / °F

- Measurement accuracy:** Laboratory:
 $\pm 0.2\text{ }^{\circ}\text{C}$, $36.0\text{ }^{\circ}\text{C} \sim 39.0\text{ }^{\circ}\text{C}$
($\pm 0.4\text{ }^{\circ}\text{F}$, $96.8\text{ }^{\circ}\text{F} \sim 102.2\text{ }^{\circ}\text{F}$)
- Display:** Liquid Crystal Display, 4 digits plus special icons
- Acoustic:** The unit is turned ON and ready for the measurement: 1 short beep
Complete the measurement: 1 long beep (1 sec.) if the reading is less than $37.5\text{ }^{\circ}\text{C} / 99.5\text{ }^{\circ}\text{F}$, 10 short «beep» sounds, if the reading is equal to or greater than $37.5\text{ }^{\circ}\text{C} / 99.5\text{ }^{\circ}\text{F}$.
System error or malfunction: 3 short «bi» sounds
- Memory:** Auto-Display the last measured temperature
30 readings recall in the memory mode

- Backlight:** The display light will be GREEN for 4 seconds, when the unit is turned ON.
The display light will be GREEN for 5 seconds, when a measurement is completed with a reading less than $37.5\text{ }^{\circ}\text{C} / 99.5\text{ }^{\circ}\text{F}$.
The display light will be RED for 5 seconds, when a measurement is completed with a reading equal to or higher than $37.5\text{ }^{\circ}\text{C} / 99.5\text{ }^{\circ}\text{F}$.
- Operating temperature:** Body mode: $16\text{--}40.0\text{ }^{\circ}\text{C} / 60.8\text{--}104.0\text{ }^{\circ}\text{F}$
Object mode: $5\text{--}40.0\text{ }^{\circ}\text{C} / 41\text{--}104.0\text{ }^{\circ}\text{F}$
- Storage temperature:** $-20\text{--}+50\text{ }^{\circ}\text{C} / -4\text{--}+122\text{ }^{\circ}\text{F}$
15-95 % relative maximum humidity
- Automatic Switch-off:** Approx. 1 minute after last measurement has been taken.
- Battery:** 2 x 1.5 V Batteries; size AAA
- Dimensions:** 150 x 40 x 39 mm
- Weight:** 81 g (with batteries), 56 g (w/o batteries)
- Reference to standards:** ASTM E1965;
IEC 60601-1; IEC 60601-1-2 (EMC)

This device complies with the requirements of the Medical Device Directive 93/42/EEC.
Technical alterations reserved.
According to the Medical Product User Act a biennial technical inspection is recommended for professional users.
Please observe the applicable disposal regulations.

14. www.microlife.com

Detailed user information about our thermometers and blood pressure monitors as well as services can be found at www.microlife.com.

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