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microlife®

microlife®

Important Information - Retain for future use!

EN

- **Please read this manual carefully before using this device.**
- **This device is not intended for medical use in hospitals: For household use only.**
- **To avoid any risk of electric shock when using the pad please observe the following:**
 - Do not use if wet. If the pad has become damp or wet, completely dry the pad before use.
 - Only use the pad in dry rooms, not e.g. in the bathroom.
 - Check that the pad is undamaged each time before use, i.e. do not use it if there is visible damage to the hand control, switches, plug or the pad itself.
 - Do not insert pins or other sharp objects.
- **To avoid overheating:**
 - Do not leave the pad switched on unsupervised; switch it off when you leave the room.
 - Do not use the pad while folded.
 - Avoid creasing the pad.
 - **Do not cover the hand control!** To ensure that the hand control functions correctly, do not place it on or under the heating pad or cover it with other objects.
 - The cable should not be wound around the hand control.
- **Do not use on a person incapable of movement, a baby, a person insensitive to heat, or animals.** Do not let children play with the device.
- **Ensure that children do not use this device unsupervised.** Children under 8 years of age must not use this device unless a parent or guardian has set the control to the lowest temperature setting 1.
- This device may be used by children aged 8 or over and by people who have reduced physical, sensory or mental abilities or inadequate experience and/or knowledge, provided they are supervised or have been instructed on how to use the device safely and have understood the resulting dangers.
- **Excessive application at a high setting can result in skin burns.**
- **When the device is likely to be used for an extended period, i.e. by the user falling asleep,** controls should be adjusted to a setting recommended for continuous use.
- **Examine the device frequently for signs of wear or damage.** If there are signs of wear or damage or the device has been misused, please contact the manufacturer or authorised agent prior to any further use.
- **If the power cable is damaged, it must be replaced by the manufacturer, an authorised agent or similarly qualified persons in order to avoid any hazards.**
- **Do not attempt to open or repair the device yourself.** Special tools are required to re-assemble the device.
- **This device emits very weak electrical and magnetic fields which may disturb the function of your cardiac pacemaker.** Therefore, we recommend you to consult your physician and the manufacturer of your cardiac pacemaker before using this device.
- **Allow the device to cool down before folding and storing it in a dry place.** Do not place heavy objects on it to prevent buckling.

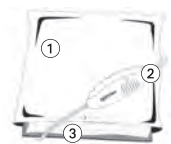
⚠

Not suitable for children between 0-3 years! Children under 3 years of age are unable to respond to overheating.

⌚

Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

1. How to Use a Microlife Heating Pad

①

Soft fabric cover – removable and machine washable. Fastener.

②

Hand control with slider for heat settings and ON indicator.

③

Pad

- ⚠

WARNING: Never use the pad without the cloth cover. Allow some air circulation between the pad and skin during use. The device is not to be used for warming animals.
1.

Put the cloth cover ① on a flat surface. Open the fastener.

2.

Insert the pad ③ completely into the cover, fitting one side at a time. Make sure the cable and hand control ② remain outside the cover.
3.

The pad will lie flat, filling the entire cover. Close fastener.
4.

Plug into a 220-240 volt AC mains socket and set the hand control to the desired heat setting using the slider.
 - Select the temperature setting on the slider – this switches on the heat pad. An indicator will illuminate, showing that the appliance is connected to the power supply and is switched on.
 - Setting 1 is recommended for continuous use.
 - The pad is switched off by moving the slider to the «0» position. The indicator light will then turn off.
5.

The hand control will automatically turn off the heater after approximately 90 minutes, then the indicator light will flash until reset or turn off. To reset the timer and start a further heating period, move the slider to the off «0» position and then back to the desired heat setting.
6.

Unplug when finished.

2. Cleaning Instructions

⚠

WARNING: Always unplug the power cable from the mains socket before undertaking any cleaning of the product.

1.

The cover can be machine washed after removal from the pad according to the washing instructions given on the cover.

2.

The pad must **not** be machine washed or dry-cleaned. The FH 80 & FH 100 (PVC surface) can be cleaned by using a soft damp cloth. The FH 200, FH 200C, FH 300, FH 310 & FH 320 (cotton or velvet surface) can be cleaned with a brush when dry.

3.

Do not allow the hand control to get wet. Do not use any detergents or solvents as they may damage the device.

4.

Never hold the hand control under water. Otherwise moisture may enter the hand control and cause damage.

5.

Check the hand control for signs of damage. Dry the pad and its fabric cover completely before switching it on again.

🧤

Cleaning and user maintenance shall not be made by children without supervision.

3. Explanation of Symbols

Wash at 40 °C!

Do not iron!

Do not wash!

Do not dry-clean!

Bleaching not permitted!

Tumble-dry!

Iron at low temperature!

Do not dry in the tumble dryer!

4. Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. During this guarantee period, at our discretion, Microlife will repair or replace the defective product free of charge. Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by leaking batteries.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Textile cover.

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local Microlife service. You may contact your local Microlife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the

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microlife®

original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

5. Technical Specifications

Model/Type: FH 80/A100, FH 100/A100, FH 200/B200,
FH 200C/B200, FH 300/B100, FH 310/C100,
FH 320/C100
Voltage: 220 - 240 V 
Power: 100 W
Frequency: 50 Hz

Technical alterations reserved.

6. www.microlife.com

Detailed user information about our products and about our services can be found at www.microlife.com.

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